



DenSureFit Snug Up Application Guide (5ml Kit)

Kit Includes:

- 1 x 5ml DenSureFit Silicone Cartridge
 - 3 x Mixing Tips
 - Instructional Card
-

Step-by-Step Instructions

For upper and lower snug ups, always complete the upper application first, then proceed with the lower. When performing a lower-only snug up, ensure upper denture or partial is securely in place on the gumline before starting.

STEP 1: Clean, Rinse, and Dry Your Denture

- Use Dawn Dish Soap or a similar soap that is free of moisturizers and conditioners.
- Rinse thoroughly.
- Dry completely.

STEP 2: Apply Silicone

- Attach a new mixing tip to the cartridge..
- *For Uppers:*
 - Express a small amount of silicone to the middle of the inner wall of the denture, just above the trough.

For Lowers:

- Express a small amount of silicone to the middle of both interior walls of the denture.
- Use only what's needed to fill new voids from gum shrinkage.

STEP 3: Seat Your Denture

For upper and lower snug ups, always complete the upper application first, then proceed with the lower. When performing a lower-only snug up, ensure upper denture or partial is securely in place on the gumline before starting.

- *For Uppers:*
 - Place directly into your mouth and press upward into your suction spot.
 - Suck your thumb for 1 minute.
 - Continue pressing upward with medium pressure for 3 more minutes (4 minutes total).
- *For Lowers:*
 - Align denture on your lower gum ridge.
 - Bite down gently in your natural bite position.
 - Check your bite in the mirror—upper and lower front teeth should align.
 - Bite down with medium pressure for 4 minutes.

STEP 4: Assess the Fit

- Remove and test suction/snugness.
- If the denture feels secure, you're done.
- If not, repeat with a small amount of additional silicone. Avoid overbuilding.

STEP 5: Trim the Excess

- Use small cuticle scissors to trim any loose, hanging, or uncomfortable areas of silicone.
 - Do NOT trim flashing that feels comfortable—it may help with your seal.
-

Important Storage Tip:

- Leave the used mixing tip ON the cartridge between uses. Silicone will cure inside the tip and seal the cartridge, keeping the rest of the material fresh.
-

When to Call Customer Support:

- If your denture is still loose after performing a snug up application.
- If the silicone begins peeling or lifting early.
- If discomfort or irritation occurs.
- If you are unsure how to proceed.

Contact: Customer Support: 844-682-3368

Mon-Fri 8am-5pm PST

Website: www.densurefit.com