

## Low Speed Handpiece Maintenance

### Low speed Maintenance

1. Clean exterior surface with alcohol. (Do not use harsh cleaners, ultrasonic cleaners, or introduce cleaner into motor.)
2. Insert a bur and tighten.
3. Remove handpiece from tubing.
4. Add 2 to 3 drops of oil into drive air tube.
5. Attach handpiece to tubing, and run at high speed for 10 to 20 seconds.
6. Consult with manufacturer before autoclaving any low speed handpieces for recommended procedures.

### Low Speed Lubrication

1. Add 2 to 3 drops of oil into drive tube.
2. Insert a bur.
3. Run handpiece 10 to 20 seconds

### Low Speed Tips

- Lubricate after each use.
- Keep manufacturers recommended procedures on file for reference.
- Never autoclave detachable motors.
- Set air pressure to manufacturers specifications.
- Use high viscosity lubricating and cleaning oil.