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**Hi, I'm Cris<sup>EN</sup>**

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**Hallo, ich bin Cris<sup>DE</sup>**

---

**Instrucciones de uso. Instructions of use.**

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# Instructions<sup>EN</sup>

## Product purpose:

Cris is a set of four vaginal dilators of different sizes that helps to improve the condition of the vagina and to enjoy sexual intercourse again with no pain.

## Instructions for proper use:

- Apply water-based lubricant to the opening of the vagina and massage gently for 2 to 3 minutes. Make sure you are in a comfortable and relaxed position.
- To start, use the smallest of the four dilators and apply lubricant to the tip.
- Gently and gradually insert the dilator into the vagina.
- Perform the exercises once a day for 10 to 15 minutes, inserting and removing the vaginal dilator continuously. If you feel discomfort in the vagina, stop using the dilator.
- When the smaller element can be inserted comfortably and without discomfort, continue the exercises with the next size dilator until the larger one can be used without discomfort.
- It is advisable to visit a pelvic floor physiotherapist for follow-up and to see how the treatment is progressing.
- After each use, wash the dilator with water and neutral soap or with a specific product.
- Do not use the dilators on inflamed or sore areas.
- Do not use during menstruation, in case of vaginal infection or during pregnancy.

## Composition:

High-quality hypoallergenic silicone, free of latex and phthalates, with a smooth finish.

## Possible risks associated to their use:

- This product is for personal use. Do not share it to prevent the spread of infection.
- Do not use silicone or oil-based lubricants, as they can harm the product's silicone or leave it feeling permanently slimy.
- It is recommended to wash the product thoroughly with water and neutral soap or with a specific product before and after each use.
- It is important that no abrasives or harsh disinfectants containing petroleum-based substances, tea tree oil, alcohol or bleach are used to clean the product.
- For proper maintenance of the product, it is recommended to store it in a dry place, away from dust and direct sunlight.
- It is recommended to store the product in an antibacterial bag.
- Keep out of reach of children.

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