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**Hi, I'm Uma<sup>EN</sup>**

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**Salut, je suis Uma<sup>FR</sup>**

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---

**Instrucciones de uso. Instructions of use.**

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# Instructions<sup>EN</sup>

## Product purpose:

Uma is a set of interchangeable vaginal weights with different weights to strengthen the pelvic floor in a voluntary and progressive way.

## Instructions for proper use:

- A water-based lubricant is recommended for better insertion.
- Start using Uma for 10-15 minutes daily and increase this time gradually, up to 30 minutes per day. Do not use during sudden movements.
- Wash after each use with neutral soap and water or a specific product.
- Do not use on inflamed or injured areas.
- The extraction cord must always remain on the outside.

## Composition:

High quality hypoallergenic silicone.

## Possible risks associated to their use:

- This product is for personal use, do not share it to avoid spreading infections.
- Do not use silicone or oil-based lubricants that may permanently damage or give the product's silicone a slimy feel.
- We recommend washing the product thoroughly with water and neutral soap or with a specialized product before and after each use.
- Do not use abrasives or harsh disinfectants containing petroleum-based substances, tea tree oil, alcohol or bleach to clean the product.
- For proper maintenance of the product, we recommend storing it in a humidity-free, dust-free place and out of direct sunlight.
- We recommend storing the product in an antibacterial bag.
- Keep out of reach of children.

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