


Sueno Pro Quick Start Guide

Inserting Batteries


1) Remove a battery from its packaging, remove battery tab and allow the battery to sit for a few minutes to activate.



2) Push the battery door up until it clicks, you should then be able to open it fully.

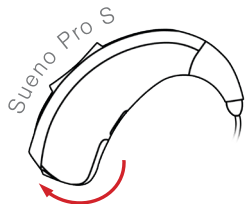

3) Place the battery in the battery compartment, ensuring the + symbol on the battery matches the + symbol on the battery compartment.

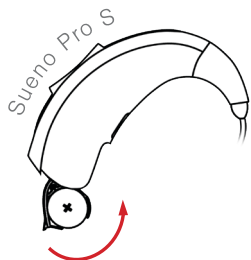

Please do not open the battery door any wider than shown here. This might damage the device.

4) Push the battery door up until it clicks shut.

Power On Delay

Your device will be ready for use 6 seconds after initial switch on.

Multi-Function Rocker Switch

Your device is fitted with a multi-function rocker switch, used to control the programs & volume. The switch features 2 buttons, one at the top (+), and one at the bottom (-).



These buttons can be operated in 2 different ways a long press is used for program changes, a quick press is used for volume changes.

Programs

Your device is supplied pre installed with 3 included programs:

- 1) White Noise 2) Pink Noise 3) Waves**

To change programs, long press the multi-function rocker switch on the top (+) to change in order **1 - 2 - 3** or bottom (-) to change in reverse **3 - 2 - 1**.

Volume Control

To control the volume, quick press the multi-function rocker switch on the top (+) for volume up or bottom (-) for volume down.

Battery Information

BATTERY
SIZE

312




For information on how to change batteries see user guide pages 10-11 (Sueno Pro S) & 20-21 (Sueno Pro R).

Fitting Of The Receiver Unit (Sueno Pro R)

Installation Select a receiver unit (lengths 1 or 2) according to the size of your ear. Slide the unit onto the RIC housing until you hear a click.

Removal To remove the receiver unit, take hold of it between your thumb and forefinger and pull it off the RIC housing carefully.

Installation

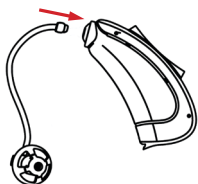

Removal




Fitting Of Tubing (Sueno Pro S)

Installation Thin tubes are installed onto the device using a threaded connection, turn the tube **clockwise** to install.

Removal Thin tubes are removed from the device using a threaded connection, turn the tube **anti-clockwise** to remove.

Installation

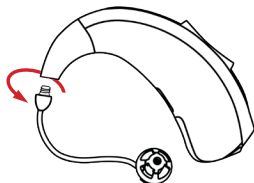

Removal

