cimilre S7 Dual Pro Dual Motor Rechargeable Breast Pump Operation Manual





Use your phone to scan the QR code for manuals in other languages.

Table of Contents

| 1. | Introduction | |
|-----|----------------------------------------|-------|
| | 1.1 Key Features | |
| | 1.2 Indications for Use | 4 |
| | 1.3 Explanation of Symbols | 5-6 |
| 2. | Warnings and Cautions | 6-8 |
| 3. | Components | 9 |
| 4. | Cleaning | |
| | 4.1 Before using your pump | 10 |
| | 4.2 After using your pump | 10 |
| 5. | Assembly | 11 |
| 6. | Pumping | 12 |
| | 6.1 Positioning | 12 |
| | 6.2 Touch Controls | 13 |
| | 6.3 LCD Screen | |
| | 6.4 Battery Information | |
| | 6.5 Pump Operation | |
| | 6.6 Suction Settings | |
| _ | 6.7 Massage and Expression Modes | |
| 7. | Breast Shield Sizing | |
| | 7.1 Understanding Size | |
| | 7.2 Test your breast shield size | |
| | 7.3 Determining the right fit | |
| 8. | Breast Milk Storage | |
| | 8.1 How long can I store breast milk? | |
| | 8.2 How do I thaw and use stored milk? | |
| | Troubleshooting | |
| | Disposal | |
| | Technical Information | |
| | Electromagnetic Compatibility | |
| | Description of IP Classification | |
| 14. | Warranty | 29-30 |

PLEASE READ THIS MANUAL IN ITS ENTIRETY, INCLUDING ALL INSTRUCTIONS AND WARNINGS, BEFORE USE.

1. Introduction

Congratulations on your decision to provide your baby with breast milk and thank you for choosing the Cimilre S7 Dual Pro breast pump to help you reach your breastfeeding goals. While we hope you enjoy your Cimilre S7 Dual Pro, it is recommended to breastfeed your baby whenever you can. The Cimilre S7 Dual Pro was innovated with moms in mind, as it combines premium quality with a modern design. The Cimilre S7 Dual Pro is a safe, efficient, and effective choice for pumping moms.

1.1 Key Features

- Dual Motor Pump with Customized Controls: Easily adjust suction levels for each breast independently for maximum comfort and efficiency.
- Massage & Expression Modes: Effortlessly transition from Massage to Expression Mode by pressing a single button.
- Customized Touch Controls: Easily adjust suction levels and cycle speeds for maximum comfort and an effective pumping session.
- Built-in Rechargeable Battery: With a battery life of 2+ hours between charges, the Cimilre S7 Dual Pro is portable, allowing for ease of use while pumping on the go.

| | 97-14, Seongnam-ro, Mokcheon-eup, Dongnam-gu, Cheonan-si, | Phone : +82-41-553-0941 Email : cimilre@cimilre.kr | |
|--------------|-----------------------------------------------------------------|-------------------------------------------------------|--|
| Manufactured | Chungcheongnam-do, Korea | | |

Please read this document completely and follow the instructions exactly. To prevent personal injury and material damage, please observe the warning and safety information provided in this document. Identification of danger levels:

| MARNING Potentially dangerous situation that could result in serious injury. | |
|---------------------------------------------------------------------------------------|--|
| CAUTION A possibly dangerous situation that could result in slight bodily injur | |
| NOTICE Useful information or information you need to know when usin this product. | |

1.2 Indications for Use

The Cimilre S7 Dual Pro breast pump is intended to be used by lactating women to express and collect milk from their breasts.

1.3 Explanation of Symbols

| Symbol | Description |
|------------|--------------------------------------------------------------------------------------------------------------------------------|
| | Manufacturer |
| M | Date of manufacture |
| SN | Serial number |
| \bigcirc | General prohibition sign |
| | General warning sign |
| | Caution |
| ③ | Notice |
| | Refer to instruction manual |
| ★ | Type BF applied part |
| | CLASS II equipment |
| | DC |
| \sim | Alternating current |
| IP22 | Degree of protection against harmful ingress of water or particulate matter. Please see section 11 for the detail information. |
| | Temperature limit |
| US AGENT | US Agent |

| ø | Humidity limitation |
|-------------|----------------------------------------------------------------------------|
| | Atmospheric pressure limitation |
| <u>بې</u> : | Keep dry |
| Ţ | Fragile, handle with care |
| X | This product should not be mixed with other commercial waste for disposal. |
| Ċ | Power(Start or stop each pump) |
|) | Battery Status |
| ~~ | Adjust the level of suction |
| SYNC | Control both pumps simultaneously |
| CYCLE | 3 cycle settings per level in SYNC Mode |
| | LED night light |

2. Warnings and Cautions

PROHIBITION: To reduce the risk of electrocution.

- Do not modify this equipment without authorization of the manufacturer.
- Do not disassemble or modify the product. The device has no user-serviceable parts.

WARNING: To reduce the risk of electrocution.

- · Always unplug immediately after use.
- · Do not use while bathing.
- Do not place or store product where it can fall or be pulled into a tub or sink.
- Do not place or drop into water or other liquid.
- Do not reach for a product if it has fallen into water. Immediately unplug from electrical outlet.



WARNING: To avoid fire, electrocution, or serious burns.

- · Do not leave the product unattended when plugged into an electrical outlet.
- Close supervision is necessary when this product is used near children or persons with disabilities.
- Place product out of reach of children.
- Use the product only for its intended use as described in this manual. Do not use attachments not recommended by the manufacturer.
- Never operate this product if it has a damaged cord or plug it is not working properly, or if it has been dropped or damaged, or dropped into water.
- · Keep cord away from heated surface.
- Never use while sleeping or when drowsy.
- · Never drop or insert any object into any opening or tubing.
- Do not use outdoors, or operate where aerosol (spray) products are being used or where oxygen is being administered.
- Always inspect power adapter and battery pack wires prior to use for damage or exposed wire. If damage is found, immediately discontinue use of power adapter or battery pack and call Cimilre customer service.
- The Cimilre S7 Dual Pro should not be positioned so that it is difficult to operate the adapter.



WARNING: To avoid health risk and reduce the risk of injury.

- If you are a mother who is infected with Hepatitis B, Hepatitis C, or Human Immunodeficiency Virus (HIV), pumping breast milk will not reduce or remove the risk of transmitting the virus to your baby through your breast milk.
- Before each use visually inspect the individual components for cracks, chips, tears, discoloration or deterioration. In the event that damage to the device is observed, please discontinue use until the parts have been replaced.


- This product is the intended for use by a single user only and should not be shared between users.
- Do not drive while pumping.
- Never use while pregnant, as pumping can induce labor.
- Clean and sanitize all parts that come in contact with your breast and breast milk prior to first use.
- · Wash all parts that come in contact with your breast and breast milk after every use.
- Inspect all appropriate pump components before each use.
- Do not continue pumping for more than 2 consecutive pumping sessions if no results are achieved.
- Do not thaw frozen breast milk in a microwave or in a pan of boiling water.
- If tube becomes moldy, discontinue use and replace tube.
- If breastmilk backup occurs, you must clean tube and make sure it is dry before your next pumping session.

CAUTION: To avoid minor injury.

- Do not wrap cord around adapter body.
- Use only the power adapter that comes with the powered breast pump.
- Pump only with the breast pump in an upright position.
- · Do not shorten tubing.
- Make sure the voltage of the power adapter is compatible with the power source.
- Plug the power adapter into the breast pump first and then into the wall socket.
- Do not use antibacterial or abrasive cleaners/detergents when cleaning breast pump or breast pump parts.
- Never put breast pump motor unit in water or a sterilizer, as you can cause permanent damage to the breast pump.
- Do not attempt to remove the breast shield from your breast while pumping. Turn the breast pump off and break the seal between your breast and breast shield with your finger, then remove breast shield from your breast.
- If pumping is uncomfortable or causing pain, turn the unit off, break the seal between the breast and the breast shield with your finger and remove the breast shield from your breast.
- Plastic bottles and component parts become brittle when frozen and may break when dropped.
- Bottles and component parts may become damaged if mishandled, e.g. dropped, over tightened or knocked over.
- · Take appropriate care in handling bottles and components.
- · Do not use the breast milk if bottles or components become damaged.

3. Components

Cimilre reserves the right to substitute any component or accessory with a replacement of equivalent performance.

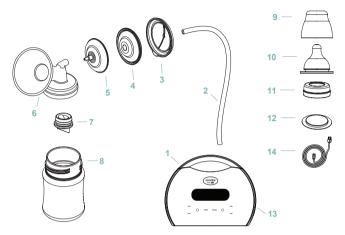

| Included Components | | | | |
|---------------------------|--------------------------------------------|--|--|--|
| 1 - Dual Motor Rec | hargeable Breast Pump, Cimilre S7 Dual Pro | | | |
| 2x Milk Collection Kit | 7 - Valve | | | |
| 14 - USB-C Cable | | | | |

4. Cleaning

4.1 Before using your pump

Clean and sanitize the parts before using your pump for the first time.

- · Backflow protectors
- Breast shields
- · Bottles, bottle discs, and bottle covers
- Nipples
- Valves

WARNING: Do not wash or boil tubing. Tubing needs to remain dry to avoid any potential mold contamination.

Cleaning and sanitizing instructions:

- 1. Only use drinking-quality water or bottled water for cleaning.
- 2. Wash hands thoroughly.
- 3. Wash parts with a clean dish cloth or soft brush in warm soapy water.
- 4. Rinse parts with clear water.
- 5. Sanitize the parts by placing in boiling water for 5 minutes.
- 6. Safely remove all parts from water with tongs.
- 7. Allow all parts to air dry in a clean area.

4.2 After using your pump

Take apart and wash all parts that come in contact with your breasts or breast milk after use to avoid milk residue and bacteria growth.

- 1. Thoroughly wash your hands before handling each breast pump part.
- Disassemble each part that comes into contact with breast milk including the backflow protectors, breast shields, bottles, bottle disks, bottle covers, nipples, and valves.
- 3. Rinse each part in cool water to remove milk residue.
- 4. Separately wash each part in the sink with a soft brush or clean dishcloth; OR soak each part in warm soapy water for 5 minutes and thoroughly rinse them. Alternatively, place the disassembled parts, except for the tubing, on the top rack of your dishwasher.
- 5. Dry each part with a clean towel and/or allow them to completely air dry on a sanitary surface.
- 6. Wipe the main body with a dry, clean towel after each use.
- Store completely dry parts in a clean bag or container. Do not store any damp parts.


We recommend to sanitize all parts that come in contact with your breast and breast milk daily.

5. Assembly

- 1. Wash hands before touching breast pump parts.
- 2. Place breast pump on a clean, dry, and secure surface.
- Insert the backflow protector silicone into the backflow protector bottom (the part that connects to the breast shield). Ensure that it fits securely and that the backflow protector silicone edge fits around the rim of the backflow protector bottom.
- Snap on the backflow protector top (the component that connects to the tubing) to the assembled backflow protect bottom.
- 5. Push the valve onto the underside of the breast shield.
- Connect the backflow protector assembly to the top of the breast shield. Use a twisting motion to attach, and later remove this assembly, to avoid damaging it.
- 7. Screw the bottle onto the underside of the breast shield.
- Connect the tubing to the backflow protector assembly and connect the other end of the tubing to the Cimilre S7 Dual Pro(for single pumping attach the cap on one of the airflow ports).
- 9. Attach power adapter to the Cimilre S7 Dual Pro.
- 10. Plug power adapter into a standard electrical outlet.



- 1. Insert silicone membrane into lower protector.
- 2. Snap on the protector top to the assembled protector bottom.

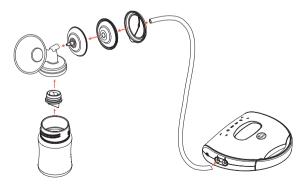

6. Pumping


Gently massage your breasts from the breast toward the nipple or apply warm compresses before using. Express milk from each breast for 10-15 minutes. It is recommended to pump frequently rather than pumping for a long duration. (Every 2-3 hours, 8 times a day).



Consult with your health care provider regarding recommendations for breastfeeding, including breastfeeding while pumping.

6.1 Positioning

1. Before pumping, get into an upright position to prevent milk from back flowing and maintain that position throughout your pumping session.

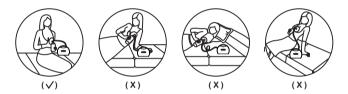

2. Center the breast shield on your breast so that the nipple is drawn into the flange.

It is important that the breast shield is positioned properly before pumping begins to avoid any pain that may be caused by rubbing or twisting of your nipple onto the side of the flange. If double pumping, place the second breast shield in place.




While some discomfort may be felt when first using a breast pump, using a breast pump should not cause pain. If you are unsure about breast shield sizing, please contact a Lactation Consultant or Cimilre representative who can help you get a proper fit.



When pump is powered on for the first time after unboxing, the suction may be preset to a higher level than desired. Adjust the level to a preferred setting before first use.

6.2 Touch Controls

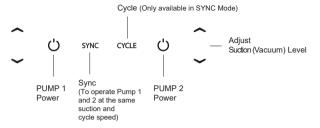

Enabling and Disabling the Touch Control Sound


- Enabling mute: While the power is on, touch and hold the up and down level control on either the left or right side at the same time for 3 seconds.
- Disabling mute: While the mute mode is enabled, touch and hold the up and down level control on either the left or right side at the same time for 3 seconds. (Same method as enabling.)

Night Light

Dual 💷

- The LED light appears by pressing the cimilre logo.
- cimilre LED can be adjusted with 3 dim levels.
 - Press the button one more time at level 3 to turn off the LED.
 - Leaving the pump idle for 30 minutes without pressing the power button or the LED button will automatically power off the device.

6.3 LCD Screen

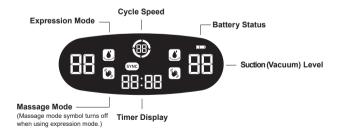


Cycle Speed : In SYNC Mode, you can choose from 3 different cycle speeds per level by pressing the cycle control. When you are not in SYNC Mode, each pump will default to the highest cycle speed per level as indicated in the chart on pages 17.

6.4 Battery Information

| No. | Description | |
|-----------------------------------------------------------------------------------------------------------------|----------------------------------------------------------------------------------------------------------------|--|
| 1 | Operating time: Approx 2+ hours | |
| 2 | When Charging - Bars flash progressively inside the battery indicator to show that the pump is charging. | |
| Fully Charged 3 - Battery indicator shows 3 full bars and the outline of the battery blinks continuously. | | |

Battery Icon - Different Levels of Charge

| | | | - TP |
|-------------|------------------------------------------------|------------|-------------|
| Low Battery | 1/3 Charge - Connect USB-C cable soon | 2/3 Charge | Full Charge |

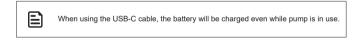

6.5 Pump Operation

- When the pump is powered on for the first time after unboxing, the suction may be preset to a higher level than desired. Adjust the level to a preferred setting before first use. After first use, the Cimilre S7 Dual Pro will automatically remember your last settings for your next pumping session.
- 1. Wash hands before touching breast pump and parts.
- 2. Connect the tubing to each air flow port.

- 3. To begin pumping, press and hold the power control for one of the pumps for 3 seconds to turn the pump on (this feature is to prevent the device from unintended operation). Once the pump is powered on, press the power control of the other side of the pump just once to activate.
- 4. To single pump, only power on one of the pumps. If you are double pumping and wish to single pump, press the power control to pause the motor for the breast you will no longer be pumping.

5. To adjust suction (vacuum) strength while in SYNC Mode, use the \checkmark velocity level control to select preferred suction strength/level. (Levels 1-5 (Massage Mode) / Levels 6-12 (Expression Mode))











- 6. To adjust the suction (vacuum) strength for each breast independently, press the SYNC control to power SYNC off ("SYNC" light in center of the screen will turn off). Then you can adjust the level of each pump by pressing the arrow control.
- 7. To return to using the same suction level for both sides, press the SYNC control. The SYNC light will appear in the center of the screen, and the suction level will default to the lowest suction level set between the two pumps.

In SYNC Mode, you can choose from 3 different cycle speeds per level by pressing the cycle control. When you are not in SYNC Mode, each pump will default to the highest cycle speed per level, as indicated in the chart on page 17.

- 8. When you wish to stop expressing when in SYNC Mode, tap either one of the power controls. To stop expressing when pumping each side independently, tap the power control for each side. To stop expressing when single pumping, tap the power control on the side you are pumping. Then remove the milk collection kit from your breast.
- 9. Close the bottle(s) tightly with bottle disk and cover, and store safely.









| cimilre S7 Dual Pro | | | | |
|---------------------|---------|------|---------------|-----------------------|
| | Suction | mmHg | Cycle Speeds | |
| | Level | | ⊘ sync | SYNC |
| | 1 | 60 | 80 | 80 75 |
| | | | | 70 75 |
| Massage Mode | 2 | 80 | 75 | 70 65 |
| | 3 | 100 | 70 | 70 65 60 |
| | 4 | 120 | 65 | 65 60 55 |
| | 5 | 140 | 60 | 60 55 50 |
| | Suction | mmHg | Cycle Speeds | |
| | Level | | ⊘ SYNC | SYNC |
| | 6 | 170 | 40 | 40 35 30 |
| | 7 | 190 | 40 | 40 35 30 |
| Expression Mode | 8 | 210 | 35 | 35 30 25 |
| | 9 | 230 | 35 | 35 30 25 |
| | 10 | 250 | 30 | 30 25 20 |
| | 11 | 280 | 30 | 30 25 |
| | 12 | 300 | 25 | 20 25 20 |
| | | | | 15 |

6.6 Suction Settings

- The Cimilre S7 Dual Pro will automatically remember your last settings for your next pumping session. If you want to start pumping on different settings, adjust the settings before you start expressing.
- 2. The suction (vacuum) level refers to the suction strength; this is measured in millimeters of mercury (mmHg). When you are using the Cimilre S7 Dual Pro, you will usually find that expressing is most effective on the highest suction level that is comfortable for you.

However, it's important not to set suction level so high that it makes you uncomfortable or causes pain as this can hinder your milk flow.



6.7 Massage and Expression Modes

- Massage Mode : Suction pattern with fast cycle speed and low suction to help encourage a letdown and to start milk flowing.
- Expression Mode : Suction pattern with slower cycle speed and higher suction to express milk gently and efficiently.

7. Breast Shield Sizing


7.1 Understanding Size

Pumping should not hurt. This guide is a starting point to help determine your optimal size based on your nipple diameter.

How to find the right size flange for you:


Step 1: Using a ruler or measuring tape, measure the diameter of your nipple at base (across middle) in millimeters (mm). *1cm = 10mm. Do not include areola.

Step 2: Take your measurement and add 4mm to determine your breast shield size. *Example: If your nipple size measures 24mm in diameter, the recommended breast shield size is 26-28mm.

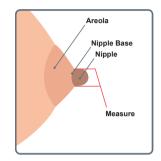

7.2 Test your breast shield size

- **Good Fit** : The nipple, or the nipple and a small part of the areola, enters the tunnel when pumping. The nipple moves freely or slightly touches the tunnel with no discomfort.
- Too Small : During pumping, your entire nipple rubs against the sides of the flange and is uncomfortable.
- **Too Large :** During pumping, more of your areola is drawn into the breast pump flange with your nipple. You may experience your areola rubbing up against the side of the flange tunnel.

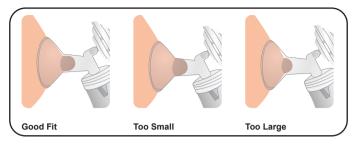

7.3 Determining the right fit

- Does your nipple rub the sides of flange tunnel to the point of causing discomfort?
- Do you see excessive areola being pulled into tunnel?
- Do you see any redness?
- · Is your nipple or areola turning white?
- · Do you feel unexpressed milk after pumping?

If you answered 'YES' to any of these questions, consider trying a new size by following the measuring instructions above. If you are still unsure if you selected the correct size, see a lactaction consultant, or breastfeeding specialist for assistance in choosing the right size breast shield.

8. Breast Milk Storage

8.1 How long can I store breast milk?

| | Storage Locations and Temperatures | | |
|----------------------------------------------------------------|-----------------------------------------------------------------|----------------------------|----------------------------------------------------------|
| Type of Breast Milk | Countertop 77°F (25°C) or colder (room temperature) | Refrigerator 40°F (4°C) | Freezer 0°F (-18°C) or colder |
| Freshly Expressed or Pumped | Up to 4 hours | Up to 4 Days | Within 6 months is best Up to 12 months is acceptable |
| Thawed, Previously Frozen | 1-2 hours | Up to 1 Day (24 hours) | Never refreeze human milk after it has been thawed |
| Leftover from a Feeding (baby did not finish the bottle) | eding (baby did not Use within 2 hours after the baby is finish | | by is finished feeding |

Source: cdc.gov

8.2 How do I thaw and use stored breast milk?

- 1. Always thaw the oldest breast milk first. Remember first in, first out. Over time, the quality of breast milk can decrease.
- 2. There are several ways to thaw your breast milk:
 - a. In the refrigerator overnight.
 - b. Set in a container of warm or lukewarm water.
 - c. Under lukewarm running water.
- Never thaw or heat breast milk in a microwave. Microwaving can destroy nutrients in breast milk and create hot spots, which can burn a baby's mouth.
- 4. Use breast milk within 24 hours of thawing in the refrigerator (this means from the time it is no longer frozen or completely thawed, not from the time when you took it out of the freezer).
- 5. Once breast milk is brought to room temperature or warmed after storing in the refrigerator or freezer, it should be used within 2 hours.
- 6. Never refreeze breast milk once it has been thawed.

Source: cdc.gov

9. Troubleshooting

| Condition | Possible Causes | Checks | Solutions |
|-------------|------------------------------|------------------------------------------------------------------------------------------------------------------|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| No power | Adapter | Confirm that power cord is not damaged and is connected to standard electrical outlet. | If cord is damaged, order replacement at cimilrebreastpumps.com. Plug into standard outlet. If it is still not working, contact Cimilre at (888) 394-2422. |
| No suction | Tubing | Check the tubing to make sure it is connected to the air flow ports on the pump. | Connect the tubing to the air flow ports on the pump. |
| | Valve | Check if they are properly connected to breast shield. | Disassemble and properly reassemble. |
| | Suction level | Listen for abnormal pump sound/noise. | Contact Cimilre at (888) 394- 2422. |
| | Defective assembly kit | Check the connection of the kit. | Disassemble and reassemble the kit. |
| Low suction | Breast Shield | Examine the breast shield for potential cracks or chips. Check breast shield sizing (see section 7). | Visit cimilrebreastpumps.com to purchase and replace breast shield. |
| | Valve | Check if there is damage to the valve. | Valves should be Cimilre brand and should be changed every 1-3 months. Visit cimilrebreastpumps.com to purchase and replace valve. |
| Pain | Suction too high | Check the suction level. | Decrease suction level and consult with your lactation consultant or healthcare provider. |

10. Disposal



As an electrical and electronic equipment, the device must be disposed of according to a specialized procedure for collection, pick-up and recycling or destruction in accordance with Directive 2012/19/EU of the European Council on Waste Electrical and Electronic Equipment [WEEE]. Please properly dispose the Class II medical device in accordance with local, state and federal regulations.

11. Technical Information

| Contents | Description | | |
|-----------------------------------------------|---------------------------------|-------------------------------|--|
| Rated voltage and frequency | Input: DC 5V, 2A | | |
| Power consumption | 0.4 A | | |
| Battery / Life | 3.7V 2300mAh Li-P | olymer / 2 hours + | |
| Protection on electronic shock | Class II, 🔲 TYPE | BF Applied Part Ҟ | |
| Degree of protection against ingress of water | IP Classification: IF | 22 | |
| Operation mode (Continuous operation) | Expression, Massa | ge, SYNC Mode | |
| Suction levels | Expression Mode | 6 - 12 Levels | |
| | Massage Mode | 1 - 5 Levels | |
| Cycle speeds (see page 17 for | Expression Mode | Fixed (24-35) | |
| adjustable cycle speeds in SYNC Mode) | Massage Mode | Fixed (52-93) | |
| Single/Double | Single/Double | | |
| Timer function | Automatic Off (30 min ± 1 min) | | |
| Noise | Less than 45 dB | | |
| Backflow protection | Yes | | |
| Cleaning Method for Accessories | Boiling water | | |
| Operating conditions | | | |
| Ambient temperature | 41 to 104°F | | |
| Relative humidity | 20 to 80 % | | |
| Air pressure | 700 to 1,060 hPa | Temperature Humidity Pressure | |
| Transport and storage conditions | - | | |
| Ambient temperature | -4 to 122°F | | |
| Relative humidity | 20 to 95 % | | |
| Air pressure | 500 to 1,060 hPa | Temperature Humidity Pressure | |

12. Electromagnetic Compatibility

| The Cimilre S7 Dual Pro is intended for use in the electromagnetic environment specified in the table below. The customer or the user of the Cimilre S7 Dual Pro should ensure that it is used in such an environment. | | |
|------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|------------|----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| Emission test | Compliance | Electromagnetic environment—guidance |
| RF emission - CISPR 11 | Group 1 | The Cimilre S7 Dual Pro uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment. |
| RF emission - CISPR 11 | Class B | The Cimilre S7 Dual Pro is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage network that is supplied to buildings used for domestic purposes. |
| Harmonic emissions IEC 61000-3-2 | Class A | puiposes. |
| Voltage fluctuations /flicker emissions IEC 61000-3-3 | Complies | |

Guidelines and Manufacturer's Declaration – Electromagnetic Emissions

| Outdance and | Manufacturer 5 | Declaration - E | ectromagnetic minutity |
|-----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|------------------------------|------------------------------|-----------------------------------------------------------------------------------------------------------------------------------------------------------|
| The Cimilre S7 Dual Pro is intended for use in the electromagnetic environment specified below. The customer or the user of the Cimilre S7 Dual Pro should ensure that it is used in such an environment. | | | |
| Immunity test | IEC 60601 test level | Compliance level | Electromagnetic environment—guidance |
| Electrostatic discharge (ESD) IEC 61000-4-2 | ± 6 kV contact ± 8 kV air | ± 6 kV contact ± 8 kV air | Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%. |

Guidance and Manufacturer's Declaration - Electromagnetic Immunity

| Electrical fast transients IEC 61000-4-4 | ± 2 kV for power supply lines ± 1 kV for input/ output lines | ± 2 kV for power supply lines | Mains power quality should be that of a typical commercial or hospital environment. |
|----------------------------------------------------------------------------------------|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| Surges IEC 61000-4-5 | ± 1 kV differential mode ± 2 kV common mode | ± 1 kV differential mode ± 2 kV common mode | Mains power quality should be that of a typical commercial or hospital environment. |
| Voltage dips, short interruptions and voltage variations IEC 61000-4-11 | < 5% U T (> 95% dip in U T) for 0.5 cycle 40% U T (60% dip in U T) for 6 cycles 70% U T (30% dip in U T) for 30 cycles < 5% U T (>95% dip in U T) for 5 s | < 5% U T (> 95% dip in U T) for 0.5 cycle 40% U T (60% dip in U T) for 6 cycles 70% U T (30% dip in U T) for 30 cycles < 5% U T (>95% dip in U T) for 5 s | Mains power quality should be that of a typical commercial or hospital environment. If the user of the Cimilre S7 Dual Pro requires continued operation during a power interruption, it is recommended that the Cimilre S7 Dual Pro be powered from an uninterruptible power supply. |
| Power frequency (50/60 Hz) magnetic field IEC 61000-4-8 | ± 1 kV differential mode ± 2 kV common mode | ± 1 kV differential mode ± 2 kV common mode | Power frequency magnetic fields should be measured in the intended installation location to ensure that it is sufficiently low. |
| NOTE. U_{τ} is the AC mains voltage prior to application of the test level. | | | |

| Conducted RF IEC 61000-4-6 | 3 Vrms 150 kHz to 80 MHz | 3 Vrms | Portable and mobile RF communications equipment should be used no closer to any part of the device, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. |
|-------------------------------|--------------------------------|--------|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| Radiated RF IEC 61000-4-3 | 3 V/m 80 MHz to 2.5 GHz | 3 V/m | |
| | | | Recommended separation distance $d = 1, 2 \sqrt{P}$ $d = 1, 2 \sqrt{P}$ 80 MHz to 800 MHz $d = 2, 3 \sqrt{P}$ 800 MHz to 2.5 GHz |
| | | | Where (P) is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m). |
| | | | Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, should be less than the compliance level in each frequency range ^b . Interference may occur in the vicinity of equipment marked with the following symbol: (()) |
| | Iz and 800 MHz, th | | / range applies. s. Electromagnetic pro- |

NOTE 2. These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people. a. Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the device is used exceeds the applicable RF compliance level above, the device should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the device.

b. Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3V/m.

Recommended separation distances between portable and mobile RF communications equipment and the Cimilre S7 Dual Pro

The Cimilre S7 Dual Pro is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of Cimilre S7 Dual Pro can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the Cimilre S7 Dual Pro as recommended below, according to the maximum output power of the communications equipment.

| Rated | Separation distance according to frequency of transmitter [m] | | |
|-----------------------------------------------|---------------------------------------------------------------|-------------------------------|-----------------------------------|
| maximum output power of transmitter [W] | 150kHz to 80MHz d = 1,2 √P | 80MHz to 800MHz d = 1,2 √P | 800MHz to 2.5GHz d = 2,3 √P |
| 0.01 | 0.12 m | 0.12 m | 0.23 m |
| 0.1 | 0.38 m | 0.38 m | 0.73 m |
| 1 | 1.2 m | 1.2 m | 2.3 m |
| 10 | 3.8 m | 3.8 m | 7.3 m |
| 100 | 12 m | 12 m | 23 m |

For transmitters rated at a maximum output power not listed above, the recommended separation distance (d) in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE 1. At 80MHz and 800MHz, the separation distance for the higher frequency range applies. NOTE 2. These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

13. Description of IP Classification

| 1: Protected against solid foreign objects of 50 mm Ø and greater 2: Protected against solid foreign objects of 12,5 mm Ø and greater 3: Protected against solid foreign objects of 2,5 mm Ø and greater 4: Protected against solid foreign objects of 1,0 mm Ø and greater 5: Dust-protected | |
|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|--------|
| N1 3: Protected against solid foreign objects of 2,5 mm Ø and greater 4: Protected against solid foreign objects of 1,0 mm Ø and greater | |
| 4: Protected against solid foreign objects of 1,0 mm Ø and greater | |
| | |
| 5: Dust-protected | |
| | |
| 6: Dust-tight | |
| IPN ₁ N ₂ 0: Non-protected | IPN/Na |
| 1: Protection against vertically falling water drops | |
| 2: Protection against vertically falling water drops when enclosure tilted up to 15° | |
| 3: Protected against spraying water | |
| N ₂ 4: Protected against splashing water | |
| 5: Protected against water jets | |
| 6: Protected against powerful water jets | |
| 7: Protected against the effects of temporary immersion in water | |
| 8: Protected against the effects of continuous immersion in water | |

14. Warranty

LIMITED MANUFACTURER'S WARRANTY

The manufacturer provides the following limited warranties on the Cimilre S7 Dual Pro:

THE MANUFACTURER PROVIDES A LIMITED 24-MONTH MECHANICAL WARRANTY FOR MAIN PUMP:

The manufacturer provides a LIMITED MECHANICAL WARRANTY that the main pump will be free from manufacturing defects in materials and workmanship under normal personal use for the 24-month period after purchase by the original owner. Proof of purchase (copy of your original receipt, purchase order, or invoice), the date that the product was purchased, and the location of purchase will be required. Replaced or repaired products are warranted only for the period remaining under the initial warranty.

THE MANUFACTURER ALSO PROVIDES A LIMITED 90-DAY WARRANTY FOR ALL OTHER PARTS:

The manufacturer provides a LIMITED WARRANTY that all other parts will be free from manufacturing defects in materials and workmanship under normal personal use for the 90-day period after purchase by the original owner. Proof of purchase (copy of your original receipt, purchase order, or invoice), the date that the product was purchased, and the location of purchase will be required. Replaced or repaired products are warranted only for the period remaining under the initial warranty.

At the manufacturer's discretion, the manufacturer will replace the main pump and/or any part that proves to be defective in material and/or workmanship under normal use and service. PLEASE NOTE: WARRANTIES DO NOT COVER NORMAL "WEAR AND TEAR." This warranty is extended to the original consumer owner only and shall not apply to any products or parts of products which have been (A) Repaired by unqualified persons not using original spare parts or altered in any manner unless previously authorized in writing by the manufacturer; or (B) Damaged as a result of misuse, negligence, accident, or faulty use or maintenance; or (C) Used or stored in a manner contrary to the manufacturer's written instructions; or (D) Damaged by the effects of improper cleaning and/or use of aggressive cleaners. The warranty does not cover misplacement or theft of the breast pump or supplies.

THIS IS THE EXCLUSIVE WARRANTY BY THE MANUFACTURER, WHICH DOES NOT MAKE ANY OTHER WARRANTY OF ANY KIND, INCLUDING BUT NOT LIMITED TO THE IMPLIED WARRANTY OF MERCHANTABILITY.

This breast pump is intended for single-user use only. Sharing a breast pump can cause health hazards and voids the manufacturer's warranty. The user of this product assumes all responsibilities and risks for use of this product. The user should consult with a health care professional before using this or any other equipment.

THIS WARRANTY IS LIMITED IN SOLELY TO THE ABOVE AND THIS WARRANTY AND ANY WARRANTIES IMPLIED BY STATE LAW WILL APPLY ONLY FOR THE APPLICABLE WARRANTY PERIOD SPECIFIED HEREIN, ALSO, THE MANUFACTURER WILL NOT BE LIABLE FOR ANY LOSS, DAMAGE, EXPENSE OR INCIDENTAL OR CONSEQUENTIAL DAMAGES OF ANY KIND. WHETHER BASED ON WARRANTY. CONTRACT OR NEGLIGENCE AND ARISING IN CONNECTION WITH THE SALE. USE OR REPAIR OF THE PRODUCT. SOME STATES DO NOT ALLOW THE EXCLUSION OR LIMITATION OF CONSEQUENTIAL DAMAGE OR LIMITATIONS ON HOW LONG AN IMPLIED WARRANTY LASTS, SO THE ABOVE LIMITATIONS OR EXCLUSIONS MAY NOT APPLY. THIS WARRANTY GIVES YOU SPECIFIC LEGAL RIGHTS. AND YOU MAY ALSO HAVE OTHER RIGHTS WHICH VARY FROM STATE TO STATE UNLESS OTHERWISE CONTRARY TO STATE LAW GOVERNING THE PURCHASE. THE MANUFACTURER'S LIABILITY WILL NOT EXCEED THE CONTRACT PRICE FOR THE PRODUCT. CLAIMED TO BE DEFECTIVE OR UNSUITABLE

To make a warranty claim visit cimilrebreastpumps.com/warranty.

Please note that items returned without a Replacement Authorization Number will be returned to Sender.

CIMIRLE S7 Dual Pro Operation Manual

Cimilre Co., Ltd.

97-14, Seongnam-ro, Mokcheon-eup, Dongnam-gu, Cheonan-si, Chungcheongnam-do, Korea Phone : +82-41-553-0941 Email : cimilre@cimilre.kr

US AGENT Symmetrical Health, LLC

615 Main Street Toms River, NJ 08753 Phone: (888) 394-2422 Email: help@cimilrebreastpumps.com

2023.11.02 REV0