NewSmile

# Impression Guide

**You Got This!**

---

**Disclaimer:** Please carefully follow instructions or results may be hindered and our smile guarantee will be voided.

## **PRECAUTIONS**

Putty may cause an allergic skin reaction, and cause severe eye irritation.

**Precautionary statements Prevention:** Wear protective gloves, protective clothing and eye protection. Wash hands thoroughly with soap and water before and after handling.

In case skin irritation occurs, seek medical attention.

**IF IN EYES:** Rinse with caution with plenty of water for several minutes. In case contact lenses are worn remove and continue rinsing. If you feel unwell get medical attention. After contact with eyes: rinse with copious amounts of water for at least 15 minutes. Get medical attention.

Do not swallow. If ingestion occurs, flush mouth with water and get medical attention.

### **GET YOUR IMPRESSION KIT**

Once you get your kit, book a 1-on-1 video call with us to make your impressions right the first time, because replacement kits may be an additional charge.

### **VIEW YOUR NEW SMILE**

Send us back your impressions with the return label provided, and in 3-4 weeks our licensed dentists and orthodontists will create a 3D preview of your new smile.

### **GET YOUR ALIGNERS**

If all looks good, we'll start making your aligners and in 4 weeks, you'll get your aligners delivered to your door.

### **FINISH ON TIME**

Make regular check-in's to ensure you are progressing correctly to finish on time.

# Making a Good First Impression

---

Impressions are the most important part when starting the process, so be sure to read this entire guide before you begin. Don't stress, and by the time you read through our easy-to-follow guide, you'll be an impression taking pro.

# In This Kit

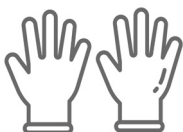
---

(4) Containers of color putty\*

(4) Containers of white putty

(1) Pair of gloves

(2) Upper Impression Trays

(2) Lower Impression Trays

(1) Mouth spreader for photos

(1) Prepaid shipping label


\*Dental Impression Putty colors may vary.

†Tray color will vary depending on size. If you need a different size than what you currently have, please contact us on our website.

If you did not receive a return shipping label, or have lost the one received, please email us with your full name and order number.

# How to put on mouth spreader

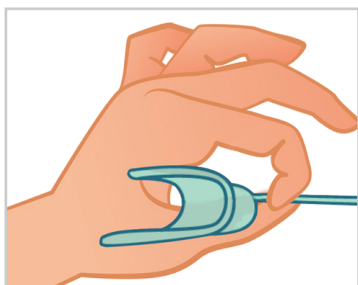---


1

Hold the mouth spreader in a "U" shape and place one side of the wing on one side of your mouth.

2

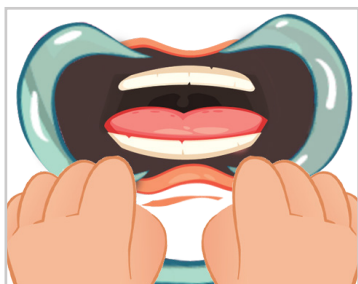Place the other wing of the mouth spreader on the other side of the mouth.

3

Take your photos from all angles required - front, left side, right side, upper, and lower angle.

# Getting The Perfect Impression

---

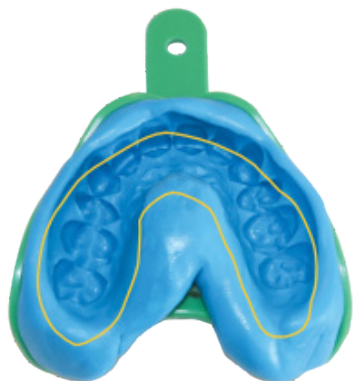
Brush and floss your teeth before taking your impressions. This will remove plaque and any food bits to provide accurate impressions so that we are able to create the best fitting aligners for you.

**Note:** You can practice with two green colored putties together - two colored ones will not harden. Do NOT mix the green and white putty together to practice - they will harden. Do NOT throw any putty away after practicing. You will separate the colored putty back into equal parts, to mix with the white putty during your video call.

# What we need from your impressions

---



## Visible gum line

Get a deep impression by pressing the tray into your teeth until they firmly touch the bottom of the tray all the way around. This allows the putty to capture the entire tooth (and not half a tooth).

## Even pressure on both sides

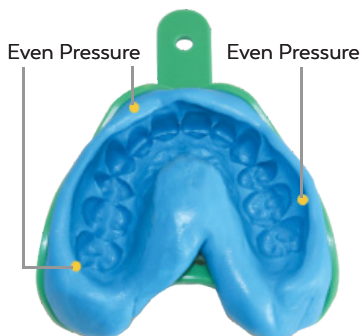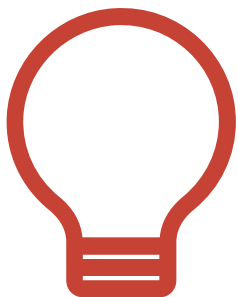Using both hands, use even pressure on both sides, and press the putty into your teeth from your back teeth (the molars), and continue towards your front teeth. This will create an accurate imprint of your teeth into the putty.

# Wait! Before You Start...

## Book a Time for a Video Call

---

You're ready to take the perfect impression, so set yourself up for success by following these final tips:

-  Book a time to take your impressions with an Impression Specialist
-  Stay focused, so you mix the putty quickly for an even color
-  Have a timer to time yourself
-  Have a towel handy, some drooling may occur
-  Don't forget to relax

**Got it? Good. Let's do this!**

# Practice

---

Then book a 1-on-1 video call before doing the real thing.

 **Facebook:** NewSmileToday

 **Email Us:**

Canada: [team@newsmilelife.ca](mailto:team@newsmilelife.ca)

USA: [team@newsmilelife.com](mailto:team@newsmilelife.com)

UK: [team@newsmilelife.co.uk](mailto:team@newsmilelife.co.uk)

AU: [team@newsmilelife.com.au](mailto:team@newsmilelife.com.au)

# Practice

---

We all know practice makes perfect.

Feel free to practice before your video call, and send us pictures of your practice impressions via email or online chat and we'll provide real-time feedback. Practice until you feel confident with a deep impression, having a distinct gum line, and clearly captured teeth. If you don't have time to practice, no worries, just go ahead and book a 1-on-1 video call with us so we can guide you through.

# 2

## Ready, Set...

---

This is it. All your training, preparation and hard work has gotten you up to this point. In less than the time it takes to watch an episode of your favorite Netflix show, you'll be done with your impressions. Grab your stopwatch, it's time to impress yourself. It will take roughly 4 minutes and 30 seconds per impression.

## Step 2a: Mix It Up

Take one color putty and one white putty from their container cups, and lay each color over one another. Start your timer and mix the putty by stretching both colors on one end, and folding them over one another continuously for 30-40 seconds. Pull and fold the putty until the entire mixture is one even and solid color.

 **Total time: 30-40 seconds**

## Step 2b: Perform Rolling

Roll the putty into a log-like shape about the width of your palm. This should take about 10 seconds.

 **Total time: 10 seconds**

## Step 2c: Place in Tray

Place the log-shaped putty into one of your empty trays. Be sure to use your thumbs to press the putty to evenly distribute the putty throughout the tray. This should take about 10 seconds.

**Total time: 10 seconds**

## Step 2d: Impression Time

It's always good to be in front of a mirror when taking your impressions (even when doing a 1-on-1 video call). This will help you align the tray so your teeth will sink into the center of the putty. Place the tray in your mouth so your teeth sink in the middle of the rolled putty. When ready, use your fingers from both hands to press the tray into your teeth with equal pressure on both sides, starting from the molars of your teeth (at the back). Press the putty until your teeth firmly touch the bottom of the tray (keep pressing until your teeth touch the tray).

## Step 2d: Impression Time

Once the rear molars have touched the bottom of the tray, work your way to the front so your teeth touch the bottom on all sides of the tray. Place your lip over the tray, so the putty will seep out of the tray, and into your gums. With your lip over the tray, gently press above the lip and on the sides of your cheeks (with other hand holding tray in place), so the putty creates an imprint of your gums (and gumline). Repeat this process for the remaining 3 impressions.

Important: once the impression has hardened, and you remove the tray from your mouth, do not remove the hardened putty from the tray.

We know making your impressions can be tricky! That's why you should book an impression session with us.

**Total time: 3 minutes 30 seconds**

# Want to Win Money Back?

**Silly question, we know. Here's how:**

- ☺ Take a selfie while you're taking your impressions (it may be the practice one) and hold our impression kit up in the photo.
- ☺ Post it on your Instagram, Tiktok and/or Facebook
- ☺ Be sure to tag us on IG: @NewSmileLife and use hashtag **#HelloConfidence #NewSmileLife #ClearAligners**

We will pick a random winner each month to receive a 3-month supply of teeth whitening!

3

## Get Ready for Your Close-Up

---

It's time to grab the mouth spreader from your kit to take some photos! Don't forget this this step! We need these to make your aligners.

# Step 3: Photo Time

This step is important because it's required in order to make your clear aligners. Our network of licensed dentists and orthodontists use your photos (in combination) with your impressions, to analyze your bite and to properly align everything. Bite down naturally in all photos.

## We'll Need Photos From These Angles

---

Once you've taken photos from all angles (your front, left, right, upper, and lower views) upload them with the Photo Upload Link emailed by our Customer Success Team. Send your photos as soon as possible, so we can make your 3D Treatment Preview showing your straighter smile. This is an essential step for your NewSmile™ journey.

If you've deleted, junked or didn't receive a photo upload link via email, contact our Customer Success Team.

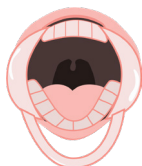
Canada: [team@newsmilelife.ca](mailto:team@newsmilelife.ca)

USA: [team@newsmilelife.com](mailto:team@newsmilelife.com)


UK: [team@newsmilelife.co.uk](mailto:team@newsmilelife.co.uk)

AU: [team@newsmilelife.com.au](mailto:team@newsmilelife.com.au)

**Lower Teeth from Above**


**Upper Teeth from Below**

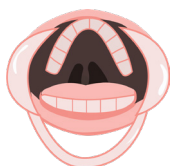
**Biting Down from the Front**

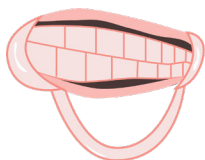
**Biting Down from the Right**

**Biting Down from the Left**
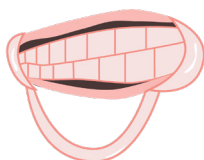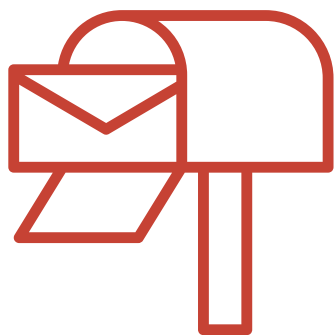
## **Step 4 : Sending Them Back**

Congratulations! You've finished the most important step on the first part of your journey. Place your completed impression molds back into the NewSmile™ box. There's a prepaid shipping label to place over the original shipping label (with your address) on the outside of the box. Drop that off at a post office, and wait for an update from the NewSmile™ Team. Delivery delays occur every so often, so track your impression return online, and contact us if you have not heard from us in 8-10 business days after shipping your impressions back to us.

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