Impression Guide

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You Got This!

Vous l'avez !

Making a Good First Impression

Congrats on taking the first step of your NewSmile journey—this call for a big high five.

Impressions are the most important part of the process, so be sure to read this whole guide before starting. Don't stress, and by the time you read through our easy-to-follow guide, you'll be a certified impression taking pro.

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In This Kit

- (1) Reminder to book a time
(To prevent any delays)
- (8) Containers of Dental
Impression Putty*
(These are the real deal ones)
- (4) Impression Trays
(2 Tops and 2 Bottoms)
- (1) Mouth Spreader for Photos
- (1) Prepaid Shipping Label
(use the same box to ship your
impressions back)

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*Dental Impression Putty colors may vary.

¹Impression Tray colors may vary depending on size. The standard size included in this kit. For a different tray size, email us or chat with us on Facebook.

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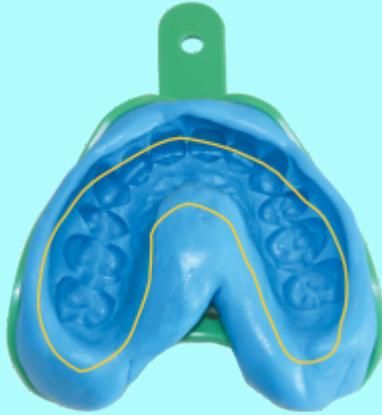
Getting The Perfect Impression

Nobody's perfect. However, there are a few things to focus on to get your impressions close to perfection.

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Visible GumLine

Push the impression tray far enough up on your teeth to get to your gums so that your gum line shows in the impression when you pull it off.

Note: You can practice with two green colored putties together - two colored ones will not harden. Do NOT mix the green and white putty together to practice - they will harden. Be sure to apply enough pressure to get it all the way to the gums. If you mix two green putties to practice, do NOT throw them away after. You will separate into equal parts to mix with the white putty during your video call.

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Even Pressure
Une pression

Even Pressure

Make sure to apply even pressure to your molars and your front teeth to capture both your teeth and gum line. It's easiest to place your thumbs on your back teeth initially and move them to your canines when applying pressure.

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The Most Common Mistakes

Before you take your impressions, here are a few common mistakes to avoid.

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Improper Putty Mixing

The putty must be completely mixed and one solid even color. Just remember you only have 40 seconds to mix the putty. The best way to do this is to stretch apart and fold the two colors together over and over.

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Mordre à

Biting Through Trays

If you're seeing a part of the tray, you may have accidentally bit all the way through the putty. Depending on where your teeth fit into the tray before using the putty, you may need to adjust the putty in the tray to compensate. In other words, if your teeth are more towards the outside of the tray, compensate with the putty by pushing it a bit more towards the outside of the tray than the inside. If the tray doesn't fit, contact us via email or Facebook chat.

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Le liseré

Missing Full Gum Line

In this case, this person would need to press their front teeth deeper in order to capture the gums. When your teeth sink into the putty, you want the putty to come out of the tray to go up into the gums, and make an imprint of your gums as well.

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Wait! Before You Start... Book a Time for a Video Call

Now that you know how to get the perfect impression, set yourself up for success by following these pro tips:

- Stay focused, you don't want the putty to harden before you sink your teeth into it
- Use a stop watch to time yourself
- Take impressions in front of a mirror
- Have a towel handy, some drooling may occur
- Don't forget to relax

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Got it? Good. Let's do this!

Practice

Then book a 1-on-1 video call before doing the real thing.

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Step 1: Practice

We all know practice makes perfect.

Follow the directions in Step 2 (pg 16). Send us pictures of your practice impressions via email or online chat and we'll give you real-time feedback. Practice until you feel confident you can get an impression with an apparent gum line, clearly captured teeth and even pressure. Then we recommend to book a 1-on-1 video call with us.

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Ready, Set...

This is it. All your training, preparation and hard work has gotten you up to this point. In less than the time it takes to watch an episode of your favorite Netflix show, you'll be done with your impressions. Grab your stopwatch, it's time to impress yourself. This will only take about **4 minutes and 30 seconds per impression.**

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Step 2a: Mix It Up

Take one color of each putty out of their cups and start your timer. You should mix the putty for 30-40 seconds. Combine the putty in your hands until the entire thing is one solid color.

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 **30-40 secondes**

Step 2b: Perform Rolling

Roll the putty into a snake-like shape about the width of your palm. This should take about 10 seconds.

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10 secondes

Step 2c: Place in Tray

Place the snake-shaped putty into one of your empty trays. Make sure to use your thumbs to evenly distribute the putty throughout the tray. This should take about 15 seconds.

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 15 secondes

Step 2d: Impression Time

Looking in the mirror, place the tray in your mouth. Make sure your teeth will sink into the middle of the rolled putty. Once you press the tray into your teeth, use both thumbs to press the rear of the tray on each end. Press until your teeth on both sides touch the bottom of the tray firmly (then stop pressing).

Press firmly (do not bite) from the rear molars of your thumb, working your way to the front of the tray. You should feel your teeth firmly touch the bottom on both sides all the way around. Put your lip over the tray, and you should feel the putty seeping up your gums on all sides. Once your lip is over the tray, gently press your lip and the sides of your cheek so the putty creates an imprint of your gums (and gum line).

Repeat this process for your remaining 3 impressions.

We know these impressions can be tricky! That's why you should book an impression session with us.

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Want to Win Money Back?

Silly question, we know. Here's how:

- Take a selfie while you're taking your impressions (could be the practice one) and holding our impression kit
- Post it on your Instagram, Twitter and/or Facebook
- Be sure to tag us on IG: **@NewSmileLife** and use hashtag **#HelloConfidence #NewSmileLife #ClearAligners**

We will pick a random winner each month to receive a 3-month supply of teeth whitening!

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Get Ready for Your Closeup

Make sure to grab your mouth spreader from your kit, this will help to ensure that we are getting the right shots of your teeth and bite.

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Step 3: Photo Time

This step is super important. Our licensed dental team uses these photos in combination with your impressions, to analyze your bite and make sure we properly align your teeth **and bite**. Make sure to bite down naturally in all photos. We've all taken a selfie (or 100), so this should be pretty easy.

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We'll Need Photos From These Angles

Quick reminder that when biting down, make sure to bite naturally. Once you've taken photos from these angles, submit them using the link you were sent from our Customer Service Team. Please send in photos as soon as possible, this is an important step to your NewSmile journey.

If you've lost or never received a link, contact our Customer Service Team.
Canada: team@newsmilelife.ca or USA: team@newsmilelife.com

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Canada: team@newsmilelife.ca / USA: team@newsmilelife.com

Lower Teeth from Above

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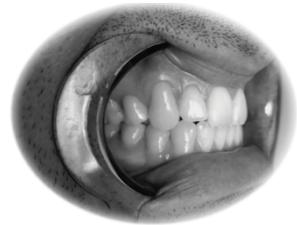
Upper Teeth from Below

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Biting Down from the Front

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Biting Down from the Right

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Biting Down from the Left

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Step 4 :Send Impressions

Congratulations! You finished the biggest step. Now all that's left to do is place your **four impression molds** back the NewSmile box you received. We've already printed a prepaid label for you. Throw that in your mailbox, put your little flag up, and keep an eye out for a confirmation email from the NewSmile Team.

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For More Questions Contact Us:

Facebook :
NewSmileToday

Email Us:
Canada: team@newsmilelife.ca
USA: team@newsmilelife.com

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