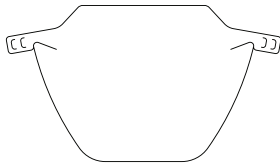
PART A
PARTIE A

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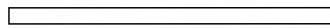
Shield / Bouclier

Head Strap / Sangle de tête

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

Step 1: Inspect & Sterilize - Use either a) alcohol based sanitizer/wipes, b) glutaraldehyde based solutions, or c) 10:1 diluted sodium hypochlorite bleach solution, wipe down both sides of all plastic components. Follow individual label guidelines for contact times - one minute for diluted bleach solution.

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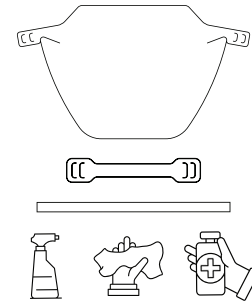
Step 2: Assemble - Wearing gloves, insert the left tab of the Shield into one of the slots on the left side of the Head Strap. Repeat this step with the right tab of the Shield and right side of the Head Strap.

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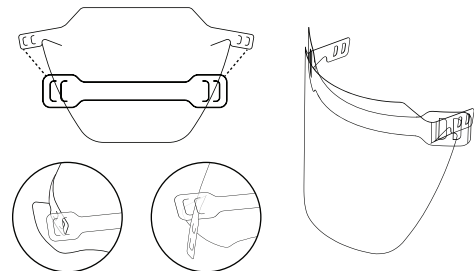Step 3: Attach Elastic - Locate the Elastic Band and weave it through the two slots on the Shield's tabs, from the inside slot to the outside. Repeat this step on the opposite side. Pull the ends of elastic band to tighten.

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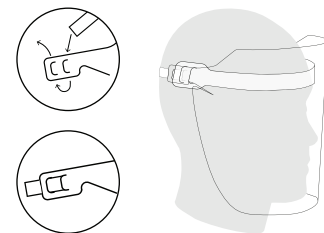** Please see reverse side for sanitizing instructions

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
PART B
 PARTIE B

SANITIZING INSTRUCTIONS
 INSTRUCTIONS DE DÉSINFECTION


Step 1: Disassemble - Wearing gloves, disassemble your Shield, in the reverse order of the Assembly Instructions.

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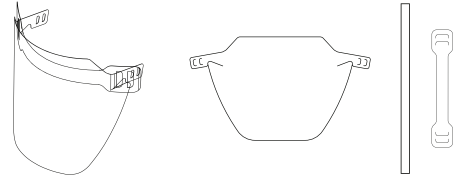Step 2: Clean - Using soap and warm water in a basin or sink, soak all parts of the Shield to remove any organic residue. (If the Shield will not be immediately reused, they can remain soaking indefinitely.)

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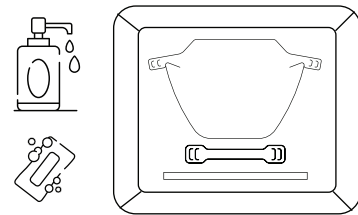Step 3: Sanitize - Dry plastic parts with cloth or paper towel. Use either a) alcohol based sanitizer/wipes, b) glutaraldehyde based solutions, or c) 10:1 diluted sodium hypochlorite bleach solution, wipe down both sides of all plastic components. Follow individual label guidelines for contact times - one minute for diluted bleach solution.

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Step 4: Dry & Store - Store Shield in a sanitary environment, in a disassembled state until next use. Prior to next use, please refer to "Part A - Assembly Instructions" on reverse side.

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