Independent living support  
**vitadora**


- P. 2  User manual
- P. 4  Gebruikershandleiding
- P. 6  Manuel d'instructions
- P. 8  Bedienungsanleitung

- P. 10  Istruzioni per l'uso
- P. 12  Manual de uso
- P. 14  Brugermanual
- P. 16  Bruksanvisning

Independent living support

# vitadora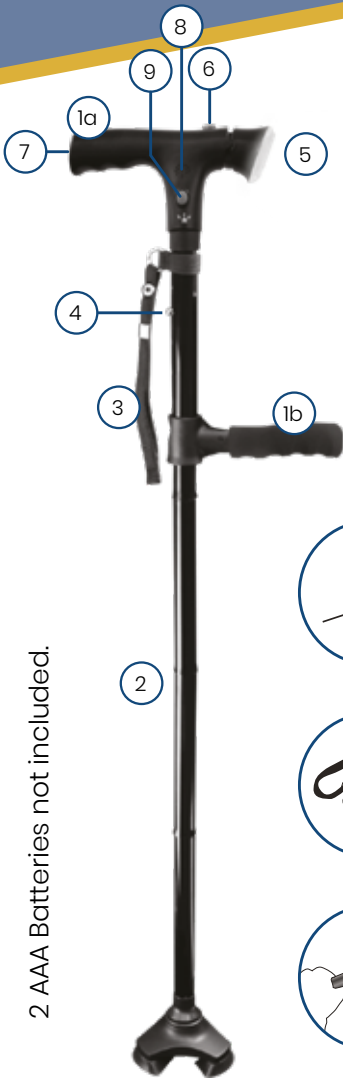




2 AAA Batteries not included.

## Product description

1. a. Handle
- b. Handle with soft foam padding, can be folded 90°
2. Cane sections
3. Wrist strap
4. Length adjustment button
5. LED-lights
6. Light switch
7. Battery compartment
8. Alarm
9. Alarm button

## Unfolding

The Vitadora walking cane has a wrist strap that keeps the sections of the cane together when the cane is folded. To unfold the cane, first unfasten the wrist strap (3). Grasp the handle (1a), hold it up and let the sections of the cane (2) unfold by the force of gravity.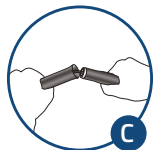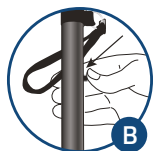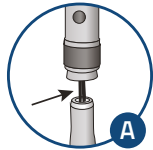

- ❶ **Important:** To prevent falls, always ensure that all of the cane sections (2) are perfectly aligned and locked in place (see Figure A) and the cane is stiff.

**Caution:** Uncontrolled release of the elastic cord (A) inside the cane can cause severe injury to unprotected body parts, particularly the eyes. If the elastic cord breaks, becomes damaged or wears out, stop using the product. Do not stretch the elastic cord more than is necessary to unfold or fold up the cane.

## Adjusting the length to suit your needs

There are five different length settings. To make it longer, hold down the length adjustment button (4) as shown in Figure B, pull up on the handle and release the adjustment button to let it lock into a lower slot. To make the cane shorter, hold down the length adjustment but-

ton (4), push down on the handle and release the adjustment button to let it lock into a higher slot.

## Using the LED-light

An adjustable light (5) is located at the end of the handle (1a). To turn the light on and off, press the light switch (6). The light pivots up and down so you can point it in the direction you wish.

## Using the alarm in emergency situations

To turn the alarm on and off, press the alarm switch (9) located at the head part of the cane handle.

## Folding up

To store the cane, separate the sections as shown in Figure C and group them together. Fasten them with the wrist strap (3). Secure the sections with the fasteners at each end of the strap.

## Insert and change batteries

This cane operates with 2 AAA batteries (not included). The battery compartment (7) is located at the end of the handle (1a), open it and insert or change batteries.

- ❶ **Important:** Low battery power will cause the light and alarm functions to lose their effectiveness. Please change to new batteries whenever the brightness or loudness of alarm weakens.
- Alkaline batteries are recommended as a stable power source. Non-Alkaline and rechargeable batteries have a tendency to decrease in power level over time, and you might mistakenly think the light or alarm is not working properly.



- Remove the batteries to prevent them from running down whenever you are planning to store the cane for more than 3 months.

## Safety instructions:

- Always consult your doctor before using a walking cane.
- Never use the cane on ice, wet or slippery surfaces.
- Do not disassemble the body (with elastic cord) of the cane.
- The product is not for medical use.
- Not suitable for skiing.
- This product does not aim to be a walking aid as per EN 1985 (on hiking sticks).
- Do not use the cane is defective.
- **Not intended for persons over 115kg.**

## DUAL HANDLE USAGE

1. Spread handle 1b and place your hands each on one handle
2. Lean forward
3. Push yourself up

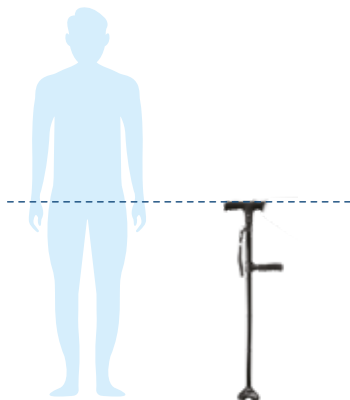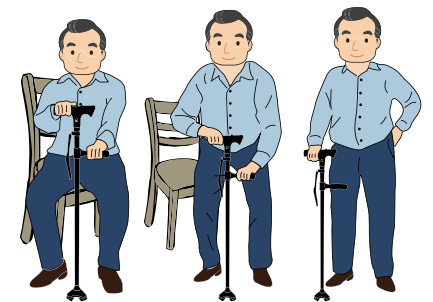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Imported & distributed by OCG Products B.V.,