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### Intended use

edelweiss CAD/CAM BLOCK is used to manufacture highly aesthetic, indirect single tooth restorations using CAD/CAM technology.

### Introduction

Through a state of the art manufacturing process, edelweiss CAD/CAM BLOCKs consists of a single glass-phase embedded in a hybrid matrix. As a result, the esthetic properties are similar to that of feldspathic glass ceramic without having the brittleness of pure ceramics.

### Composition

Barium dental glass

Bis-GMA based hybrid matrix

Pigments

Additives

Catalyst

### Technical data

Filler ratio: 82 % by weight = 65 % by volume. Inorganic filler particle size range between 0.02-3 µ.

### Shade

Available in 5 shades.

edelweiss T-BLOCK (Translucent block) in enamel shade

edelweiss C-BLOCK (Chroma block) in shades from A0, A1, A2 and A3.

edelweiss i-BLOCK in enamel and chroma shades, A0, A1, A2 and A3.

### Block Size

Available in one universal size: 12x14x18 mm

### Indications

- Veneers
- Thin veneers
- Occlusal veneers
- Inlays, onlays
- Partial crowns
- Anterior/posterior crowns
- Implant supported restorations: screw retained crowns or mesostructure and crown

### Recommended application

#### 1. Preparation design

- Minimum thickness at walls should be at least 0.5 mm. For veneers, minimal cervical thickness of at least 0.3 mm.
- Prepare margins with chamfer or rounded shoulder preparation.
- For inlays and onlays, all internal edges and angles should be rounded. Avoid having margins in direct occlusal contact with the opposing tooth.
- Minimum thickness of the restoration should be 1.5 mm in pit and fissure areas and 1.5 mm in cusp areas.

#### 2. Milling process

For information on scanning and CAD/CAM processing, please refer to the respective Instructions for Use and the manuals of the respective CAD/CAM system. The instructions by the manufacturer must be observed. Select the appropriate program for Hybrid Blocks/Composite Blocks and mill in accordance with the instructions for use by the manufacturer.

Following the milling process, check for any defects. If there are any defects like cracks or chipping, the restoration should be discarded.

If the block size, grinding or milling parameters are not available, please contact your CAD/CAM system provider in advance.

#### 3. Finishing of milled restoration

For finishing the restoration, appropriate grinding/finishing instru-ments are needed. Smooth out the attachment point of the block with fine-grain diamond burs paying particular attention to the proximal contacts. If necessary, carry out individual shape adjustments and smooth out the surface structure created by the CAD/CAM. Final polish is achieved using cotton/linen buffs.

#### 4. Pretreatment of the edelweiss CAD/CAM restoration

- Verify the fit of the restoration on the tooth.
- Coat the prepared fitting surface of the edelweiss restoration with edelweiss VENEER Bond using an applicator tip/microbrush and rub-bing it gently for at least 30 seconds.
- Gently dry edelweiss VENEER Bond using an oil and water free stream of air, so that a slightly moist layer remains on the inner surface of the edelweiss restoration without any pooling of the VENEER Bond.
- Light cure edelweiss VENEER Bond for at least 20 seconds using a light curing device.
- If conventional resin cements are used ex. Calibra Ceram, replace the edelweiss VENEER Bond with the manufacturer’s recommended bond ex. Prime&Bond Active).

#### 5. Pretreatment of prepared tooth surface

- Isolating and cleaning the preparation: During adhesive luting/ce-menting protocol thorough isolation of the operating field – preferably with a rubber dam or alternatively with cotton rolls and a saliva ejector.
- Clean the tooth surface using a polishing brush and an oil- and fluo-ride-free cleaning paste and rinse with water spray. Then lightly dry with water- and oil-free air. Avoid over-drying.
- Apply 37 % phosphoric acid gel to the prepared enamel and then flow the etchant onto the prepared dentin. The etchant should be left to react on the enamel for 15–30 seconds and on the dentin for 10–15 seconds.
- Rinse thoroughly with water and gently dry with compressed air.
- Starting with the enamel, thoroughly coat the tooth surfaces to be treated with Dentin Bonding Agent/Adhesive. The adhesive must be gently scrubbed into the tooth surface for at least 20 seconds and light cured as per manufacturer’s instructions.

#### 6. Cementation of edelweiss CAD/CAM restoration

In order to guarantee optimum esthetic results, it is highly recom- mended to use edelweiss NANO-HYBRID COMPOSITE.

- T-BLOCK (Translucent):** The final shade matching of the resto- ration can be adjusted using the appropriate composite shades. This can be further individualized by varying the composite shades accord- ing to cervical and incisal color variations in the natural tooth. Individual characterizations can be also accomplished using edelweiss EFFECT SHADES or other similar staining kits.
- C-BLOCK (Chroma):** These are color matched to correspond to shades A0, A1, A2 and A3 and can be cemented using edelweiss Enamel shade composite. For substrates that are severely discolored a dentin shade is recommended to provide a more homogenous color.
- Take an appropriate amount of the composite strip in the fingertips and roll into a ball. Using the finger tips, press the ball onto the inside of the restoration. Use the spatula instrument to evenly distribute the composite into the restoration.
- Gently seat the restoration onto the prepared tooth surface and retain it in place exerting uniform pressure.
- Light-cure with a curing light for 2 seconds and remove all excess com- posite material
- Final light-cure for 20 seconds on all surfaces and margins with a curing unit with light intensity of 1000 mW/cm²
- Cementation can also be achieved using conventional resin luting ce-ments. If conventional resin cements are used ex. Calibra Ceram, it is recommended to use manufacturer’s corresponding bonding agent ex. Prime&Bond Active (Dentsply).

#### 7. i-BLOCK (implant block):

The edelweiss implant block features a built-in drill hole that integrates precisely to a titanium base interface or similar attachment.

- Abutment crown as a single unit (Crown + titanium base): The CAD/ CAM fabricated crown can be bonded onto the titanium base ex- tra-orally. Screw the crown/titanium base attachment onto the im- plant. Seal of the screw channel with composite intraorally.
- Crown and mesostructure as 2 units: The CAD/CAM manufactured mesostructure can be bonded onto the titanium base extra-orally and then screwed onto the implant with the abutment screw intra-orally.
- For cementing to titanium base follow the instructions of the manufac- turer.

#### 8. Finishing and polishing the completed restoration

After having adhesively cemented the restoration, adjust occlusion/ articulation with suitable finishing instruments.

Work the interproximal areas with finger-strips then polish with polish-strips.

Polish the cervical areas with silicone-polishing cup. Final polishing is done with silicone polishers and cotton/linen buffs.

### Contraindications

Contraindicated in the case of any hypersensitivity or allergy to any of the ingredients.

### Side effects

No systemic side effects are known. Contact allergies with products of similar composition have been reported in isolated cases. In such cases discontinue use and contact a physician.

### Shelf-life

Expiration date and the lot number is marked on the containers and packaging. Do not use after the date of expiration.

It is recommended for optimal performance, to be stored at room tem- perature away from direct sunlight and high humidity.

### Important

To ensure optimal clinical results, we recommend that all edelweiss components in this system are used in conjunction with each other.

<p><b>Note</b></p> <p>May only be supplied to dentists and dental laboratories or on their behalf. Keep away from children! Only for use by dentists!</p>
---

Date of issue of this package information leaflet

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