

RIPsnore™

Rest in peace. Australia's original anti-snoring device.



Ripsnore: Mandibular Advancement Splint (MAS)

MAS Treatment for Mild & Moderate Obstructive Sleep Apnoea (OSA) & Snoring


The Ripsnore™ Device

Ripsnore™ was originally developed in Australia by a team of professionals who noticed a need for an affordable, high quality and comfortable mandible splint solution, that provides both snoring relief and treatment for mild/moderate obstructive sleep apnoea. When CPAP fails, mandible advancement splints are usually the next option, along with weight loss and positional sleep, or a combination of all treatments.

Without costing a fortune, *Ripsnore™* is a "first step" approach to stopping or reducing snoring, mild/moderate OSA, and teeth grinding (bruxism). While other similar devices can cost thousands of dollars, *Ripsnore™* is available at an incredible low price with a complete money back guarantee.

How it works

Ripsnore™ is designed to hold the lower jaw slightly forward, thus moving the base of the tongue away from the back of the airway and soft palate. This allows the throat a clear passage for easy breathing to silence snoring, and to assist with mild/moderate OSA. As *Ripsnore™* fits over the teeth it can be also be used to treat teeth grinding

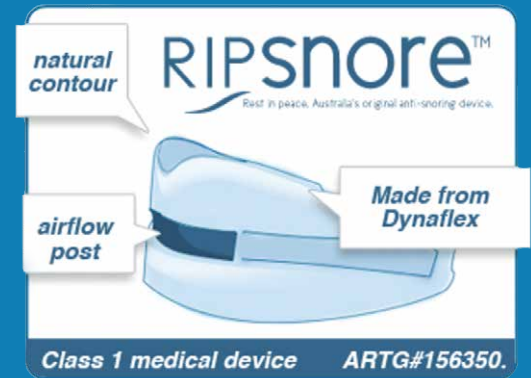


Ripsnore™ is a very effective treatment for stopping or reducing snoring and aiding mild/moderate Obstructive Sleep Apnoea (OSA).

If patients have mild or moderate sleep apnoea, they should try Ripsnore as a cost effective first step. If your patients are diagnosed with chronic Obstructive Sleep Apnoea, then they should seek professional medical advice.


Fitting Ripsnore™

Patients will receive full instructions on fitting the device when ordered. The product is prepared for moulding by placing it in boiling water to soften the base material. This process ensures the very best results for obtaining the perfect individual fitting. Unlike pre-moulded devices, which are universal fittings, the *Ripsnore™* device is a perfect mould of your unique teeth and mouth cavity – providing the most superior and comfortable fitting.

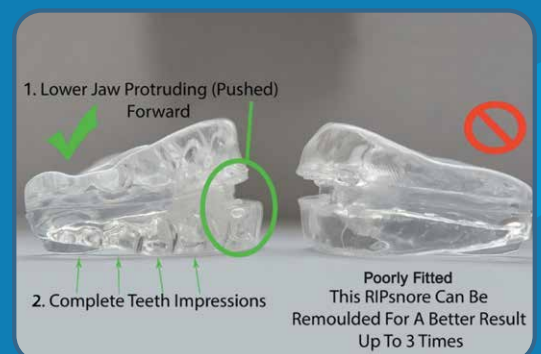




Xrays without and with Ripsnore™ fitted

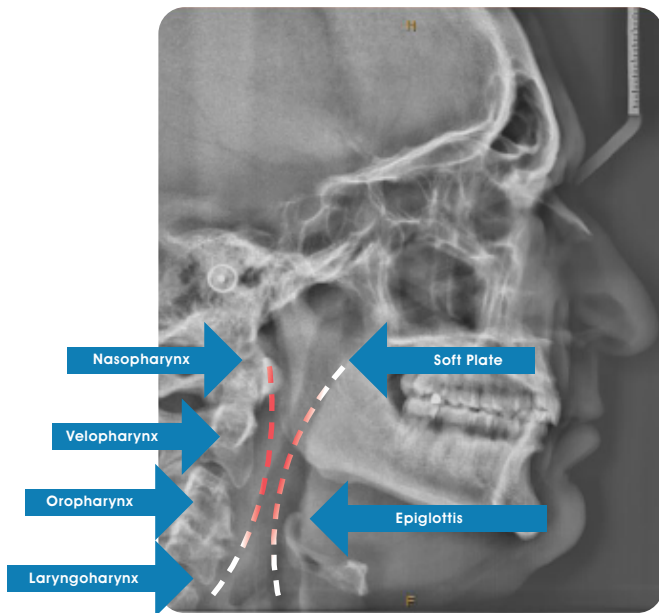

Neutral Lower Jaw Position

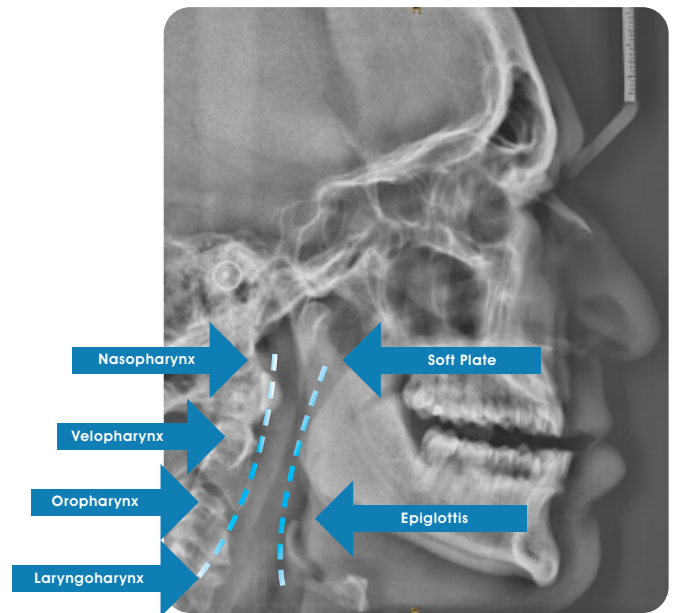

Jaw Thrust Position with Ripsnore™ Opening the Airways

Don't hesitate to reach out!

ACN 129 964 950

ABN 49 129 964 950

Please contact us via email:
orders@ripsnore.com.au for an immediate response.

Australia:

Phone: 1300-473805

Fax: 07 3112 1932

PO Box 16283, City East
Brisbane, QLD, 4002

www.ripsnore.com.au

