


# PUTTING ON GOWN



**1.**

Perform Hand Hygiene.

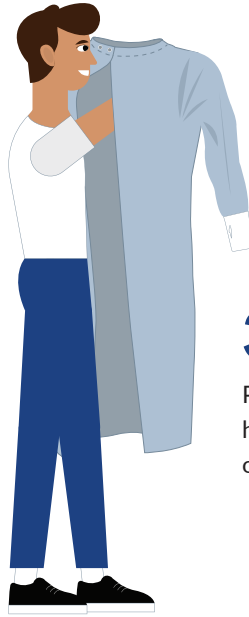


**3.**

Place the gown over head with vertical opening in back.




**4.**

Extend arms through the sleeves and insert hands into the wrist guard and thumb hole for secure closure.

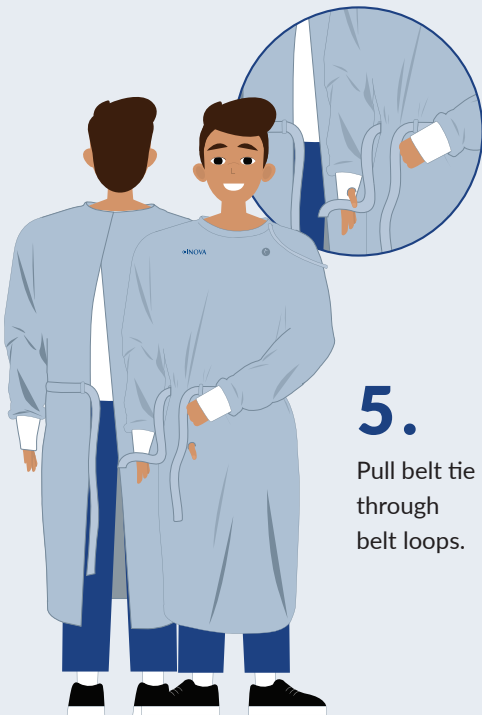

**5.**

Pull belt tie through belt loops.

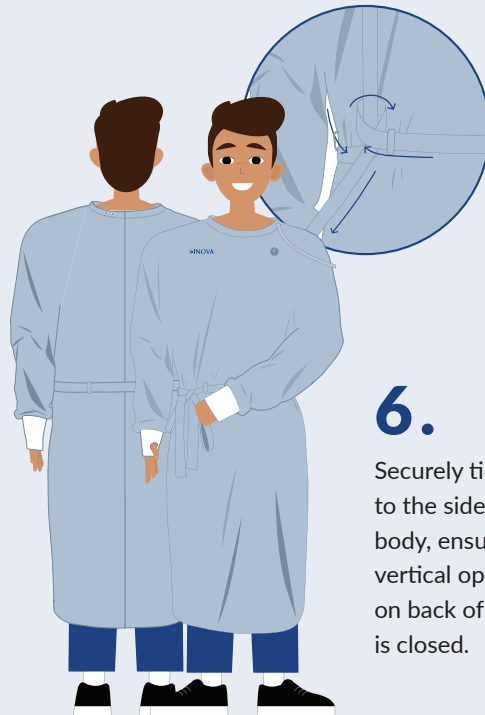

**6.**

Securely tie belt to the side of the body, ensuring vertical opening on back of gown is closed.



**7.**

If eye and/or mask protection is required, don at this time according to established procedures.



**8.**

Perform hand hygiene. Put on gloves and extend to cover the wrist.

SCAN FOR INSTRUCTIONAL VIDEO




la forma

www.LaformaMed.com | (844) 897-5199 | info@LaformaMed.com


Inova™

# REMOVING GOWN

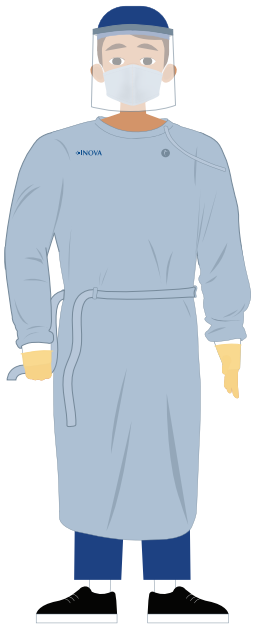

**1.**


Untie belt, allowing the back of the gown to open.

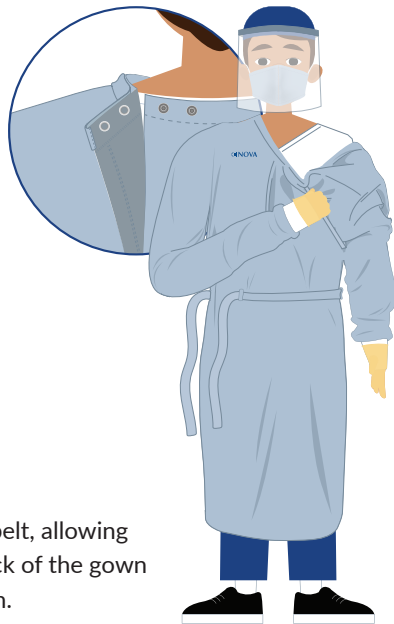

**2.**

Snaps along neck line are connected to an easy to grasp pull-cord.

Remove the gown by pulling down on the pull-cord in one swift motion, disconnecting the snaps.

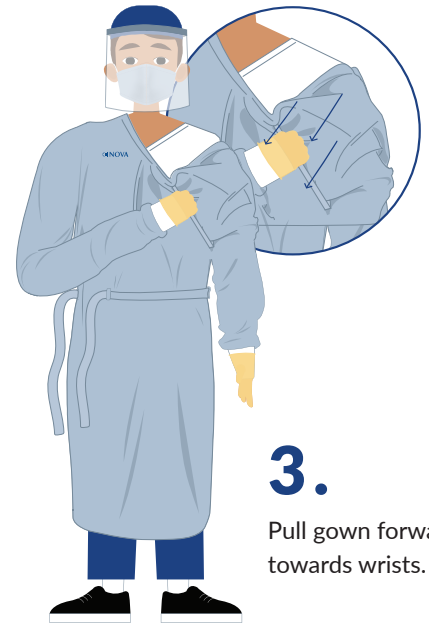

**3.**

Pull gown forward towards wrists.

**4.**


Remove gloves and dispose in the waste container.

**5.**

Allow the gown to slide down off the arms inside out. Touching only the inside of the gown, roll the gown on itself into a ball and place in linen hamper.

**6.**

Perform Hand Hygiene

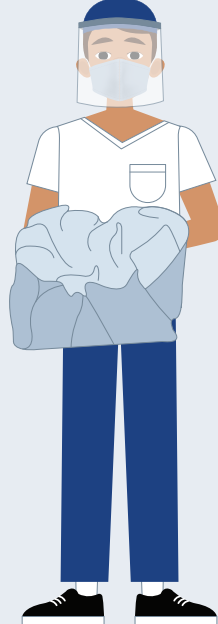

**7.**

If eye and/or mask protection is required, doff at this time according to established procedures.

**8.**

Perform hand hygiene immediately after removing all PPE.



SCAN FOR INSTRUCTIONAL VIDEO



la forma

www.LaformaMed.com | (844) 897-5199 | info@LaformaMed.com

Inova™