


**CLIA WAIVED
AVAILABLE!**

UScreen²

DRUG TEST CUP

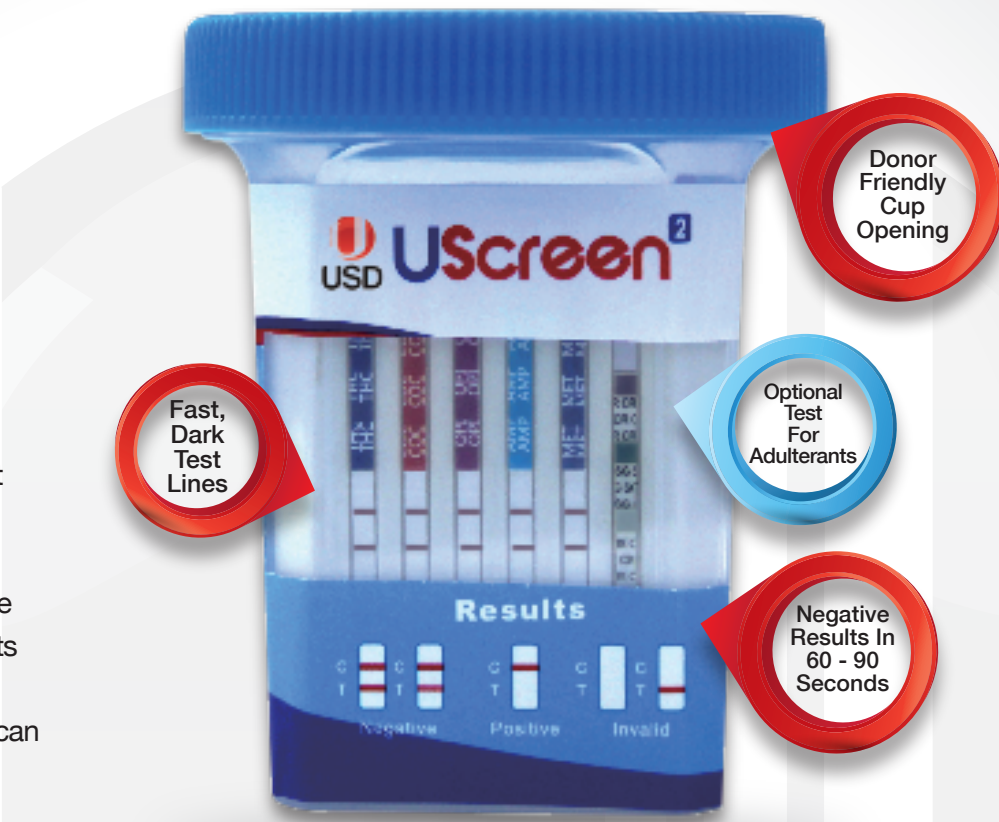

A lot of instant screens brag that you get negative test results in around 5 minutes. We don't call 5 minutes instant! UScreen urine tests will give you negative results with solid, dark test lines in 90 seconds or less! And UScreen can be ordered with an optional test strip for adulterants.

UScreen Features

- Tests for Up to 12 Drugs
- Fast, Dark Test Lines
- Individual Test Strip for each Drug
- Most Results Within 60-90 seconds
- Non-negative Results in 5 Minutes
- Donor Friendly Cup Opening
- Optional Test for Adulterants
- Remarkable Pricing & Quality

UScreen Cup Tests Available For:

Marijuana	(THC)	Amphetamine	(AMP)
Cocaine	(COC)	Methamphetamine	(MET)
Opiates 300	(MOP)	Barbiturates	(BAR)
Opiates 2000	(OPI)	Benzodiazepines	(BZO)
Phencyclidine	(PCP)	Methadone	(MTD)
Oxycodone	(OXY)	Buprenorphine	(BUP)
Tricyclic Antidepressants	(TCA)	Propoxyphene	(PPX)
Ecstasy	(MDMA)		

Using the UScreen Cup Drug Test

Step 1

Have the donor provide a specimen and return the cup to the collector. Check the temperature strip to assure valid specimen.



Step 2

Replace the cap and set the cup on a flat surface and remove label. If test cup is equipped with an optional adulterant test strip, please replace cap and proceed to Step 3 prior to Step 4. If not, proceed to Step 4.



Step 3

If equipped, place the slot on the adulterant color guide over the adulterant strip on the cup. If any of the test pads match any of the colors on the left, adulteration is indicated. In this case, collect a new specimen in a new cup and retest.

TEST AND READING TIME	ABNORMAL		NORMAL
	LOW	HIGH	
Creatinine	Negative 10		10 100 200mg/dl 1:CR CR C 1:CR
Specific Gravity	1.000	1.003	1.015 1.025 1:SG SG 1 1:SG 1:SG
Oxidant	Positive		Normal 1:OX OX 0 1:OX


Step 4

Interpret results. Negative results can be interpreted as soon as all test lines are visible. Do not interpret positive results before 5 minutes.



Reading the Results

Read the results when all upper "Control" or "C" lines have appeared. Negative results can be read as soon as all "Test" or "T" lines are visible. (Wait 5 minutes to determine a Positive Result)

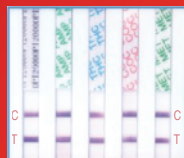

Negative

A negative result is indicated by the presence of the lower "Test" or "T" Line for each designated drug.




Negative

The presence of even a very light "Test" or "T" Line indicates a negative result.



Positive

A positive result is indicated by the absence of the lower "Test" or "T" Line for a specific drug. Wait 5 minutes to determine a positive result.

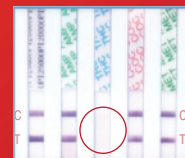

Invalid

An invalid result is indicated by the absence of the upper "Control" or "C" Line. If this occurs run a second test.