

# ADEPT MEDICAL APPROVED DISINFECTANTS

**WARNING - DO NOT USE ANY CLEANING METHOD THAT EXCEEDS A TEMPERATURE OF 65 °C**

## LIQUIDS

ORION Laboratories - 70% Isopropyl Alcohol\* - [MSDS](#)  
 \*Do not use 70% Isopropyl Alcohol on STARBoard with an IBN product code lower than IBN001536

Jaychem Industries - 2% Chlorhexidine / 70% Alcohol (Pink) - [MSDS](#)

Betadine - Povidone Iodine (7.5%) - [MSDS](#)

C-Tec - Nature Clean (Diluted 1:10 with water)

BODE Chemie GmbH - Kohrsolin FF - [MSDS](#)

STERIS Coverage Spray TB\* - [MSDS](#)

\*Do not use STERIS Coverage TB on STARBoard with an IBN product code lower than IBN001536

CaviCide by Metrex Research - [MSDS](#)

## DISSOLVABLE

Du Pont - Rely on Virkon Tablets - [MSDS](#)

## SURFACE WIPES

PDI Sani-Cloth Bleach - [MSDS](#)


PDI Sani-Cloth AF3 - [MSDS](#)

Reynard Health Supplies - Surface Disinfectant Wipes - [MSDS](#)

Diversey - Oxivir Tb Wipes - [MSDS](#)

CaviWipes by Metrex Research - [MSDS](#)

Clinell Wipes - [MSDS](#)

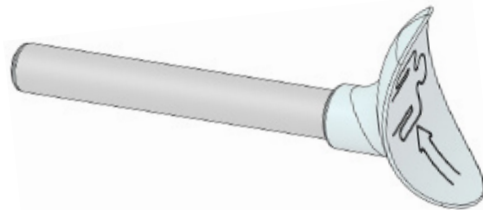

## DISCOLORATION

Some disinfectants may cause slight discoloration to the soft blue parts on the STARSupport. They will not effect the strength and will remain fit for purpose.