



ADEPT MEDICAL APPROVED DISINFECTANTS

Approved Disinfectant list is specific for early release STARBoards bought prior to IBN001536. Please check product label to confirm and if unsure, check with local distribution agent.

WARNING - DO NOT USE ANY CLEANING METHOD THAT EXCEEDS A TEMPERATURE OF 65 °C

APPROVED

LIQUIDS

- Jaychem Industries - 2% Chlorhexidine / 70% Alcohol (Pink) - [MSDS](#)
- Betadine - Povidone Iodine (7.5%) - [MSDS](#)
- C-Tec - Nature Clean (Diluted 1:10 with water)
- BODE Chemie GmbH - Kohrsolin FF - [MSDS](#)
- CaviCide by Metrex Research - [MSDS](#)

SURFACE WIPES

- PDI Sani-Cloth Bleach - [MSDS](#)
- PDI Sani-Cloth AF3 - [MSDS](#)
- Reynard Health Supplies - Surface Disinfectant Wipes - [MSDS](#)
- Diversey - Oxivir TB Wipes - [MSDS](#)
- CaviWipes by Metrex Research - [MSDS](#)
- Clinell Wipes - [MSDS](#)

DISSOLVABLE

- Du Pont - Rely on Virkon Tablets - [MSDS](#)


NOT APPROVED

DIVERSEY VIREX TB - [MSDS](#)

70% ISOPROPYL ALCOHOL - [MSDS](#)

STERIS COVERAGE SPRAY TB - [MSDS](#)

These disinfectant cleaners are not approved by the manufacturer as they may cause damage to some of the STARSystem components after prolonged use.

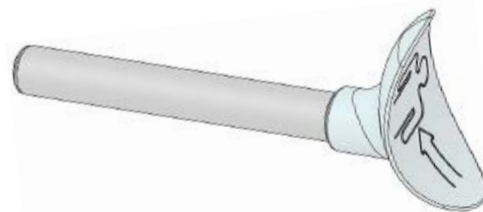

DISCOLORATION

Some disinfectants may cause slight discoloration to the soft blue parts on the STARSupport. They will not effect the strength and will remain fit for purpose.