

# ADEPT MEDICAL TRAINING PLATFORM



AdeptMedicalTraining.com

## About the Training Platform

The Adept Medical Training Platform has been produced to assist Adept Medical's Table Accessories users by providing information on our product portfolio.

With a step-by-step pictorial guide on unboxing, set-up, usage, cleaning, re-boxing, repairs, warranty and replacement parts, it has been developed to support our customers in every step.




For a better navigation experience, we recommend to use Google Chrome, Safari, Firefox browsers when using the Adept Medical Training Platform.



# Information Overview

On The Training Platform, you will find relevant information about the company and its team, information on how to use the product range, related downloadable documents and product support information.

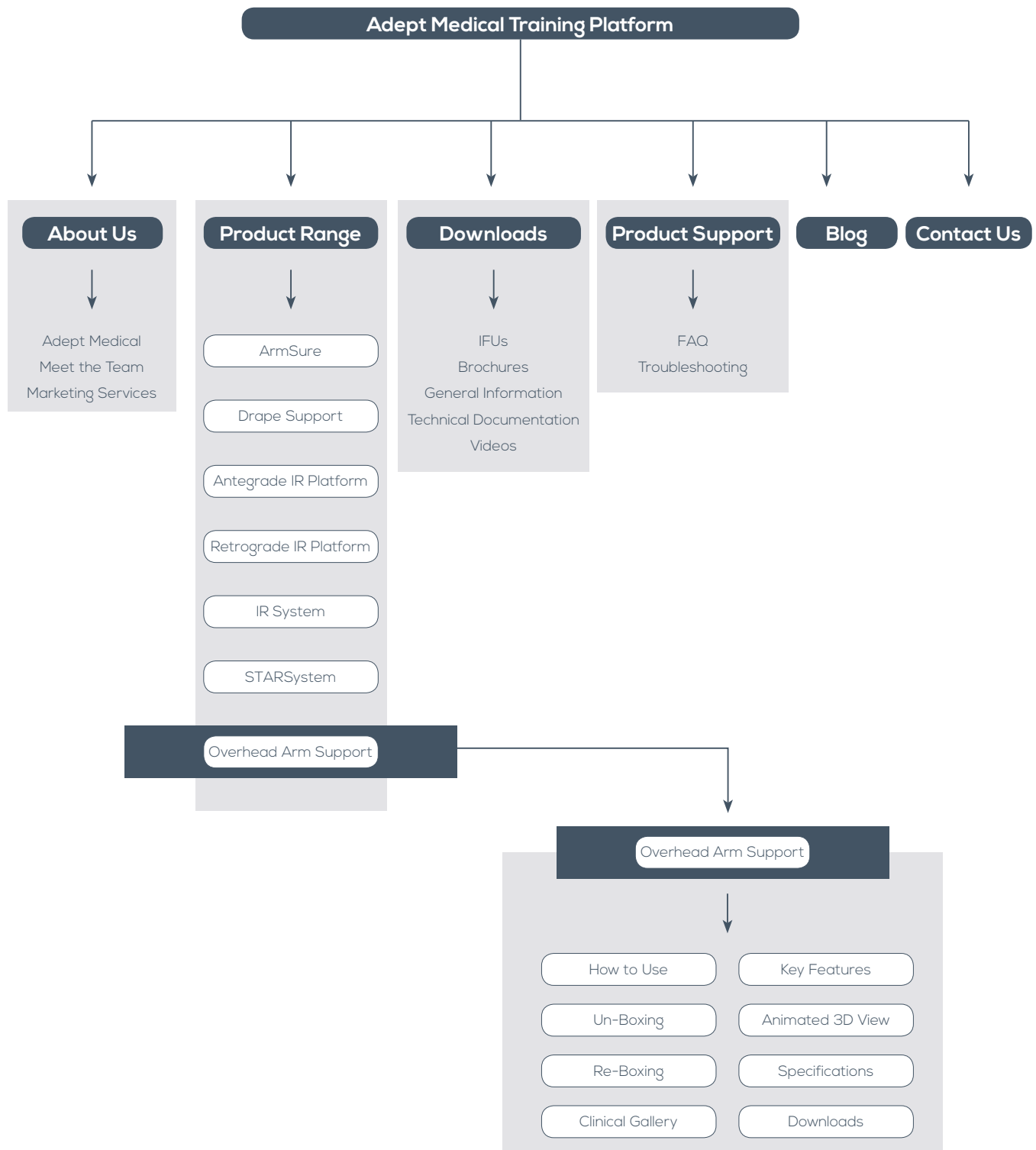

## PRODUCT RANGE



### ArmSure

The ideal system for arm immobilisation, protecting the sterile field, with the added benefit of reducing scatter radiation. Particularly useful in the increasing number of procedures performed under conscious sedation where patient restlessness can be an issue.

[Learn More](#)


### Drape Support

Designed to support a sterile drape over the patient, the extremely versatile Drape Support offers a wide range of imaging table placement options. With two articulating joints, it can be easily manoeuvred to suit procedural and patient requirements.

[Learn More](#)


### Antegrade IR Platform

An ideal work surface for antegrade femoral approach.

[Learn More](#)

### Overhead Arm Support



Designed to comfortably support the supine patient's arms, it eliminates extreme shoulder flexion, allows abdominal access and removes unwanted artefacts when imaging with C-Arm or CT machines. This versatile device will support one or both arms in the supine position and a single arm when used with the posterior oblique position.

The Overhead Arm Support provides ease of use for the operator and facilitates improved patient comfort, procedural outcome and repeatability.

[How to Use](#)
[Key Features](#)
[Unboxing/Reboxing](#)

3D

[Promo Video](#)

#### Product Details

[Specifications](#)
[Clinical Gallery](#)

#### Downloads

[Instructions PDF](#)
[Brochure PDF](#)

## Overview

On each product you can navigate through the Key Features, Specifications, How to Use, an animated 3D view, clinical images, promotional content and relevant PDF downloadable content.

## Key Features

The product Key Features helps to give a general overview of the features and benefits of each product.



- Supports the patient's arm to allow abdominal access and removing artefacts from the imaging site.
- Designed to be used with rotating table and imaging equipment, the device has been engineered to fit inside the bore of standard CT machines and can be used with any C-Arm imaging centre.

- The device can support one or both arms.
- Soft, pliable, latex free polyurethane straps may be used for additional patient security and comfort.



- Can be used with the patient in both the supine position and posterior oblique position.
- Daggerboard placement allows for ease of placement, which can either be beneath or above the mattress.

# Learn How to Use the Product Range

Each product page provides a How to Use section where you can find detailed information including pictures, diagrams, videos and descriptions to provide steps on product usage.

The How to Use section provides key information on important processes related to our product range, including assembly, placement and patient positioning.

ADEPT MEDICAL   Home   About Us   Product Range   Downloads   Trial Info   FAQ   Contact Us




## How to Use

### Overhead Arm Support

#### Attaching the Locking Leg Assembly to the Daggerboard

Insert the locking leg into the corresponding component located on the daggerboard. Ensure the locking mechanism is facing away from the daggerboard. This will allow the clinician to adjust the height during patient positioning.


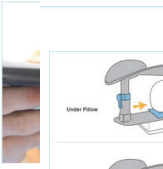

Check underneath the daggerboard that the release clip is fully engaged, ensuring complete security of the locking leg assembly.

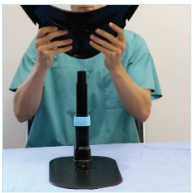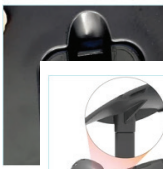

#### Attaching the Armrest Wing

To attach the Armrest Wing to the locking leg assembly, align the Armrest Wing on the locking leg assembly post and gently push down until it is completely engaged.


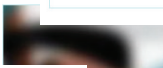

The area located underneath the Armrest Wing should be aligned to the top section of the locking leg.



#### Daggerboard Placement


To insert the daggerboard between the mattress and the image table surface, lift the mattress slightly and slide the daggerboard fully under the patient's head.

The daggerboard can also be inserted underneath a pillow on top of the mattress. Simply slide the daggerboard fully underneath the pillow into the correct position.



Videos, images and diagrams illustrate the best way to use the product.


Detailed descriptions will give you a better understanding on how the product is used.

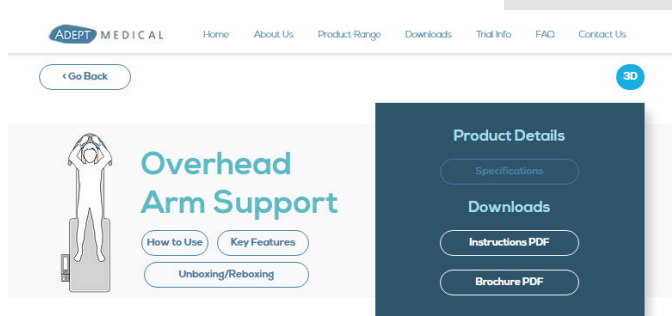

## Product Specifications

You will find all the product specifications including diagrams, dimensions and weights. There is also important information on IBN codes, including their locations on each product.

## Specifications

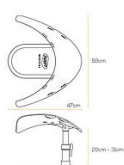

Overhead Arm Support

**Dimension**  
20cm (31cm) x 47cm x 59cm

**Weight**  
15kg



## IBN CODE INFORMATION

The IBN number is a unique serial number issued under each product. This will be recorded against the hospital by the issuing company. You will be asked for this if any components require repair.

Your IBN is located on this label on your product

### IBN Code Examples

IBN Product Label



IBN Product Label Example



IBN Carton Label Example



## How to Care for the Product


This section contains information about how to care for the products, and our listed range of approved cleaners. This ensures complete sterility between cases and that they will be returned correctly after trial periods.



## CARING FOR YOUR PRODUCT



High performance engineering plastics and carbon-fibre composite have been selected for our product range, giving superior rigidity, strength and durability, which ensures that the products have a long life. As a result, the products have low radiolucency, are light-weight yet strong, and have excellent resistance to chemical attack associated with commonly used cleaning products.

[Cleaning Instructions](#)

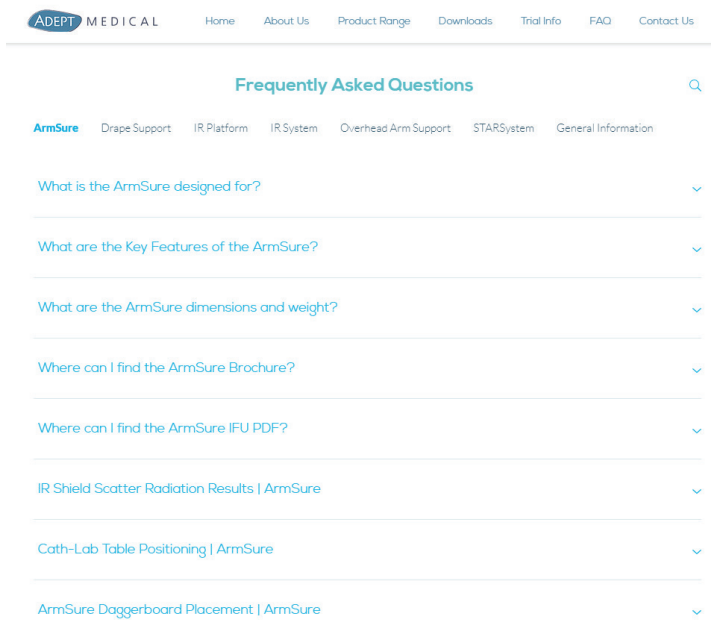

## Easy Navigation

The FAQ is designed so you can easily find the relevant information related to your product queries. The search function allows customers to search by key words, so information can be efficiently accessed. The FAQ is categorised by product to ensure you can navigate to the information required.

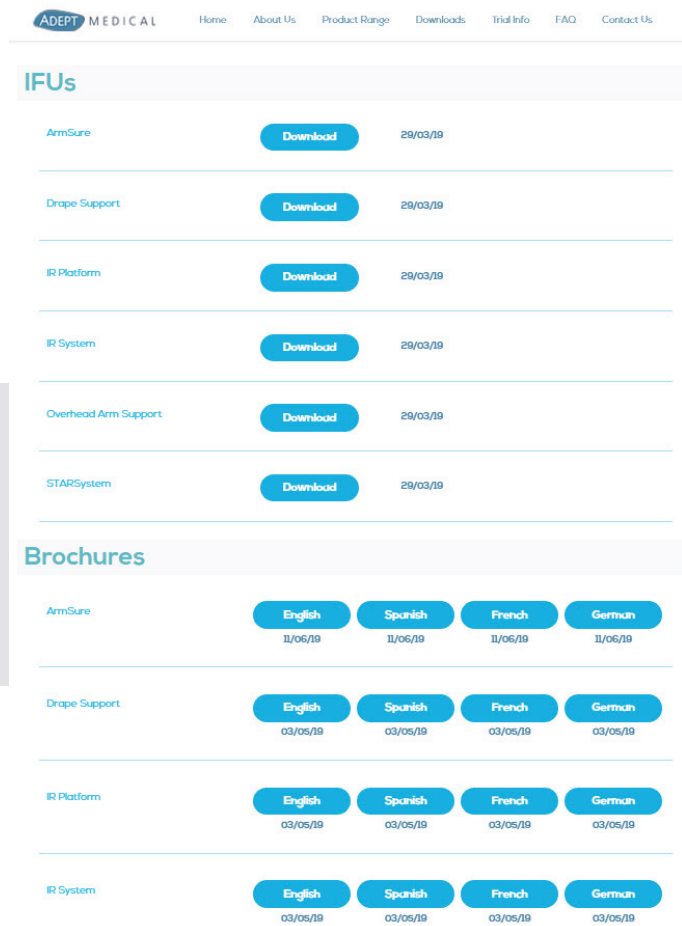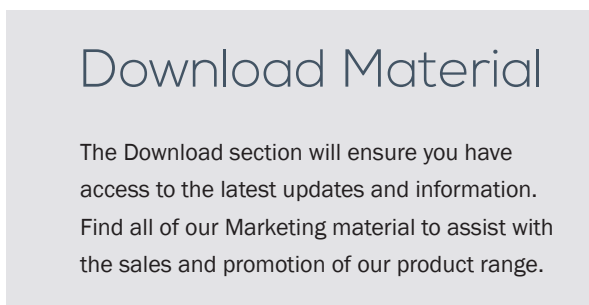




## 3D

ADEPT Medical Overhead Arm Support by ADEPT

3D

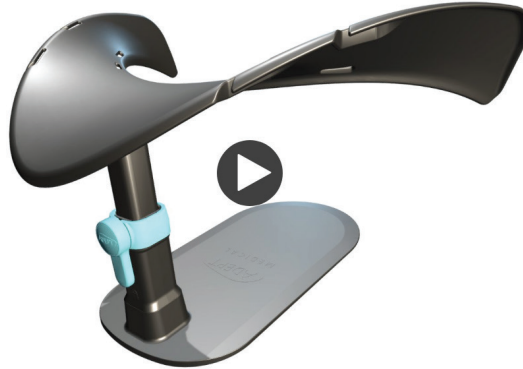

## Animated 3D View

This tool facilitates a better understanding of the product through an interactive and animated 3-D view.

## Un-boxing &amp; Re-Boxing

This section has step-by-step instructions for the un-boxing and re-boxing of our products.



## TRIAL INFORMATION

If you are undertaking a trial, you will need to retain the original box you received your product in, the inner packaging and correctly re-box your product.

Ensure that all components of the product have been thoroughly cleaned and free of any contaminants before returning at the end of your trial period.

## Unboxing




## Reboxing



## MARKETING SUPPORT

Adept Medical offers marketing support to all our distributors for the promotion of our Adept Medical Table Accessory product range. If you require the Adept Medical catalogue, a pull-up banner, Hi-Res images, brochures, or other marketing collateral with your branding, we are happy to assist you.

Enquiries

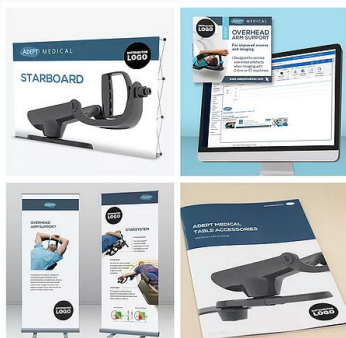

The Adept Medical marketing team is happy to guide you with any artwork requirements or requests, including:

- Table Accessories catalogue branded with your logo
- Individual product brochures branded with your logo
- Artwork for digital banners
- Artwork for pull-up banners
- Artwork for backdrops
- High-Res imagery suitable for websites or print

## Marketing Enquiries

Adept Medical offers marketing support to all our distributors for the promotion of our Adept Medical Table Accessory product range. If you require the Adept Medical catalogue, a pull-up banner, Hi-Res images, brochures, or other marketing collateral with your branding, we are happy to assist you.

## Video Downloads

All relevant Adept Medical video content can be downloaded, giving you the ability to access the videos on your own devices and take them on site visits to demonstrate the product range.

### Promo Videos

⋮



**Drape Support Promo** | 02:06

Versatile sterile drape management. Designed to support a sterile drape over the patient...



**ArmSure Promo** | 01:35

The ideal system for arm immobilisation, protecting the sterile field with the added b...



**STARSystem Promo** | 02:52

A combination of the STARboard, STARSupport and STARTable, the STARSy...



**Overhead Arm Support Pro...** | 02:27

For improved access and imaging, Adept Medical introduces the Overhead Arm Sup...



**IR System Promo** | 02:26

Adept Medical introduces the IR SYSTEM, comprising the Adept STARboard, Extensi...



**IR Platform Promo** | 01:20

The IR Platform is the ideal over patient work surface for Vascular and Interventional Rad...



## TROUBLESHOOTING

### COMMON MISTAKES

[ArmSure](#)[IR Shield](#)[X-ray Shield](#)[Overhead Arm Support](#)[Drape Support](#)[STARSystem | STARBoard](#)[IR Platform](#)[STARSystem | STARTable](#)[IR System | STARBoard](#)[STARSystem | STARSupport](#)

## Troubleshooting

The Troubleshooting section highlights common mistakes made when setting up and using the product range and has been developed to help users troubleshoot when using the product range.

### ArmSure

Common Mistake	Solution
The daggerboard is not pushed fully underneath the mattress, causing product instability.	To ensure correct patient positioning and product security throughout the procedure, push the daggerboard fully underneath the mattress, so the IR Shield slot is firmly against the mattress.
The ArmSure is not inserted underneath the table at an appropriate position, causing difficulty when positioning the patient.	Ensure the ArmSure is placed underneath the mattress so the patients' hand does not hang below the arm support surface and is fully supported by the arm support.


## ADEPT MEDICAL BLOG

Updates/Events/News

## Adept Medical Blog

Discover Adept Medical's latest news, including product launches, product updates, events, awards and upcoming tradeshows.


[All Posts](#) [Articles](#) [Products Improvements](#) [Launches](#) [Events](#) [Awards](#) [Q](#)




Jul 17 • 1 min

Adept's Matt Lazenby is interviewed by LINNC TV during LINNC 2019

Matt Lazenby, Adept's Product Development Manager, was interviewed by LINNC Online in...

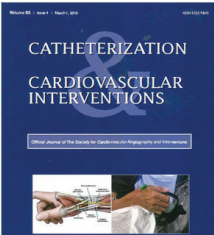

Jun 7 • 1 min



Jun 5 • 1 min

Image Table Accessories at LINNC, Paris 2019

The Adept Medical Team has had a great exhibit at LINNC in Paris, an Interventional...

## Coming Soon

Detailed cleaning instructions

Product Maintenance information

Distributor login area housing files specific to your organisation