

Concrobium Broad Spectrum Disinfectant II is a germicidal detergent formulated to clean, disinfect and deodourize in a simple one-step process. Use Concrobium Broad Spectrum Disinfectant II for all disinfection of hard, non-porous surfaces in residential, coils and drain pans* of HVAC & Equipment, restoration, decontamination and remediation sites (Fire, water, sewage, trauma scene decontamination and remediation)**, healthcare facilities, hospitals, intensive care, operating rooms, emergency areas, ambulances, dental and medical offices, dental labs, dental operatories, nurseries, nursing homes, schools, hotels and motels, public areas, public restrooms, athletic facilities, cafeterias, institutional kitchens, restaurants, prisons, military institutions, by police and EMS, zoos, as well as other areas/facilities concerned with the hazards of cross contamination. *Drain pans must be pre-cleaned and dry before applying this product. **Prior to disinfecting, remove heavy soil and excess water from contaminated area, clean and dry surfaces thoroughly. Then follow Disinfecting Directions. Unless otherwise stated, in 400 ppm hard water and 5% serum at a contact time of 10 minutes:

Kills over 99.9% of bacteria, specifically:

- Pseudomonas aeruginosa
- Salmonella enterica
- Staphylococcus aureus

Kills over 99.9% of viruses, specifically:

- HIV-1 (Human Immunodeficiency virus) in the presence of 400 ppm hard water and 5% serum in 1 minute

TUBERCULOCIDAL:

Mycobacterium bovis BCG in the presence of 400 ppm hard water and 5% serum in 5 minutes

DIRECTIONS FOR USE

CLEANING AND DISINFECTING DIRECTIONS: Gross filth and heavy soil must be removed before applying cleaning solution. Apply to surfaces by cloth, sponge, brush, coarse spray or by immersing equipment using normal cleaning methods. All hard surfaces must be wet thoroughly, and remain wet for at least 10 minutes and allowed to air dry. Thoroughly rinse all treated food contact surfaces with potable water prior to use. For surfaces and/or objects that may come into direct contact with children at the mouthing stage of development, a rinse with potable water is recommended. As the solution becomes dirty discard and replace with fresh solution. This is a complete product. **DO NOT MIX WITH OTHER CHEMICALS.**

DECONTAMINATION OF CRITICAL OR SEMI-CRITICAL DEVICES PRIOR TO TERMINAL STERILIZATION/HIGH LEVEL

DISINFECTION: This product is not to be used as a terminal sterilant/high level disinfectant on any surface or instrument that (1) is introduced directly into the human body, either into or in contact with the bloodstream or normally sterile areas of the body, or (2) contacts intact mucous membranes but which does not ordinarily penetrate the blood barrier or otherwise enter normally sterile areas of the body. This product may be used to preclean or decontaminate critical or semi-critical medical devices prior to sterilization or high-level disinfection.

ODOUR CONTROL: It effectively controls odours by killing odour-causing bacteria while simultaneously chemically neutralizing their odours and leaving a pleasant fragrance. Follow "CLEANING AND DISINFECTING DIRECTIONS".

Deodourizing and Cleaning of carpet/Fabrics: To Deodourize and Clean Carpets, Fabrics, and other Materials: Concrobium Broad Spectrum Disinfectant II can be used on most fabrics that can be cleaned with water-based cleaners. Not for use on wool, silk, or suede. Always test for colour fastness and material compatibility in an

inconspicuous area prior to use. Apply by sprayer, to ensure even coverage. Do not rinse, use hot water extraction wand to remove product from treated surfaces and then dry stroke carpet to remove excess moisture. Carpet can be air dried or fans may be used if carpet needs to dry more quickly.

Storage and Disposal: Do not contaminate food, feed, or water by storage or disposal.

Storage: Store at room temperature. Do not leave containers of this product open during storage. Store in original container in areas inaccessible to children and persons unfamiliar with its use.

Disposal: For information on disposal of unused, unwanted product, contact the manufacturer or the appropriate municipal/provincial/territorial agency or product stewardship organization.

Container Handling: Nonrefillable container. Do not reuse or refill this container. Clean container promptly after emptying. Triple rinse as follows: Fill container 1/4 full with water and reclose the container. Agitate vigorously and dispose of rinsate. Repeat two more times. Wrap container and put in trash or offer for recycling if available.

FIRST AID

IF ON SKIN: Take off contaminated clothing. Rinse skin immediately with plenty of water for 15-20 minutes. Call a Poison Control Center or doctor for treatment advice. Have the product container or label with you when calling a poison control center or doctor or going for treatment.

PRECAUTIONARY STATEMENTS: HAZARD TO HUMANS AND DOMESTIC ANIMALS. CAUTION: Harmful if absorbed through skin. Avoid contact with eyes, skin or clothing. Wash hands with soap and water before eating, drinking, hewing gum, using tobacco or using the toilet. Remove and wash contaminated clothing before reuse.

*** CONCROBIUM BROAD SPECTRUM DISINFECTANT II KILLS HIV-1 ON PRE-CLEANED ENVIRONMENTAL SURFACES/OBJECTS PREVIOUSLY SOILED WITH BLOOD/BODY FLUIDS** in all settings in which there is an expected likelihood of soiling of hard, non-porous surfaces/objects with blood or body fluids, and in which the surfaces/objects likely to be soiled with blood or body fluids can be associated with the potential for transmission of human immunodeficiency virus type 1 (HIV-1) (associated with AIDS virus).

SPECIAL INSTRUCTIONS FOR CLEANING AND DECONTAMINATING SURFACES/OBJECTS SOILED WITH BLOOD AND BODY FLUIDS WHICH MAY CONTAIN HIV-1 (AIDS virus). PERSONAL PROTECTION: Wear disposable rubber or latex gloves, gowns, face masks and eye coverings when performing manual cleaning of these surfaces. **CONTACT TIME:** Leave surfaces wet for 1 minute. This contact time will not control other common types of viruses and bacteria.

CLEANING PROCEDURE: The solution may be used to remove and clean blood and other body fluids from surfaces and objects before disinfection. For lightly soiled areas wipe off visual soil/clean with solution, and disinfect surface with a second application of the solution. For large spills the contaminated area must be flooded with the solution before cleaning, then decontaminated with fresh solution. Infectious materials disposal: Any materials used in the cleaning process that may contain blood or body fluids are to be disposed of immediately in accordance with local regulations for infectious materials disposal.

Questions? Comments? / Questions? Commentaires? 1.866.811.4148
Made in the USA / Fabriqué aux États-Unis
Imported by / Importé par : Siemons International Inc.
48 Galaxy Blvd, Suite 413, Toronto, Ontario