

**Product reference :**  
**3PMM50 Schoeller Masks Medical Face Mask**

**CE** These face masks bear CE marking to demonstrate compliance with MDD 93/42/EEC Annex I Health & Safety requirements -

**EN 14683 : 2019 Medical Face masks**

**Bacteria filtration efficiency** **BFE ≥ 98%**

**Microbial cleanliness ≤ 30** **Breathability < 40**

**CLASS 1**  
 MEDICAL DEVICE

**TYPE IIR**

Suitable for use in situations where moderate to light amounts of fluid or aerosols are produced. **Please note this product is not classified as PPE**

**Splash Resistance ≥ 16.0**


These medical face masks are intended to be worn by patients and other persons to reduce the risk of spreading of infections, particularly in epidemic or pandemic situations

**- NON STERILE**  
**- SINGLE USE ONLY**

**3 layer structural protection - comfortable & breathable - shaped nose clip - ear loops - fitted to face**

**Instruction for use:**

**Warnings & cautions**

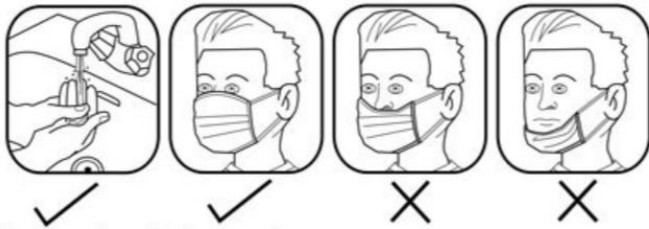

**How to put on the face mask -**

- Clean your hands with soap and water or hand sanitizer before touching the mask.
- Remove mask from package, take out the mask, hold by the two ends/ ear loops with the nose bridge at the top - white side should be facing towards the inside to touch the face, blue side should be facing to the outside.
- Cover the nose, mouth and jaw with the mask and loop the ear loops over each ear.
- Adjust the position of the mask, mould or pinch the stiff edge over the bridge of the nose for a comfortable fit.
- The mask should cover the nose, mouth and chin at all times during wearing.
- At no time should the mask hang around the neck of the wearer.

**How to remove the face mask -**

- Clean your hands with soap and water or hand sanitizer before touching the mask.
- Avoid touching the main part of the mask as this is contaminated, only touch the ear loops.
- Hold both of the ear loops and gently lift and remove the mask.
- Dispose of the mask according to the requirements of the hospital or environmental protection department.
- Clean your hands with soap and water or hand sanitizer.

- Please read the instructions on how to put on the face mask and how to remove the face mask, carefully before use.
- This product is single use, it should not be re-used.
- In case of liquid splashing, mask damage, moisture and/ or significant increase in respiratory resistance during wear please remove and replace immediately.
- Please check the packaging before use, if the packaging is damaged - do not use.
- Non sterile masks should not be used in areas with strict microbiological index control.

**Storage method:**

This product should be stored in a cool, dry, corrosive gas free, well ventilated and clean environment.

**Manufacturer -**

Empteezy Ltd  
 4 Muir Road  
 Livingston  
 Scotland  
 EH54 5DR  
 www.empteezy.co.uk  
 Contact : advice@empteezy.co.uk  
 Tel: +44 (0)1506 430309

**EU Representative -**

Empteezy Benelux  
 Populierstraat 3  
 8800 Roeselare  
 BE 0474553395  
 Belgium  
 www.empteezy.be  
 Contact: info@empteezy.be  
 Tel: +32 51 63 68 54

**Test House -**

HeX Lab  
 Porte des Batisseurs, 145  
 B-7730 Estaimpuis  
 www.hex-group.eu  
 Contact : info@hex-group.eu  
 FI ID Number: 202223  
 Report ID Number:  
 EMPT01-2-03-200911-1-PDIFF-14683