

DPL01 Plus/DPL02 Plus FFP3 NR USER INSTRUCTIONS

Product:	DPL01 Plus/DPL02 Plus FFP3 NR	Manufacturer:	
Model No.(s):	DPL01 Plus FFP3 & DPL02 Plus FFP3	Denroy Limited, 9-11 Balloo Drive, Balloo Industrial Estate, Bangor, Co. Down, BT19 7QY	
Specification:	EN 149:2001+A1:2009,	www.denroy.com	
UKCA Regulation:	Personal Protective Equipment Regulations (Regulation 2016/425) as brought into UK law and amended	EU Regulation:	PPE Regulation (EU) 2016/425

An EU & UKCA Declaration of Conformity (DOC) can be found at www.denpro.co.uk

PRODUCT INFORMATION

The DPL01 Plus/DPL02 Plus FFP3 NR is a single use filtering half mask designed to provide wearers with protection against particles as per the requirements of EN 149:2001+A1:2009. Intended use in public and private healthcare systems and work environments via distributors, retailers or supplied directly to the end users.

INSTRUCTIONS FOR USE

Use only in accordance with these instructions, as incorrect use could result in danger to life or health. Fit according to instructions BEFORE entering the contaminated area. DO NOT REMOVE whilst in the contaminated area.

Discard safely after use or replace with a new respirator if:

- Excessive clogging of the respirator causes breathing difficulty or discomfort.
- The respirator becomes damaged.
- The respirator has been worn for 8 hours.

Leave the contaminated area if:

- The respirator is damaged
- You become dizzy or have difficulty breathing
- You become irritated or other distress occurs
- You are uncertain of the nature and/ or concentration of contaminant.

WARNING

As facial hair reduces the protection offered, it is recommended that these respirators are used by clean shaven personnel only. Check fit of FFP3 mask prior to use. Check the FFP3 mask for any damage. Do not use in oxygen deficient, (less than 19%), atmospheres or in confined spaces where gases or aerosols may accumulate, and the concentration is immediately dangerous to life or health. Do not use for protection against gases or in explosive atmospheres. Single shift use only and non-reusable (NR).

FITTING INSTRUCTIONS

1. Thread top elastic strap through top clips. Repeat for bottom strap and clips.
2. Ensure hair is tied back. Your face must be free from facial hair.
3. Pull mask over your head, bringing the mask down onto your nose and face.
4. Ensure the top strap is at the crown of the head, the bottom strap is below the ears around the neck.
5. Pull the bottom straps first then the top to tighten and secure the mask. Overtightening is not necessary and may compromise comfort.
6. Ensure the seal is visible around the nose and that there is no gap over the bridge of your nose. If necessary, adjust the mask and run your fingers over the seal to ensure the seal conforms to the shape of your nose, achieving an appropriate fit.
7. Conduct a fit check. Cover the front of the respirator with both hands and exhale sharply. If the air leaks around the nose, adjust the nosepiece as per instruction 6, noting that the mask is designed so that it can sit higher or lower along the bridge of the nose dependent on the user. If air leaks around the edges, adjust the position of the respirator and/or tension of the straps.

FIT CHECK PROCEDURE

1. Cover the front of the respirator with both hands, being careful not to disturb the position of the respirator.
2. Exhale sharply. If any air leaks around the nose, readjust the nosepiece as described above. If air leaks at the respirator edges adjust the position of the respirator and/ or tension of the straps. Retest the seal. Repeat the procedure until the respirator is sealed properly. If you cannot achieve a proper fit do not enter contaminated area. See your supervisor.

Information on packaging




Latex free



See information supplied by manufacturer.



Shelf life
3 years
(see base yyyy/mm)

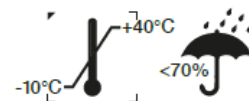

Recommended storage conditions
Temperature: -10 to +40°C
Relative Humidity: <70%

Notified Body



INSPEC International B.V.,
Beechavenue 54-62, 1119 PW,
Schiphol-Rijk,
The Netherlands
Phone: +44 (0) 16 1737 0699
Notified Body number: 2849



INSPEC International Ltd.,
56 Leslie Hough Way, Salford, Greater
Manchester, M6 6AJ,
United Kingdom
Phone: +44 (0) 16 1737 0699
Approved Body number: 0194

The Notified body conducted the PPE Regulation (EU & UKCA) type examination (Module B) and are responsible for ongoing quality surveillance Module C2.

Storage: Storage between -10 °C to +40 °C, relative humidity <70%. Must be stored in a normal climate i.e. the air must be dry, cool and free from hazardous substances. The masks must not be stored in direct sunlight. See product packaging for EXPIRY DATE and DATE OF MANUFACTURE.

Terms of Supply

The Recipient acknowledges and agrees that the Goods are supplied on an "as-is" basis, and while the Goods have been manufactured using reasonable care and skill, to the maximum extent permitted by applicable law, the Suppliers hereby exclude from these Conditions all warranties in respect of the Goods, express or implied, including but not limited to sections 13 to 15 of the Sale of Goods Act 1979. Full terms of supply can be found at www.denpro.co.uk