
# Cherry Love

Love to Love®

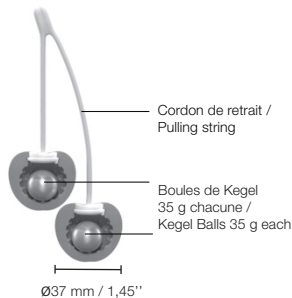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## DIRECTIONS FOR USE

- Lie down or stand up, choose a comfortable position.
- Slowly and steadily insert the longer ball into your vagina. Use a lubricant if needed.
- If you feel like it, gently start to insert the second ball. Push the balls into your vagina as far as you're comfortable with. Make sure the string connecting both balls remains outside of your vagina.
- Tighten your pelvic floor muscles to hold your Kegel balls inside of you, and go about your day. They can stay in place for up to two hours.
- To remove the Kegel balls, simply lie down or stand up and gently pull on the attached string to remove them.

## GB PRODUCT CHARACTERISTICS

- 100% soft silicone
- Weight : 35 g/ball
- Waterproof

## CAUTIONS AND WARNINGS

- We recommend the use of a water-based lubricant in order to respect its material and to extend the product's lifetime.
- Do not use your product on swollen areas or if you have small cuts. Stop using if you feel pain or discomfort.
- We strongly recommend to pregnant women to consult a doctor before use. After the delivery, it can be used only once the reeducation begun.
- Kegel balls are not intended for anal use. It could be absorbed by the sphincter.

## CLEANING AND CARE

- Your product will last longer if you look after it.
- Clean it before and after each use with an antibacterial gel, soap or wipe.
  - Do not use an alcohol-based, acetone-based or petrol-based cleaning product.
  - Store it in a place free from dust, light and heat.
  - Keep it away from toys made of other materials.

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*Thank you for choosing Cherry Love. Before discovering your product, take a few minutes to read this manual.*

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## ENVIRONMENT

The symbol of a bin crossed out indicates that this product should not be disposed of as ordinary household waste. It should be taken to your electrical and electronic waste recycling centre. By disposing of this product correctly, you will be helping to protect the environment and human health.

## ADVICE OF NON-LIABILITY

You use this product as an adult. We cannot be held responsible for the consequences of its use

## WARRANTY

In compliance with European provisions, our products are guaranteed within the limits set by law.

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