





# HANDGRIP REPLACEMENT INSTRUCTIONS


You will need the Dynamo Handgrip to complete this replacement. Can be purchased at [dynamome.com](http://dynamome.com) under Replacement Parts.

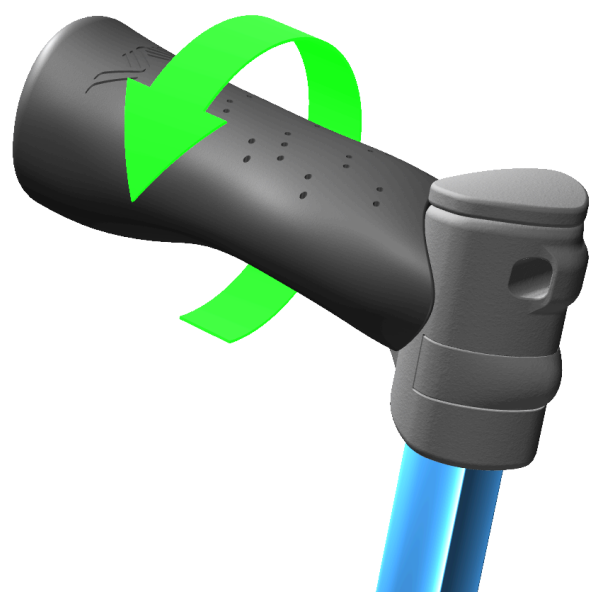


**STEP 1:**  
Twist grip and roll in one direction until grip loosens.

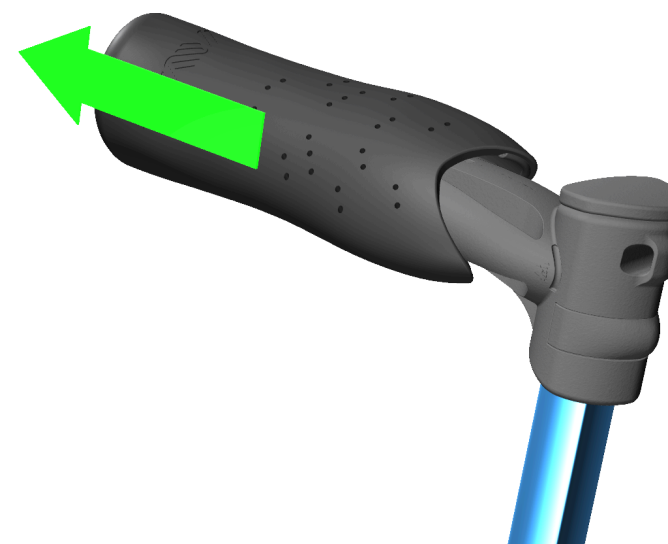

**STEP 2:**  
Once grip loosens from base, gently pull and twist off until able to be removed from cane. Set aside.

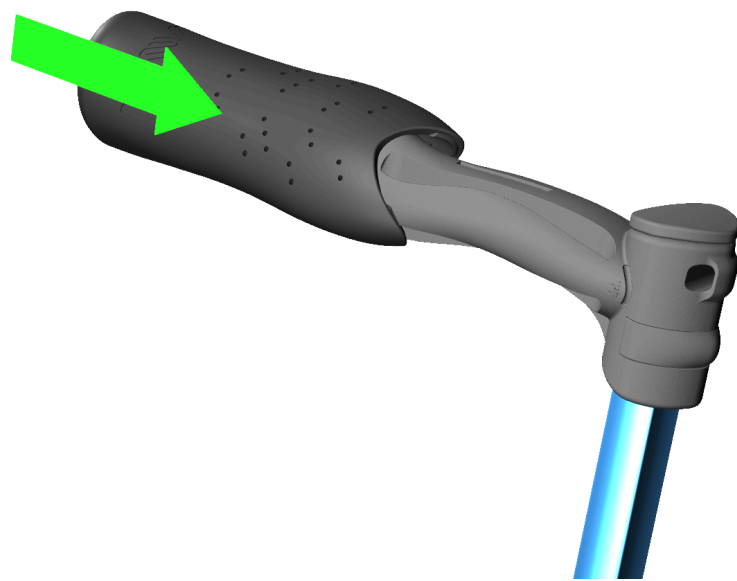

**STEP 3:**  
Take NEW cane grip, and align with cane. Making sure the "slit" is facing down.

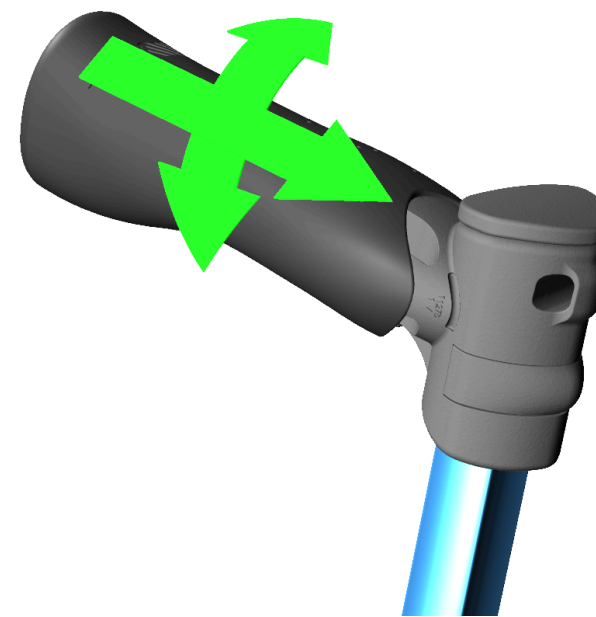

**STEP 4:**  
Push grip onto cane base until it fits snugly. You may need to "wriggle it" back and forth to get a good fit.



**STEP 5:**  
Once on, press firmly to make sure it's in place.

## QUESTIONS?

Email us at:  
[Questions@dynamome.com](mailto:Questions@dynamome.com)

Or visit us at:  
[dynamome.com](http://dynamome.com)



WARNING: To minimize the risk of injury from improper and unsafe use of this product, read all instructions and safety precautions before using.

SAFETY PRECAUTIONS: Consult a physician before using any assisted mobility device. Always inspect your mobility device prior to each use. Do not use if product or parts are missing or damaged.