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GB Thank you for having chosen Mami Cup, a reusable menstrual cup made of medical grade soft silicone. **MAMI CUP IS HYPOALLERGENIC AND LATEX-FREE.**

It is designed to be folded and inserted in the vagina as a more ecological and natural alternative to traditional sanitary napkins. Upon insertion, Mami Cup unfolds and is set in the lowest part of the vagina, lower than a tampon is, and there it is held by vaginal muscles (**fig. 1**).

It is recommended to empty Mami Cup once every 4/8 hours. It can be used during the night, while travelling or during sporting activities (swimming, running, etc.) without any risk.

1. Rim
2. Holes allowing elimination of the vacuum during removal.
3. Volume notches.
4. Ridged Exterior.
5. Stem.

BEFORE USING

- Check that the 4 holes on the upper part of the cup are open.
- Wash your hands and Mami Cup with warm water and mild soap.
- After rinsing, let Mami Cup boil for 5/7 minutes in deep water.

Your Mami Cup is now sterilized and ready to use

In order to learn to insert and remove your Mami Cup, you will need a little practice.

The first time, it is recommended to practice during the last two days of the menstrual cycle: take a little time and relax! The upper part of the vagina near the uterus is as small as a pinhead; therefore there is not any risk that Mami Cup can get lost. Furthermore, thanks to its elasticity, it cannot cause any damage.

INSERTION OF MAMI CUP MENSTRUAL CUP

Mami Cup is positioned lower than an internal tampon is (**fig. 1**).

Once Mami Cup has been disinfected, carefully wash your hands.

Find yourself a comfortable position: sitting, standing, squatting or kneeling.

Inserting Mami Cup will be easier if it is slightly wet externally.

Now you can choose which kind of folding is more suitable for you.

Folding method no. 1 (**fig. 2**)

Folding method no. 2 (**fig. 3**)

After having folded it, gently open your labia with your free hand, and gently insert Mami Cup inside your vagina, accompanying it upwards, towards the back.

Draw your fingers back and the cup will open and place automatically.

HOW TO REMOVE MAMI CUP

Squatting makes the vagina shorter, making it easier to remove Mami Cup.

Empty Mami Cup once every 4/8 hours, depending on your menstrual flow.

- Always wash your hands carefully.
- Squat or sit on the toilet. Relax in order to also relax the vaginal muscles that retain the cup.
- Gently pull the stem until you get to the base of Mami Cup. Should you not succeed in reaching the stem of the cup, help with your vaginal muscles to make its coming down easier.
- Loosen the adherence of the cup by gently pushing on its bottom (**fig. 4**) or inserting a finger along its side (**fig. 5**), so that it will easily detach from vaginal walls.

Pull the cup out delicately, to avoid spilling its contents.

Once the bottom of Mami Cup is out, tip it a little bit on one side and take it partly out, then tip it on the other side and take it completely out (**fig. 6**).

In this way, it will be easy to remove the cup upright, avoiding leakage of contained fluids.

Empty the cup in the toilet, rinse it and insert it again.

SENSE OF DISCOMFORT

THE STEM

The stem must not stick out of your vagina. Most women find the stem too long. That is the reason why it is designed to be shortened so that it reaches the orifice of the vagina without sticking out. Each woman will determine which is the length of the stem fitting her own conformation. Some women need to cut the stem completely.

Anyway, it is recommended to do this once you are certain to be able to remove Mami Cup without it. Should Mami Cup cause you any discomfort while you are sitting or walking or should it itch the orifice of your vagina, you must certainly shorten the stem.

Remove Mami Cup prior to cutting the stem.

Use clean scissors or nail clippers.

LEAKAGE

1. **Position:** most leakage problems are due to the fact that Mami Cup has not been positioned low enough in the vagina.
2. **Check** that the **stem** is correctly cut according to your conformation.
3. **Sealing effect:** check that Mami Cup is perfectly adhered to vaginal walls:
 - Check that Mami Cup is completely unfolded by running your finger along its rim.
 - Press on the bottom of the cup letting it make a complete turn.
 - Contract your vaginal muscles in order to block the cup.
4. **Mami Cup size:** if you are using a size M cup and it always moves or causes leaks, it is likely that you need to change over to size L.

CLEANING YOUR MAMI CUP

It is very important that your Mami Cup is always clean.

Wash it carefully and regularly with mild soap and warm water, making sure to remove any soap residues that could cause vaginal irritation.

It is important that holes in the upper part of Mami Cup are always open and not occluded. If needed, you can pass a needle through them.

When using a public toilet, you can wash your Mami Cup with bottled water. As soon as possible, clean it more carefully.

When you are travelling, always be careful to wash your Mami Cup with drinkable water.

With the passage of time, it is possible that the color of Mami Cup fades, but this does not compromise its efficacy at all.

FURTHER INFORMATION

The use of Mami Cup menstrual cup does not affect vaginal secretions in any way. Therefore, you can use it as a precaution in the days before the menstrual cycle starts and during it, both for light or heavy flows. To feel more comfortable, for the first times you can use Mami Cup together with a sanitary panty-liner. It is not necessary to remove Mami Cup when using the toilet, even if some women prefer to. It is possible to use Mami Cup with a coil (IUD), but it is recommended to ask your gynecologist's advice to make sure you can use both of them.

WARNINGS

Mami Cup menstrual Cup is not a contraceptive and does not protect against venereal diseases. Should you feel any pain, stinging or inflammation when urinating, immediately remove Mami Cup and ask for medical advice.

It is not necessary to remove Mami Cup when you urinate or defecate, but it is necessary to remove it before sexual intercourse.

It is recommended not to exchange your Mami Cup with other women.

Keep the cup out of reach of children and pets.

TOXIC SHOCK SYNDROME (TSS)

The use of menstrual cup dates back to the 1930s and it has never been associated with Toxic Shock Syndrome (TSS). TSS is an extremely serious and sometimes fatal disease; it is caused by a toxin of bacterial origin connected to Staphylococcus Aureus. TSS is often associated with using traditional tampons. Its symptoms are similar to those of the flu and include: sudden high fever (> 39°C), muscular pain, vomit, headache, diarrhea, vertigo, fainting, and a sort of erythematous rash similar to sunburn. Should you notice these symptoms, remove Mami Cup and ask for medical advice immediately. If you have already suffered from TSS, ask your doctor before reusing Mami Cup.

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