difrax®

Electric Nasal aspirator

Electrische Neusjesreiniger

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Elektrischer Nasensauger

NL

FR

EN

DE

ES

DK

SE

NO

FI

IS

BG

HU

PL

RO

HR

CZ

SK

EE

LV

LT

RS

KR

Fig. 1

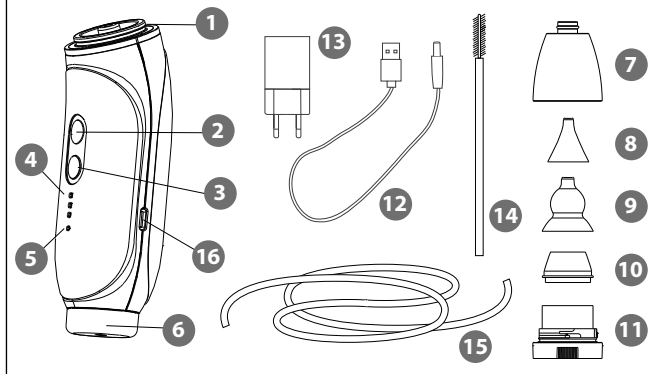
Fig. 2

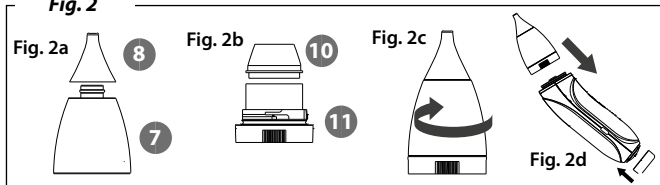
Fig. 3

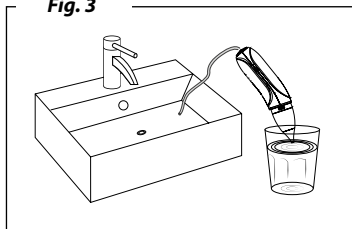
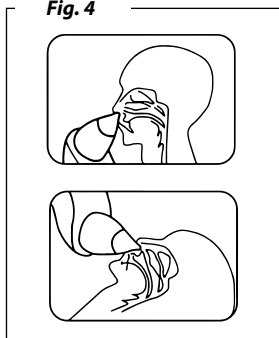
Fig. 4

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INSTRUCTIONS FOR USE

ELECTRIC NASAL ASPIRATOR MODEL NO. C168

Please read this manual carefully before using the device. Retain for future reference.

EN

NOTE

1. Make sure that the device and head are well assembled before use.
2. When any abnormality occurs, turn off the device.
3. Read the manual carefully before the first use.
- 4. Important!**
 - Keep out of the reach of children when not in use.
 - Do not store near heat sources and do not expose to extreme temperature, high humidity, or dust.
 - Avoid storing with chemicals or flammable gases.
5. If the device is damaged, immediately contact the manufacturer.
6. Do not change device settings.
7. Maintenance:
 - Check the device and all its accessories regularly.
 - Avoid long-term use. Check the device before each use.
8. Don't insert the tip into nostrils too deep.
9. Never use on ears, mouth, or eyes.
10. **For non-commercial home use only.** This nasal aspirator is for removing nasal secretions only. Do not use this device other than for its intended purposes. The manufacturer is not responsible for product misuse.
11. **For charging, use the adapter included.**

PARTS OF THE DEVICE, FIG. 1

- | | | |
|----------------------|---------------------------|-------------------------|
| 1. Connector | 7. Collection cup | 12. USB cable |
| 2. ON/OFF button | 8. Narrow tip | 13. Adaptor |
| 3. Mode button | 9. Large tip | 14. Cleaning brush |
| 4. Mode indicator | 10. Storage chamber cover | 15. Silicone drain pipe |
| 5. Battery indicator | 11. Headpiece | 16. Charging port |
| 6. Plastic cover | | |

BEFORE FIRST USE

Check the aspirator for any obvious damage and missing parts. If the device is damaged or some parts are missing, contact the device manufacturer.

- A low battery charge is indicated by the battery indicator (5). If the indicator lights red, connect the device to the power supply.
- Connect the USB cable included in the set with an original 5V/1A adapter.
- Plug the USB cable into the device port.
- Plug the adapter into a 230V AC outlet. Do not use the device during charging.
- Once the device is fully charged, the battery indicator (5) will light green.
- The device turns off automatically after 2 minutes from the start.

ASSEMBLY INSTRUCTIONS, FIG. 2

1. Join the tip (8) with the collection cup (7). See fig. 2a.
2. Put the non-return cover (10) on the headpiece (11). See fig. 2b.
3. Tighten by turning it clockwise. See fig. 2c.
4. Insert the suction head (2c) into the connector of the nasal aspirator.
5. Join the head with the base. See fig. 2d.

Choose a suitable tip.

1. The narrow tip (8) is recommended for use when the nasal secretion is loose and in a liquid form.
2. The large tip (9) is recommended for use when the nasal secretion is in a thicker, more solid form.

Insert the assembled suction head into the connector, and rotate it clockwise to secure it in the base. Make sure the head is connected correctly.

CLEANING

Manual cleaning

Disassemble all parts. Clean all head elements in warm water with mild detergent and rinse thoroughly after each use. Use a cleaning brush for better effects. Silicone elements (fig. 1, parts 8 and 9) can be scalded.

Warning! After each use, be sure to disassemble all head parts for cleaning. The device is not waterproof. Never immerse the base in liquid. Clean with a cloth.

Automatic cleaning

Assemble the nasal aspirator following the assembly instructions. Remove the plastic cover (6). Then connect the silicone drain pipe (15). Prepare clean water in a vessel. Place the end of the pipe in a sink. Press the ON/OFF button (2) to start the cleaning process (fig. 3). To ensure hygiene, clean after each use. **Do not clean the device while it is plugged in!**

HOW TO USE

Use the nasal aspirator in a comfortable position. See fig. 4.

NOTE:

Choose one of three different suction levels by pressing the mode button. Check the suction strength on the mode indicator: 1 - low, 2 - medium and 3 - high, depending on the needs and nasal secretions.

WARNING!!! Lay your baby on the back or try to hold its head during usage. Don't insert the tip into nostrils too deep. Never use on ears, mouth, or eyes.

How to remove nasal secretion more efficiently? Press the ON/OFF button to turn the device on. Insert the suction tip into a nostril. Gently move the tip in small circles to remove nasal secretion more effectively. We recommend using saline solution to soften dried nasal secretion. Spray the saline solution into the baby's nostrils. Wait for 3-4 minutes. Then suck the nasal secretion away with the aspirator. Low pressure is formed if air cannot get into the nostril, so it's easier to suck out the nasal secretion if the suction tip is close to the nasal cavity.

Always track the collection cup and headpiece during use. If there is too much nasal secretion, it may get into the suction port, which may cause malfunction of the device. If you notice that there is a lot of nasal secretion in the collection cup or headpiece, turn off the device immediately and clean it. Use a cotton swab to dry the suction port and remove nasal secretions. If the suction port has sucked a lot of nasal secretions, shake it out or blow it out with a hairdryer.

Remove the headpiece before cleaning.

Comments

The nasal aspirator generates a vacuum to remove nasal secretion. Frequent nasal congestion and runny nose caused by infections or allergies can trigger rhinitis. In case of disease or cold, please consult the doctor.

Keep out of the reach of children.

Dispose of following regulations on waste electrical and electronic equipment.

SPECIFICATION

Model: C168
Standard: 2014/35/EU (LVD)
2014/30/EU (EMC)
2011/65/EU (RoHS)

External power supply: 5V DC / 1A
Power consumed by the device: 3.7W
Built-in rechargeable battery: 1500mAh

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The controller of your personal data is Difrax with registered office in Bilthoven (Netherlands), at Rembrandtlaan 42 (hereafter Difrax).

For what purpose and on what basis do we process your data?

Your personal data will be processed to handle complaints made by you to Difrax. The legal basis for processing your personal data is Art. 6, Par. 1(f) of the GDPR, where the legitimate interest is for Difrax to be able to process your complaint. Providing of data is not compulsory, but necessary to process your complaint.

With whom will we share your personal data?

Difrax will share your personal data with recipients charged with processing personal data on behalf of and for Difrax, in particular companies entrusted with the processing of personal data as part of IT services and support. In addition, Difrax will share your personal data with other recipients if an obligation to do so arises from the law. Your data will not be relayed to third countries and international organisations.

How long will we process your data?

Your personal data will be processed for the duration of the handling of your complaint, and archived as well, but will be retained no longer than 1 year from the time of receipt of your complaint.

What are your rights?

You are entitled to the following:

- access your personal data and receipt of a copy of the processed personal data;
- rectification of your data if they are not correct;
- data deletion requests (right to be deleted) in case of circumstances provided for in Art. 17 of the GDPR;
- restriction of data processing in the cases mentioned in Art. 18 of the GDPR;
- to object to the processing of data in the cases mentioned in Art. 21 of the GDPR;

If you believe that your personal data is being processed unlawfully, you can lodge a complaint to the supervisory authority.

Contact

If you need additional information regarding the protection of personal data or if you wish to exercise your rights, please contact us at:

E-mail address: support@difrax.com

Difrax with registered office in Bilthoven (Netherlands),
Office and warehouse: Rembrandtlaan 42, 3723 BK Bilthoven (Netherlands)

STATEMENT BY THE MANUFACTURER

The manufacturer declares with full responsibility that the product complies with the essential requirements for the guidelines of the new approach leading to CE marking. The declaration of conformity issued is a confirmation.

SERVICE DATA

Customer service: +31 (0)30 229 44 22

If you have problems connecting, make sure the contact details are current. You can find these on the website difrax.com

WARRANTY

1. Difrax provides a 24-month warranty from the date of purchase, provided that the product has been used as intended, is not mechanically damaged and that the seal has not been broken. Complaints should be made at the place of purchase.
2. It is recommended that the buyer delivers the item in its original factory packaging, additionally protected against damage. Damage caused by improper protection of the article is not covered for repair under the warranty.
3. The warranty period is extended by the amount of time from reporting a defect or damage until the item is repaired.
4. The warranty repair does not cover the work listed in the manual, which the buyer is obliged to carry out themselves and at their own expense.
5. Any cleaning of items will be carried out at the buyer's expense according to the price list provided at the authorised service point and is not treated as warranty repair.
6. Warranty does not apply in the following cases:
 - a) mechanical damage to the article caused during use or during delivery of the article for repair;
 - b) damages and defects arising from:
 - the use of the article for purposes other than those for which it is intended;
 - use, maintenance, storage or installation that is incorrect or not in accordance with the instructions; use of unsuitable consumables;
 - repairs carried out by unauthorised persons; if it is established that such a repair has occurred or the article has been opened arbitrarily, the warranty will be voided;
 - modifications, structural changes or use for non-original repairs;
 - spare parts;
 - c) glass parts, light bulbs, lights;
 - d) consumables
7. The warranty card will be invalid without the name and model of the item and a purchase date confirmed by the seller's stamp and signature.
8. The above warranty for consumer goods sold excludes, does not limit, suspend, or exclude the rights of the buyer as a result of non-compliance with the contract for the items.

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Difrax B.V.