Heatbag

Use instructions

How to use it

Hot

1. Place the bag of clay in the microwave and heat it to maximum power (check your microwave specifications): 800W for 60 seconds, or 1000W for 55 seconds. Always heat it while it is at room temperature, never frozen. Make sure that any additional microwave features are turned off.

Note: Never insert the cover into the microwave, as it contains a metal zipper.

Note: If you do not have a microwave, you can heat it by boiling it. Immerse the bag in boiling water for a maximum of 5 minutes. Dry it and put it in the cover before applying it to the skin.

2. Take it out of the microwave and check if the temperature is adequate (note that the temperature will continue to rise slightly).

3. If it has not reached the desired temperature, put it back in the microwave and heat it up for 10 second intervals. If the bag of clay starts to expand, turn off the microwave. If it overheats, it may break or leak. In that case, discard the bag immediately.

Note: We recommend that you write down the heating time used for future use.

4. Once the bag of clay is at the desired temperature, put it inside the cover, close the zipper and place it on the area to be treated. Thanks to the composition of the cover, the duration of the heat will be longer. You may have to reheat the bag after one hour of use.

Note: Always insert it into the cover, never apply it directly to the skin, especially for people with sensitive skin.

5. After use, check that it has cooled down before storing.

Cold

1. Put the bag of clay in the freezer for at least 1 hour. Check if the temperature is adequate.

Note: If the bag of clay was frozen for more than 4 hours under temperatures of -18°C , we recommend putting it aside until it becomes soft before application to extend the lifetime of the product, as the cold endurance of material will reach limit and result in product cracking.

2. Take it out of the freezer and put it in the cover, close the zipper and place it on the area to be treated. Thanks to the composition of the cover, the duration of the cold will be longer. If you need to apply more cold, check the temperature. If necessary, put the bag back in the freezer until it has the desired temperature.

3. After use, put the bag back in the freezer for the next cold use, or leave it at room temperature inside its cover.

Precautions

IMPROPER USE MAY CAUSE INJURY. Carefully read instructions and warnings before use. We declare the clay bag is FDA/CE approved and conforms to the following product legislation in accordance with the following directive(s): 93/42/EEC

- Adult supervision is required. Do not allow children to heat the clay bag while unsupervised.
- Always use it inside the cover. Never apply it directly to the skin.
- Do not use over sensitive skin areas, open wounded, areas of severe swelling.
- Excessive heating could cause skin burns, clay bag deterioration, and possible leakage.
- Please, do not puncture or tear the clay bag. Therefore, we recommend keeping away from sharp objects. Before using it anytime, carefully check the clay bag for any sign of possible damage. If any is found, cool it down and discard it.
- Store the clay bag away from extreme heat. Keep out of direct sunlight.
- Please, consult with your medical support if you have a serious injury to understand how to benefit from this product for your specific case.
- If content is swallowed, call your medical center immediately.
- Do not machine wash or dry the clay bag. Hand wash only. Surface cleaning with a damp cloth or sponge.

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