

vive®

TOILET SEAT CUSHION

Owner's Manual



vivehealth.com

OVERVIEW

We are constantly answering questions and recording helpful videos to make using your Vive Toilet Seat Cushion as easy as possible. Check out the included link and QR code to help you through the process.



To see all of the FAQs in one place, visit vhealth.link/w4a

HOW TO ATTACH TO TOILET SEAT

1. Make sure the toilet seat and suction cups on the Toilet Seat Cushion are clean.



2. Line up the cushion with the toilet seat and ensure there is enough clearance in the toilet seat opening.



3. Place the cushion on the toilet seat

4. Press down lightly to make sure the suction cups attach securely.



HOW TO REMOVE FROM TOILET SEAT

Option 1: Lift the Toilet Seat Cushion gently and pull upward on the suction cup pull tabs.



Option 2: Lift the Toilet Seat Cushion gently and slide the putty knife under the suction cups to remove.



HOW TO REMOVE FOAM FROM COVER

- Locate the zipper on the bottom of the Toilet Seat Cushion.
- Unzip completely and remove the foam from the cover.



CARE INSTRUCTION

- Wipe clean with a disinfecting wipe or soap and water.
- Dry completely.

HAVE MORE QUESTIONS?

Check out our list of Frequently Asked Questions at [vhealth.link/w4a](https://vivehealth.com/w4a) for helpful answers.



And if that doesn't answer your question, our customer service team would love to help! Feel free to connect with them by phone, e-mail, or chat on our website.



service@vivehealth.com



1-800-487-3808



vivehealth.com

Distributed by

vive
health

8955 Fontana Del Sol Way
Naples, FL 34109
Made in China