

Using TePe® Denture Care

TePe® Denture Care is designed with extra-long, strong bristles to ensure good accessibility and efficient cleaning. It is important to use and care for your TePe® Denture Care properly—please read below for instructions and helpful tips.



5 Simple Steps to Using Your TePe® Denture Care:

- 1. Remove your prothesis or denture from your mouth.
- 2. Rinse any large debris off with water.
- 3. Securely hold your prosthesis in one hand.
- 4. Systematically brush all surfaces.
- 5. Rinse your prosthesis or denture one more time after brushing.

Scan the QR code to watch a video about TePe® Denture Care



Tips for using your TePe® Denture Care

- It is best to complete this in a systematic approach to ensure you don't miss any areas.
- Angle the neck in hot water to adjust to your liking.
- Replace brush once bristles start to fray or become worn.



