

CWS 5000

Patient Warming System

Advanced, Technically Superior

- Two speed fan motor and two levels of flow rates. CWS 5000 can inflate a very large blanket as well as a small blanket.
- Uses a DC motor, quieter and state of the art blower.
- CWS 5000 is controlled and operated through a unique robust software developed by Care Essentials.
- The Cocoon™ CWS 5000 is light and compact, yet very sturdy.
- Designed to be user friendly.



TEMPERATURE MANAGEMENT SYSTEM

Convective Warming Machine

Microprocessor controlled, filter status indicator, digital temperature sensor system, washable protective hose cover, built-in safety monitoring system, retractable bed hook brackets, external testing port.

Electrical	
Supply Voltage	220-240 Vac
Supply Frequency	50/60 Hz
Performance	
Temperature settings measure the average air temperature at the end of the delivery hose	Ambient 34°C, 40°C, 43°C or 46°C
Temperature accuracy of delivery air, except for the ambient temperature setting.	+/- 2°C with delivery hose cover
Higher Speed Fan	If High Flow rate required, (Press _ HIGH SPEED) button
Physical	
Dimensions	29 cm x 22 cm x 40 cm
Weight	6.0 kg
Filter Type/Life	Bacteriological HEPA filter / 2000 hours 0.2 micron



Disposable Animal Warming Blankets




DESCRIPTION

Cocoon blankets are made from soft, comfortable and light-weight non-woven fabric. All materials used in our blankets are latex free. Our unique microscopic perforations provide uniform air distribution without the risk of residue. The universal inlet port ties ensure the hose remains firmly in the blanket. The ties and strong fabric allow the use of same blanket on the patient from pre-warming through surgery to recovery.

USAGE GUIDE

- An Incontinence pad may be placed under the animal in the central area.
- Place the blanket either under or over depending on blanket model
- White side of blanket faces the animal
- Insert the heater hose into any blanket inlet port.
- Don't occlude the inflation tube at the blanket inlet or when closing the cage door (depending on blanket model)
- Monitor animal temperature frequently. Hospital's protocol to be followed.
- Replace the blanket if wet or soiled.
- Position the animal either dorsal or lateral.

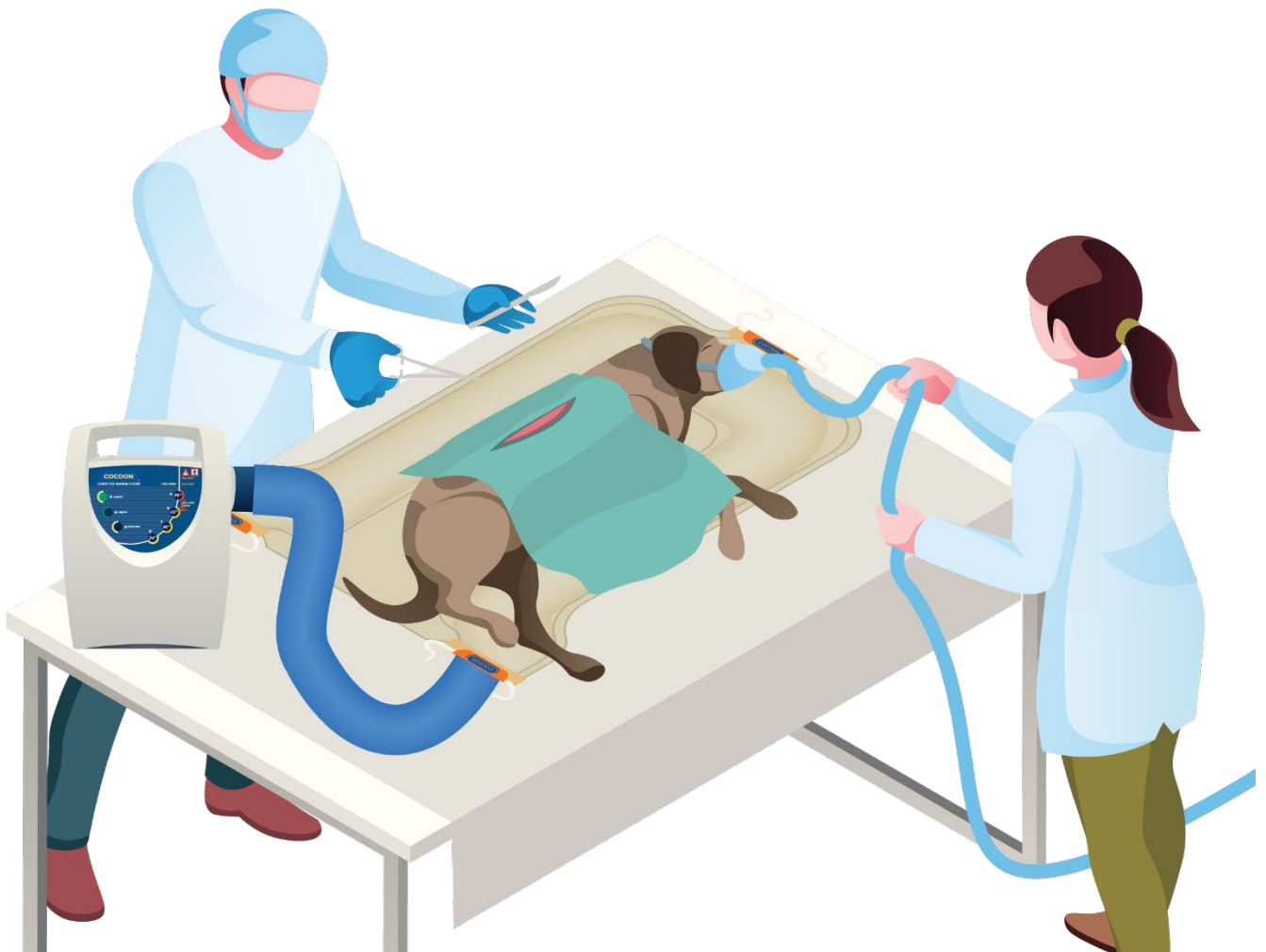

Care Essentials Pty Ltd
103 Mornington Street, North Geelong, VIC 3215
Phone: +61 3 5277 1455 Fax: +61 3 5277 1411
Email: queries@careessentials.com.au





Disposable Animal Warming Blankets







BLANKET MODELS

Model	VET001	VET002	VET003	VET004
Name	Cocoon Vet Under-Blanket Medium to Large	Cocoon Vet Under-Blanket Small	Cocoon Vet Over-Blanket Medium to Large	Cocoon Vet Over-Blanket Small
Size	126cm L x 79cm W	86cm L x 79cm W	123cm L x 83cm W	66cm L x 86cm W
Hose Ports	3	3	2	2
Usage	Anaesthesia, Surgery	Anaesthesia, Surgery	Caged animal, Recovery, Critical care	Caged animal, Anaesthesia, Critical care, recovery
Description	Under body blanket for medium to large patients	Same design as VET001 but with smaller dimensions	Over body blanket for medium to large patients	Same as VET003 but with smaller dimensions

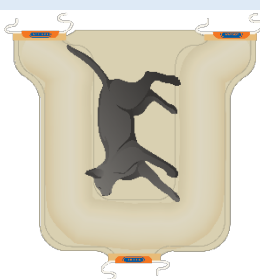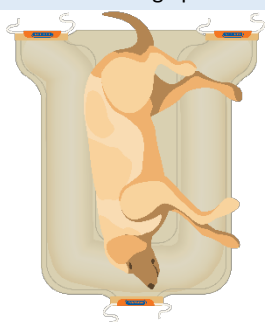

Model	VET005	VET006	VET007	VET008
Name	Cocoon Vet Over-Blanket Large	Cocoon Vet Over-Blanket Extra Large	Cocoon Vet Tube Blanket Medium to Large	Cocoon Vet Tube Blanket Small to Medium
Size	100cm L x 120cm W	100cm L x 195cm W	180cm L x 14cm W	99cm L x 14cm W
Hose Ports	1	1	1	1
Usage	Anaesthesia, Recovery, Critical care	Anaesthesia, Recovery, Critical care	Caged animal, Anaesthesia, Recovery, Critical care	Caged animal, Anaesthesia, Recovery, Critical care
Description	Over body blanket for large patients	Over body blanket for extra-large patients	Tube style blanket for medium to large patients	Same as VET007 but with smaller dimensions

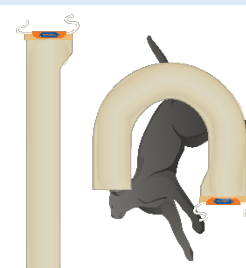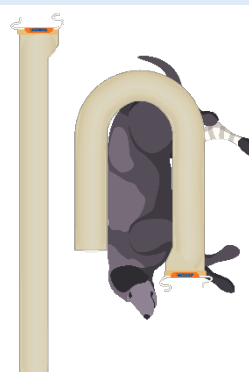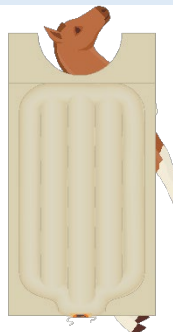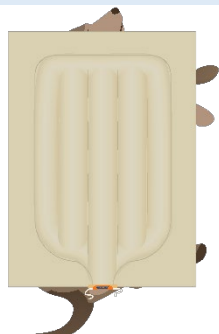

Care Essentials Pty Ltd
103 Mornington Street, North Geelong, VIC 3215
Phone: +61 3 5277 1455 Fax: +61 3 5277 1411
Email: queries@careessentials.com.au




Disposable Animal Warming Blankets



UNIQUE COCOON INNOVATION

Model	VET009
Name	Cocoon Vet Over-Blanket Large
Size	100cm L x 120cm W
Hose Ports	1
Usage	Anaesthesia, Recovery, Critical care
Description	Ideal for pre-surgery therapy or post-surgery recovery with extra-long inlet port for easy access to patient inside kennel.

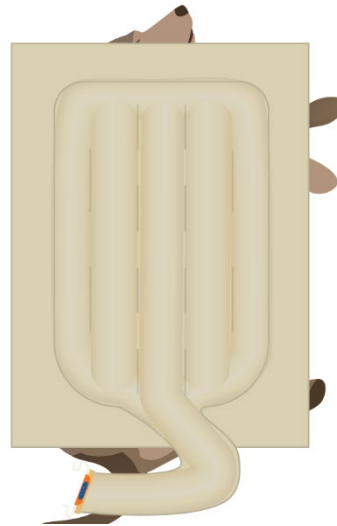

BENEFITS

- Warm patient directly through kennel door
- Versatile design to allow inlet port to fit through kennel of any size
- Access the kennel without interrupting warming therapy



Care Essentials Pty Ltd
103 Mornington Street, North Geelong, VIC 3215
Phone: +61 3 5277 1455 Fax: +61 3 5277 1411
Email: queries@careessentials.com.au

