



Gelmax Mat Ultra: improves working environment during examinations & surgeries.

Many surgeries such as arthroscopy-based surgeries generate large quantities of flushing liquid (ie irrigation fluid), urine, blood, etc. There is a big risk of slipping or falling on a wet floor – but we have the solution!

Gelmax Mat Ultra is a superabsorbent single-use mat with a high absorption capacity. Despite being only a few millimetres thick, the mat absorbs and can contain large quantities of fluids and also bacteria.

Gelmax Mat Ultra is capable of absorbing up to 9 litres of water or 3.5 litres of saline solution. It can be placed under the operating table, in front of the feet of the surgical team. Another application is on the examination table of wet examinations – under the patient with a sterile sheet over – to collect flushing liquid, preventing the soiling of the patient and equipment. The cloth does not leak or drip as the absorbed fluids remain inside and are turned into a gel. Since both liquid and bacteria are retained in the gel this product contributes to the infection control of surgeries and examinations.

The result is a drier, more hygienic, safer and pleasant working environment – also meaning less time and money spent on cleaning. Depending on the fluid absorbed – the mat can be disposed of in the usual manner. Turn-around time is reduced considerably, allowing for more operations to be carried out.

Benefits of preventing slips and fall injuries:

- Reduce Workers' Compensation claims
- Reduce lost workdays
- Improve employee safety and satisfaction

Reference	9520241
Size	72 x 37 cm
Weight	apx. 120 g
Application	Absorbs waterbased liquids such as saline, urine and blood
Use	Disposable product for single use, do not cut outer cover, do not stand on the mat - risk of slipperiness
Sterile	No
Absorbency	9 litres of tap water or 3.5 litres of 0.9% NaCl
Composition	Core: cellulose and sodium polyacrylate. Outer cover: polypropylene non woven
Disposal	Dispose of as clinical waste. Releases carbon dioxide & water when incinerated
Packaging	25 pcs/plastic bag (to hang on wall) x 4 = 100 pcs/box



Tour Maine-Montparnasse
33, avenue du Maine
75755 Paris Cedex 15
E-mail: cleanis@cleanis.com

