



Needlestick prevention that **meets your needs**

- ▶ Virtually eliminates exposure
- ▶ Single-handed, passive activation
- ▶ Hands remain behind the sharp
- ▶ Integrated safety mechanism
- ▶ Prevents recapping and needle removal
- ▶ Remains safe through disposal
- ▶ Easy to use, requires minimal training
- ▶ Safe and effective for patients

Catalog No.	Description	
VanishPoint® U-100 insulin syringes 100 per box/800 per case		
15391	0.5mL VanishPoint insulin syringe (half-unit)	30G x 3/16" NEW
15371	0.5mL VanishPoint insulin syringe (half-unit)	30G x 5/16" NEW
15291	0.5mL VanishPoint insulin syringe	30G x 3/16"
15271	0.5mL VanishPoint insulin syringe	30G x 5/16"
15221	0.5mL VanishPoint insulin syringe	30G x 1/2"
10291	1mL VanishPoint insulin syringe	30G x 3/16"
10271	1mL VanishPoint insulin syringe	30G x 5/16"
10281	1mL VanishPoint insulin syringe	29G x 5/16"
10211	1mL VanishPoint insulin syringe	29G x 1/2"

1mL VanishPoint® syringes 100 per box/800 per case		
10131	1mL VanishPoint tuberculin syringe	27G x 1/2"
10151	1mL VanishPoint tuberculin syringe	25G x 5/8"
10161	1mL VanishPoint syringe	25G x 1"

VanishPoint® Allergy Syringe Tray 25 per tray/1000 per case		
10134	1mL VanishPoint allergy syringes	27G x 1/2"

3mL VanishPoint® syringes 100 per box/600 per case		
13021	3mL VanishPoint syringe	27G x 1 1/2"
10301	3mL VanishPoint syringe	25G x 5/8"
10391	3mL VanishPoint syringe	25G x 1"
13011	3mL VanishPoint syringe	25G x 1 1/2"
10311	3mL VanishPoint syringe	23G x 1"
10321	3mL VanishPoint syringe	23G x 1 1/2"
10331	3mL VanishPoint syringe	22G x 1"
10341	3mL VanishPoint syringe	22G x 1 1/2"
10351	3mL VanishPoint syringe	21G x 1"
10361	3mL VanishPoint syringe	21G x 1 1/2"
10371	3mL VanishPoint syringe	20G x 1"
10381	3mL VanishPoint syringe	20G x 1 1/2"

5mL VanishPoint® syringes 100 per box/600 per case		
10mL VanishPoint® syringes 100 per box/600 per case		

5mL and 10mL VanishPoint® syringes are available with a variety of needle lengths and gauges. For a complete list, please contact RTI or visit our website at www.retractable.com.



CE 2797
© 2008, 2019 Retractable Technologies, Inc.
All rights reserved. M005 REV.15 EN, JUL 2019



Tel: (852) 2892 0033 Fax: (852) 2892 1433
Email: admin@synemed.info



The new standard for safety®



Effectively reduces the risk of needlestick injuries

A study comparing the effectiveness of passive SED (safety-engineered devices) with that of active SED, concluded that passive SED (such as the VanishPoint® syringe) significantly reduced the needlestick injury rate.¹

Attached needle design:

- Maintains a closed system
- Prevents leakage and contamination
- Reduces deadspace
- Prevents unnecessary needle changes

"Changing needles between drawing vaccine from a vial and injecting it into a recipient is unnecessary."²

¹A. J. Goris, et al., "Reducing Needlestick Injuries from Active Safety Devices: A Passive Safety-Engineered Device Trial," *AOHP Journal*, vol. 34, no. 2 (Spring 2014), pp. 14-18.
²CDC (2002), General recommendations on immunization: recommendations of the Advisory Committee on Immunization Practices (ACIP) and the American Academy of Family Physicians (AAFP), *MMWR* 51(RR02):1-36.

Safe for you and your patients



1

Use standard procedure for drawing up medication.



2

Prepare and give injection using aseptic technique according to institutional policy.




3

While the needle is still in the patient, fully depress the plunger to activate retraction.



4

Dispose of VanishPoint® syringe in an appropriate sharps container.

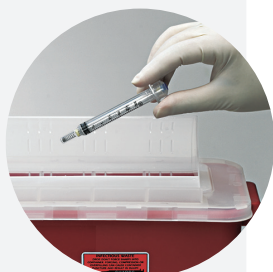


THE VANISHPOINT® ADVANTAGE

Effective safety with automated retraction technology

Clear, unobstructed calibrations allow for accurate dosing

Triple beveled, lubricated needle provides patient comfort

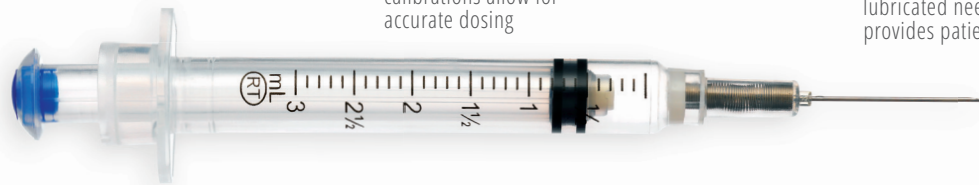

EASY, ONE-HANDED ACTIVATION

NO CONTAMINATED SHARP EXPOSED



Needle retracts directly from patient into syringe

Pre-removal activation prevents exposure to contaminated sharp



VanishPoint® insulin syringes **NOW AVAILABLE WITH**

- 5mm and 8mm needles
- Half-unit increments for accurate small dose delivery



Activated VanishPoint® syringes require less disposal space than other safety syringes and prevent disposal-related injuries.


Other available safety products



EasyPoint® Needles



VanishPoint® Blood Collection Set

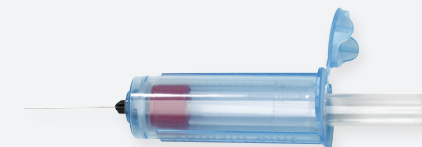

VanishPoint® Blood Collection Tube Holder



VanishPoint® IV Catheter



Patient Safe® Syringe