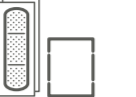
EN Food Allergy Test

The cerascreen® Food Allergy Test is a send-in blood sampling kit to determine the presence of IgE antibodies in response to 38 specific foods. Discover the concentration of IgE antibodies in your blood as an initial indicator for possible food allergies.

Your test kit contains

A		B	
C		D	
E		F	
G			

	
SKU: 99091175 CRS3-LMA_INT_000	Valid from February 2023
www.cerascreen.info	

CE
Supplier | **Lieferant** | **Levendarer** | **Provedeur** | **Fournisseur** | **Fornitore** | **Leverancier** | **Levendarer** | **Leverancier**

B: PAUL HARTMANN AG, Paul-Hartmann-Straße 12, 89522 Heidenheim, Germany
C: Ahlstrom-Munksjö Germany GmbH, Niederschlag 1, 09471 Bärenstein, Germany
D: Becton Dickinson GmbH, Tullastrasse 8-12, Postfach 101629, 6900 Heidelberg, Germany

		
--	--	---

GENERAL INFORMATION

- Do not take a blood sample immediately after strenuous physical activity.
- If you have any difficulties performing the blood sample or if the kit is incomplete, contact our customer service department, who will treat this matter confidentially (help.cerascreen.com).
- The product is for single use only.
- Do not touch the dried blood spot card within the markings and keep it dust free.
- Unfortunately, it is not possible to analyse blood samples that contain less than the required amount of blood.

CONTAGIOUS ILLNESSES

Do not use this kit if you suffer from a blood-borne contagious illness such as hepatitis or HIV.

RISKS AND SIDE EFFECTS

- After taking a sample of your blood, pressure sores may occur around the wound for a few days and/or a bruise may form at the injection site. Additionally, the puncture site may be visible for a few days.
- Note:** Any serious incident related to the product should be reported to the manufacturer and to the competent authority of the Member State where the user is established.

WARNINGS

- Store this kit out of reach of children.
- The kit may only be used by adults aged 18 and older.
- If you are pregnant or are breastfeeding, use the kit only after consultation with your medical professional or pharmacist.
- If you suffer from a congenital or acquired blood-clotting disorder or are taking anticoagulant medication, use the kit only after consultation with your medical professional.
- Use the lancets only as described in the instructions for use.
- If you have problems with the sight of blood or blood sampling, take the sample only in the presence of another person or contact your medical professional or pharmacist.

If you have any questions, please visit our help website: help.cerascreen.com

Directions for Use

- Check that all conditions are fulfilled for sample taking. Please read the section „Information and instructions on the test“ carefully.
- If you are a new customer, create your user account on my.cerascreen.com or in the **my cerascreen®** App. In our "Frequently Asked Questions" in help.cerascreen.com you will find a guide for that. To create your result report, log in via the **my cerascreen®** App (Android, iOS) or on my.cerascreen.com. Insert your ID-number according to your test-ID card under 'Activate Test' and follow the instructions.
- Place all contents of the test kit on a clean surface. Before you begin, choose a spot for the prick. TIP: the middle or ring finger of the left hand usually works best.
- Wash your hands with soap and warm water. Hold your preferred hand under warm water for at least **15 seconds** to stimulate blood circulation. Dry your hand carefully with a clean towel.
- Swing your preferred arm around in a circle 5 to 10 times,

fully extended if possible, in large movements to stimulate blood circulation.

- Remove the alcohol wipe from its film and use it to disinfect the tip of the preferred finger. Wait until the finger is dry again (15 sec).
- Take the lancet. (1) Rotate the protective cover once around its axis. (2) Carefully remove the protective cover. **CAUTION:** Do not use a lancet with a loose or missing protective cover.
- Place your hand on a firm surface. Place the lancet diagonally on the tip of your finger as shown in the illustration. Using the side of your fingertip will reduce discomfort. Press the lancet firmly against the finger until it releases. **CAUTION:**The lancet can only be released once!
- Encourage blood flow by standing up and holding your hand below the heart. Gently massage your finger from the palm of your hand to your fingertip until a drop of blood forms.
- Before you start collecting the blood, take the swab (**NOT the alcohol wipe**) and wipe off the first drop of blood. Wait until a big drop has formed again.

- Press the thumb of your other hand below the sampling spot and pull back on the skin slightly.
- Encourage blood flow by standing up and holding your hand below the heart. Gently massage your finger from the palm of your hand to your fingertip until a drop of blood forms.
- Before you start collecting the blood, take the swab (**NOT the alcohol wipe**) and wipe off the first drop of blood. Wait until a big drop has formed again.
- Let a large blood drop fall onto the marked circle on the dried blood spot card. In case the drop doesn't fall on its own from your finger, hold the dried blood spot card carefully against the drop without touching the paper to your finger.
- For the best evaluation of the sample, the largest possible drop of blood is required. The blood drop should cover the circle inside the marking completely.
- Repeat this process until you have completed 4 circles. If you fill up more than three circles, your sample will be evaluated, as well.
- After taking the blood, use the swab and plaster to clean and cover the wound.
- Remove the square code sticker from the test-ID card and stick it on the dried blood spot card in the area intended for the test-ID sticker.
- Allow the dried blood spot cards to **dry for 2 to 3 hours** at room temperature. Make sure that you place it flat on a dry surface, protected from direct sunlight.
- Wait until the blood has dried completely. Place the dried blood card back into the plastic bag so that the Test ID is visible (see picture).
- Open the included return mailer box and set the clear plastic sticker aside.
- Pull on one of the three safety latches in the center to lift the inlay tray. Place the dried blood card in the bottom of the box and replace the inlay tray over it.
- Close the box and seal it using the clear plastic sticker included. The spot for the sticker is denoted with: "affix here". For return shipping with UPS, please go to: <https://www.cerascreen.com/return>
- Please check if the test has been activated and dispose of the used parts of the test kit in the transparent plastic bag in the household waste.

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