

## CARRFLEX® Extreme - Approved Cleaners August 2019

**PLEASE REFER TO IMPORTANT MESSAGES ON FINAL PAGE BEFORE APPLICATION OF ANY CLEANER/DISINFECTANT.**

MANUFACTURER	PRODUCT	MAIN ACTIVE COMPONENT	CONCENTRATION	DATE TESTED
Ecolab®	A-456-II		1.6% A-456-II (1:64 Water)	02-Dec-14
Ecolab®	Actichlor Plus	Sodium Dichloroisochyanurate 30-60%	2000PPM & 10,000PPM	20-Nov-12
Quantum Pacific Limited	Actimax		According to Manufacturer Spec	11-Apr-06
ABC Compounding Co., Inc.	Aero Stain Away		According to Manufacturer Spec	11-Aug-16
SCIENCECompany	Ammonium Chloride	Cocoalkaydimethylbenzyl ammonium chloride	According to Manufacturer Spec	17-May-19
Milton®	Anti-Bacterial Tablets	Sodium Dichloroisochyanurate 30-60%	1 Tablet:2L Water	19-Jul-12
Ecolab®	Asepti Active Liquid		According to Manufacturer Spec	20-Nov-12
Ecolab®	Asepti Active Wipes	Dual-Quaternary/Alcohol	Ready to Use	20-Nov-12
Synergy Health	Azomax™ Multi Surface Wipes	Benzylkoniumchloride 21.6%	Ready to Use	15-Mar-13
Synergy Health	Azowipe™ Hard Surface Wipes	Benzylkoniumchloride 2%	Ready to Use	15-Mar-13
PreVasive®	BAC Botanical Antimicrobial Cleaner	Thyme Oil	According to Manufacturer Spec	22-May-19
MEDCHEM®	BAK 1:750	Benzalkonium Chloride	According to Manufacturer Spec	12-May-19
Clorox®	Bleach Wipes (Sodium Hypochlorite 0.1 - 1%)	Sodium Hypochlorite	According to Manufacturer Spec	11-Aug-16
Spartan Chemical Company Inc.	BNC-15™		According to Manufacturer Spec	11-Aug-16
Ameriderm Laboratories Limited	Care Source Disinfectant		According to Manufacturer Spec	10-Nov-11

## CARRFLEX® Extreme - Approved Cleaners August 2019

**PLEASE REFER TO IMPORTANT MESSAGES ON FINAL PAGE BEFORE APPLICATION OF ANY CLEANER/DISINFECTANT.**

MANUFACTURER	PRODUCT	MAIN ACTIVE COMPONENT	CONCENTRATION	DATE TESTED
Metrex® Research	CaviCide®	Isopropanol	According to Manufacturer Spec	06-Oct-16
Metrex® Research	CaviCide1™	Isopropanol	According to Manufacturer Spec	05-Jan-17
Metrex® Research	CaviWipes1™	Isopropanol	Ready to Use	05-Jan-17
Metrex® Research	CaviWipes™	Isopropanol	Ready to Use	06-Oct-16
Helix Solutions	Chlor-Clean	Sodium Dichloroisocyanurate 20-30%	1000PPM (1 Tablet per 1 Litre Water)	10-Oct-11
Agar Cleaning Systems Pty. Ltd.	Chloradet	Sodium Hypochlorite <10%	0.6% Chloradet (1:160 Water)	06-Jun-16
Ecolab®	Chlorwhite	Sodium Hypochlorite <10%, Sodium Hydroxide <10%	1% Chlorwhite (1:100 Water)	22-Oct-13
Apollo Corporation	CID A-L™ II Quaternary Disinfectant	Quaternary Ammonium Compounds	According to Manufacturer Spec	26-Sep-12
Wexford Labs, Inc.	CleanCide® Wipes	Citric Acid	Ready to Use	15-May-19
Clorox Disinfecting Wipes	Clorox Disinfecting Wipes		Ready to Use	14-Feb-11
Alco Mandala Products	DC-7		According to Manufacturer Spec	16-Feb-12
Delaval	Delaval Acid Wash		5% Delaval (1:20 Water)	06-Jun-14
Clinell®	Detergent Wipes	Propane 1,2-diol	According to Manufacturer Spec	27-May-09
ArjoHuntleigh	Disinfectant Cleanser IV		According to Manufacturer Spec	23-Nov-09
Clorox®	Dispatch® Disinfectant		According to Manufacturer Spec	16-Feb-12

## CARRFLEX® Extreme - Approved Cleaners August 2019

**PLEASE REFER TO IMPORTANT MESSAGES ON FINAL PAGE BEFORE APPLICATION OF ANY CLEANER/DISINFECTANT.**

MANUFACTURER	PRODUCT	MAIN ACTIVE COMPONENT	CONCENTRATION	DATE TESTED
Tristel Solutions LTD	Distel Surface Disinfectant (Formerly Trigene®)	Polymeric Biguanide Hydrochloride <10%	According to Manufacturer Spec	24-Oct-06
Sealed Air Diversey Care	Divercleanse	Sodium Hypochlorite <5%, Sodium Peroxide <2%	3.1% Divercleanse (3.1:100 Water)	15-Dec-09
Buckeye® International, Inc.	Eco E23 S23		256 to 1 dilution with water	05-Jan-17
Velocity Chemicals	Eco-Pine		5% Ecopine (1:20 Water)	11-Apr-06
Clorox HEALTHCARE®	Fuzion™		According to Manufacturer Spec	05-Jan-17
Spartan Chemical Company Inc.	GS Neutral Disinfectant #3502		According to Manufacturer Spec	02-Dec-14
Clinell®	Hand & Surface Sanitizer	Benzalkonium Chloride	According to Manufacturer Spec	27-May-09
Spartan Chemical Company Inc.	hdq®C2	Ammonium Chloride	According to Manufacturer Spec	30-Jun-17
Brighton Professional	Hepastat 256		According to Manufacturer Spec	27-Aug-14
Wipesplus	Hospital/Surgical Wipes	Ammonium Chloride	Ready to Use	15-May-19
(Various)	Hydrogen Peroxide 5%		Ready to Use	25-May-12
Clorox HEALTHCARE®	Hydrogen Peroxide Cleaner Disinfectant Wipes	Hydrogen Peroxide	Ready to Use	12-Apr-19
Apollo Corporation	Hygena™	Ethylalcohol 1.4%	According to Manufacturer Spec	25-Sep-12
Ecolab®	Incidin® Extra	Benzylchloride-Glucoprotamine 2%	According to Manufacturer Spec	06-Jun-14
Ecolab®	Incidin® Foam	Glucoprotamine 26%	According to Manufacturer Spec	06-Jun-14

## CARRFLEX® Extreme - Approved Cleaners August 2019

**PLEASE REFER TO IMPORTANT MESSAGES ON FINAL PAGE BEFORE APPLICATION OF ANY CLEANER/DISINFECTANT.**

MANUFACTURER	PRODUCT	MAIN ACTIVE COMPONENT	CONCENTRATION	DATE TESTED
Ecolab®	Incidin® Plus	Glucoprotamine 26%	According to Manufacturer Spec	06-Jun-14
(Various)	Isopropyl Alcohol 70%		Ready to Use	25-Feb-10
Kimberly-Clark®	Isowipes®	Isopropanol 30-60%	Ready to Use	19-Jul-12
Schulke & Mayr GMBH	Kodan® Tinkture Forte	Hydroperoxide Solution 30%	According to Manufacturer Spec	06-Jun-14
Legge Systems	Legcide HB Plus		According to Manufacturer Spec	*****
Whiteley Medical	Lemex		According to Manufacturer Spec	15-Mar-11
Livingstone International (PTY) LTD	LIV-Wipes Jumbo	70% Isopropyl Alcohol	Ready to Use	15-Mar-11
Sulco LTD	Mediwipes		Ready to Use	27-May-09
Medline Industries Inc.	Micro-Kill™ Q3 Concentrated Disinfectant, Cleaner & Deodorizer		According to Manufacturer Spec	01-Feb-18
Medline Industries Inc.	MICRO-KILL™ AF2 Wipes	Ammonium Chloride	Ready to Use	15-May-19
Medline Industries Inc.	MICRO-KILL™ Bleach Germicidal Wipes	Sodium Hypochlorite	Ready to Use	15-May-19
Medline Industries Inc.	MICRO-KILL™ Bleach-Free, Alcohol-Free Wipes	Ammonium Chloride	Ready to Use	15-May-19
Medline Industries Inc.	MICRO-KILL™ One Wipes	Ethyl Alcohol	Ready to Use	15-May-19
Pro Restore® Products	Microban X-590 Institutional Spray	Diisobutylphenoxyethoxy ethyl dimethyl benzyl ammonium chloride	According to Manufacturer Spec	11-Aug-16
Anti-Slip Anywhere	Multi Kleen		According to Manufacturer Spec	15-Mar-11

## CARRFLEX® Extreme - Approved Cleaners August 2019

**PLEASE REFER TO IMPORTANT MESSAGES ON FINAL PAGE BEFORE APPLICATION OF ANY CLEANER/DISINFECTANT.**

MANUFACTURER	PRODUCT	MAIN ACTIVE COMPONENT	CONCENTRATION	DATE TESTED
Medizar®	Multi-Surface Sanitiser Wipes		Ready to Use	17-May-19
Ecolab® Cleantec	Nature Clean		1: 38 Water	17-Dec-10
Certo®	Non Acid Bathroom Cleaner		According to Manufacturer Spec	07-May-12
PreVasive®	Noroxycdiff™	Hydrogen Peroxide	According to Manufacturer Spec	22-May-19
Ecolab®	Oasis Pro 12 Neutral Cleaner	Alcohols <10%	5% Oasis Pro 12 (1:20 Water)	06-Mar-12
Schülke & Mayr GmbH	Ocetenisept®	100g solution contains: Octenidine dihydrochloride 0.1g, 2-Phenoxyethanol	According to Manufacturer Spec	06-Jun-14
Whiteley Medical	Odacon Spray		According to Manufacturer Spec	14-Jun-11
OdorXit Naturals	Odor Eliminator Spray		Ready to Use	22-Feb-18
Sealed Air Diversey Care	Oxivir® TB Liquid and Wipes		Ready to Use	22-Apr-13
Ecolab®	Oxycide		According to Manufacturer Spec	06-Jul-14
Pal® International	PAL® Detergent Wipes		Ready to Use	27-May-09
Pal® International	PAL® Disinfectant Wipes		Ready to Use	27-May-09
PCS Process Cleaning Solutions	PCS 7000	Sodium Hypochlorite	According to Manufacturer Spec	06-Oct-17
PCS Process Cleaning Solutions	PCS Spray & Wipe Hand and Surface Cleaner		Ready to Use	06-Oct-17
Pure Green, LLC	Puregreen 24 (Pure Green)		According to Manufacturer Spec	30-Jan-09

## CARRFLEX® Extreme - Approved Cleaners August 2019

**PLEASE REFER TO IMPORTANT MESSAGES ON FINAL PAGE BEFORE APPLICATION OF ANY CLEANER/DISINFECTANT.**

MANUFACTURER	PRODUCT	MAIN ACTIVE COMPONENT	CONCENTRATION	DATE TESTED
Reynard Health Supplies	Reynard Neutral Detergent Wipes		Ready to Use	08-Feb-07
Reynard Health Supplies	Reynard Surface Sanitizing Wipes		Ready to Use	27-May-11
Sealed Air Diversey Care	Room Care R2L		According to Manufacturer Spec	24-Oct-12
STERI-7®	S-7 Xtra Wipes		Ready to Use	28-Nov-16
Buckeye® International, Inc.	Sanicare TBX™		According to Manufacturer Spec	21-Sep-10
Buckeye® International, Inc.	Sanicare™ Disinfectant Wipes		Ready to Use	21-Sep-10
Alco Mandala Products	Sanni-Rinse		According to Manufacturer Spec	16-Feb-12
(Various)	Sodium Hypochlorite		1.3% (1.3:100 Water)	*****
Hu-Friedy Mfg. Co., LLC.	Sporeclear RTU Spray or Wipes	Benzalkonium Chloride 0-0.5%	Diluted at 8% Into Water	31-Jan-16
Clinell®	Sporicidal Wipes		Ready to Use	27-May-09
Ecolab®	Spring Clean		5% Spring Clean (1:20 Water)	11-Apr-06
Noble Pine Products Co.	Steri-Fab®	Isopropyl Alcohol	According to Manufacturer Spec	11-Aug-16
Hartmann	Sterillium®	Glucoprotamine 30%	According to Manufacturer Spec	06-Jun-14
sBioMed, LLC	Steriplex SD Disinfectant Sporicide Wipes (Without Bleach)	Hydrogen Peroxide	According to Manufacturer Spec	20-May-15
Sterri-Matt®	Sterri-Matt® (Pure) Foam		Ready to Use	15-Dec-09

## CARRFLEX® Extreme - Approved Cleaners August 2019

**PLEASE REFER TO IMPORTANT MESSAGES ON FINAL PAGE BEFORE APPLICATION OF ANY CLEANER/DISINFECTANT.**

MANUFACTURER	PRODUCT	MAIN ACTIVE COMPONENT	CONCENTRATION	DATE TESTED
Sealed Air Diversey Care	Stride® Citrus Smartdose	Ethoxylated Alcohol 20-30%	1:750 Water	01-Jun-15
Sealed Air Diversey Care	Stride® HC (JF)	Ethoxylated Alcohol 20-30%	According to Manufacturer Spec	01-Jun-15
Sealed Air Diversey Care	Suma San Conc D4A		According to Manufacturer Spec	28-Oct-14
Sealed Air Diversey Care	Suma San Conc D4A	Didecyl Dimethyl Ammonium Chloride	According to Manufacturer Spec	28-Oct-14
Spartan Chemical Company Inc.	Super HDQ Neutral®		256 to 1 dilution with water	11-Aug-16
PDI	Super Sani Cloth® Germicidal Disposable Wipes		Ready to Use	02-Sep-10
Anios Laboratories	Surfa 'Safe	Chlorhydrate 1.4%	According to Manufacturer Spec	06-Jun-14
Bioscience Fresche®	Surface Plus		According to Manufacturer Spec	09-Feb-15
Reynard Health Supplies	Surface Sanitising Wipes		Ready to Use	18-Feb-09
3M	TB Quat Disinfectant	Ammonium Chloride	Ready to Use	09-Mar-18
Buckeye® International, Inc.	Terminator™ One Step Disinfectant		According to Manufacturer Spec	21-Sep-10
Apollo Corporation	Therasol™		According to Manufacturer Spec	25-Sep-12
Medline Industries Inc.	Touch Screen Cleaning Wipes	70% Isopropyl Alcohol	Ready to Use	15-May-19
Hunters Products	TRI-DET		1:80 Water	02-Dec-14
(Various)	Trix (Soap)		Ready to Use	27-Apr-10

## CARRFLEX® Extreme - Approved Cleaners August 2019

**PLEASE REFER TO IMPORTANT MESSAGES ON FINAL PAGE BEFORE APPLICATION OF ANY CLEANER/DISINFECTANT.**

MANUFACTURER	PRODUCT	MAIN ACTIVE COMPONENT	CONCENTRATION	DATE TESTED
Verna Care	Tuffie 5 Cleaning Wipes	Cocoalkyl Dimthybenzyl Ammonium Chloride 0-0.5%	Ready to Use	19-Jul-12
Apollo Corporation	Turbo Clean™	Ethanol 9.5%	According to Manufacturer Spec	25-Sep-12
Texwipe®	TX®3270	70% isopropyl Alcohol	Ready to Use	12-Apr-19
Clinell®	Universal Disinfectant Wipes		According to Manufacturer Spec	27-May-09
Clinell®	Universal Wipes		Ready to Use	12-May-19
True Blue Chemicals	Use All	Glycol 1-6%	According to Manufacturer Spec	28-Sep-12
Whiteley Medical	Viraclean®	Benzalkoniumchloride	According to Manufacturer Spec	19-Jul-12
Sealed Air Diversey Care	Virex® II 256		1:256 Water	07-May-12
Sealed Air Diversey Care	Virex® Plus	Ammonium Chloride	According to Manufacturer Spec	19-Dec-18
Dupont™	Virkon®		5% Virkon (1:20 Water)	15-Dec-09
Wexford Labs	Wexcide 128		Minimum 128 to 1 dilution with water	11-Aug-16

**CARR NEW ZEALAND LIMITED**

 Tel: 64 9 274 0869  
 6 Ra Ora Drive, East Tamaki, Auckland 2013, New Zealand  
 PO Box 204110, Highbrook, Auckland 2161, New Zealand  
 info@carrgroup.co.nz

**CARR AUSTRALIA PTY LIMITED**

 Tel: 61 2 9636 9525  
 1 Spireton Place, Pendle Hill, NSW 2145, Australia  
 PO Box 335, Pendle Hill, NSW 2145, Australia  
 info@carrgroup.com.au





## CARRFLEX® Important Messages

### CAUTION!

**To avoid premature degradation of the surface coating never use an abrasive cloth, sponge or other similar device on the surface. Carefully follow manufacturer instructions for recommended exposure time of cleaner on surface. Wipe off cleaning product residue with a clean, soft cloth after cleaning and allow the surface to dry completely before use.**

### IMPORTANT INFORMATION:

Cleaners/disinfectants can cause discolouration/bleaching on some colours. To avoid discolouration, always check on a small area first.



Information contained here is purely related to the compatibility of cleaners with CARRFLEX® materials and in no way relates to the efficacy of any cleaner for infection control or the safety of any cleaner for skin contact.

Testing info is specific to the products or formulations listed and in no way constitutes approval for other products from any brand listed without being tested.

All approvals are based on manufacturer recommended dilution with water unless otherwise specified and approval is void for any improper dilution.

If a particular cleaner/disinfectant is not listed, we advise to test the specific product in advance or to contact us to conduct testing.

#### CARR NEW ZEALAND LIMITED

 Tel: 64 9 274 0869  
 6 Ra Ora Drive, East Tamaki, Auckland 2013, New Zealand  
 PO Box 204110, Highbrook, Auckland 2161, New Zealand  
 [info@carrgroup.co.nz](mailto:info@carrgroup.co.nz)

#### CARR AUSTRALIA PTY LIMITED

 Tel: 61 2 9636 9525  
 1 Spireton Place, Pendle Hill, NSW 2145, Australia  
 PO Box 335, Pendle Hill, NSW 2145, Australia  
 [info@carrgroup.com.au](mailto:info@carrgroup.com.au)