


MINI MASSAGER SM9126N

USER MANUAL

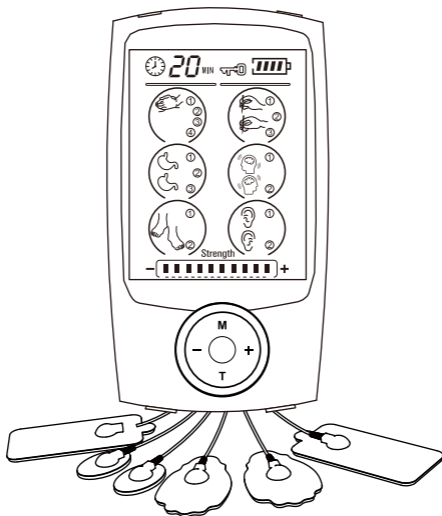

Explore Your Adventure

www.nursal.co/warranty

facebook.com/nursalonline

ACTIVATE YOUR 12 MONTH WARRANTY & GET EXCLUSIVE GIFT

Register within 2 weeks after receiving your new product.

PLEASE VISIT NOW

→ www.nursal.co/warranty

CUSTOMER SERVICE

✉ support@nursal.co

📘 @nursalonline



NOTE

1. Avoid touching the pads when the device is turned on.
2. Do not place pads near the heart, head or carotid.
3. Not suitable for use by pregnant women, people with implanted devices or heart disease.
4. Apply a few drops of water if the pads are not properly adhering to skin

A Big Thank You

We're thrilled that you have chosen to purchase the NURSAL Electronic Mini Massager. At NURSAL, we're committed to offering products that enhance your daily life.

Should you have any questions or feedback about your new product, please get in touch with our customer support team at 1-855-556-6888 or by email at support@nursal.co

Enjoy your new product!

CONTENTS

INTRODUCTION	01
SAFETY WARNINGS	02
TRANSPORT AND STORAGE	03
MAINTENANCE	04
PRODUCT SPECIFICATIONS	04
SET UP	05
OPERATING INSTRUCTIONS	08
MODE LIST	09
RECOMMENDED PLACEMENT	10
TIPS AND MAINTENANCE	12
TROUBLESHOOTING	13
WARRANTY CARD	15
CONTACT INFORMATION	16

INTRODUCTION

Our Electronic Pulse Massager delivers electronic impulses to tired and sore muscles via its electrodes. These impulses mimic signals coming from the central nervous system to trigger muscle contractions, which can be used for a variety of purposes such as increasing endurance, complementing exercise routines, or simple relaxation.

Our Electronic Pulse Massager is portable, compact, and stylish. It may be helpful in relieving minor aches and pains in various parts of the body such as the waist, shoulders, joints, hands and feet. The device is especially helpful for relieving stiff neck muscles and soreness resulting from intense exercise. Responsible, proper use may yield a variety of benefits for you and your well-being.

SAFETY WARNINGS

Danger

1. Do not use this unit if you are using an implanted electronic medical device such as a pacemaker.
2. Do not use this unit if you are wearing any electrode medical devices used for monitoring health functions

Warning














1. If you have any of the following conditions, please consult your physician before purchasing or using this device:
Acute conditions, malignant tumors, infectious diseases, pregnancy, implanted devices, heart disease, high fever, abnormal blood pressure, lack of skin sensation or numbness in the extremities, abnormal skin conditions or any condition requiring the active supervision of a physician.
2. Please remove the battery from the device when it is not in use.

Notice

1. Do not use this device while driving.
2. Do not use this device while sleeping.
3. Do not use this device in areas with high humidity such as a bathroom.
4. Keep this device out of reach of children.
5. Stop using this device and consult your physician at once if you feel pain, discomfort, dizziness or nausea.
6. Do not attempt to move the electrode heads while the device is in operation.
7. Do not use the device near the heart, on the head, near the mouth, genitals, or blemished areas of the skin.
8. Store the device in a cool, dry place, out of direct sunlight.
9. Store the device in a cool, dry place.
10. Dispose of batteries according to your local laws and regulations.

11. Portable and mobile RF communication equipment can affect medical electrical equipment. Exercise caution.

TRANSPORT AND STORAGE SYMBOLS

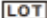	Fragile, handle with care		Type B applied part
	Keep the product in a dry place, away from		Attention, see operating manual for user
	This way up		Manufacturer
	Product package should be		Symbol for "AUTHORISED REPRESENTATIVE IN
	Unrecyclable		CE marking, Certificate issued by SGS
	Date of manufacture	FDA 510(K)	FDA 510(K) approved
	Serial number	IP22	IP code of the device
	Batch code		

MAINTENANCE

Device

1. Please use a lightly moistened cloth to clean the device, being sure to wipe gently.
2. Ensure that the device is turned off before attempting to clean.
3. Do not submerge or otherwise allow excessive contact between the device and water.
4. Storing the pads properly will enhance their longevity.

Pads

1. Make sure both pads not overlapping and make sure your skin is free from any dirt, oil or lotion.
2. Applying a few drops water before and after usage will increase the longevity of the pads.
3. Place pads on pad holder or place plastic film on the sticky side of pad to keep the gel moisture.

PRODUCT SPECIFICATIONS

Included in this standard package:

1. TENS device x 1 pc
2. Large pad x 2 pcs
3. Medium pad x 4 pcs
4. Small pad x 2 pcs
5. Electric wire x 3 pcs
(when use 4 line cable, make sure to use at least 1 pc white(positive) +1 pc blue line (negative) at the time.)
6. USB cable & AC adapter x 1 set
7. Instruction manual x 1 pc
8. Collection board for pad and wire x 1 pc
9. Bag x 1 pc


Features

- Large blue LCD
- 2 outputs with 16 modes
- 20 levels of intensity
- 20 minute default timer, can be adjustede from 10-60 minutes
- Rechargeable lithium battery
- Battery charge indicator
- AC adapter 110-240v, USB Charger 5v


SET UP

Unpack the box of the product, take the product and accessories out, and connect the electrode pad onto the device.

1. The device has two independent channels A&B . Make sure which channels (channel A or B or both) you want to use before you connect wires to the device.
2. Put pads on your skin before turn on the device.Use at least one pair of pads, one pad alone will not work.

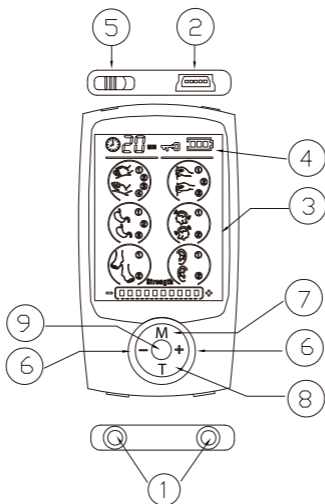

- | | |
|---------------------------|-------------------------|
| 1. Two output connector | 2. USB port |
| 3. LCD screen | 4. Battery indicator |
| 5. On/ off switch | 6. Intensity adjustment |
| 7. Mode selector | 8. Time adjustment |
| 9. Built-in Mode Selector | |

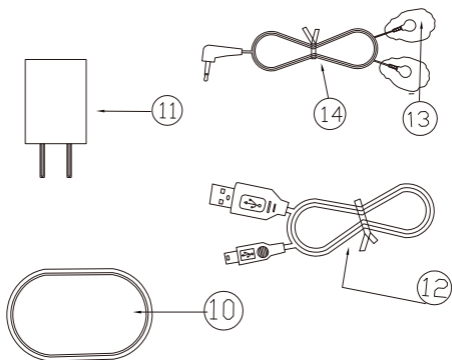

10. Pads- adhesive board

11. Charger

12. USB cable

13. Massage pads

14. Pad wires

OPERATING INSTRUCTIONS

1. Connect the pads to the wire by clipping them on, and then remove the protective film.
2. Place the pads on the desired area, and plug the wires into one of the ports on the device (Channel A or Channel B). For additional relief, use an additional pair of pads plugged into the other port on the same area.

Ensure both pads are not overlapping, and that your skin is free from any dirt, oil, or lotion.

3. Turn the device on by sliding the on/off switch at the top of the unit to the 'on' position. The unit is now activated.
4. Press the "M" button to select a different main mode (there are 6 main modes to choose from, each main mode contains different sub-modes). Press the center button to select a different sub-mode within the current main mode.

The strength setting will default to the lowest setting upon selecting a new mode.

5. Increase the intensity by pressing the + button; decrease the intensity by pressing the - button.
6. To adjust the timer, press the "T" button. 10,20,30,40,50,60mins timers are available. The default time is 20 minutes.
7. Turn off the device.
8. Remove the pads from your skin

Gently remove the pads from your skin by pulling from the corner of the pad, then return them to the provided pads holder or protective plastic film

MODE LIST

There are 6 main modes. The "M" cycles through the 6 main modes, while the silver middle button selects options (2 to 4 varieties) within that mode.

There are 16 individual modes in total:

1. Kneading

- 1) Lateral palm kneading
- 2) Flat palm kneading
- 3) Crosscut fist kneading
- 4) Vertical fist kneading

2. Acupuncture

- 5) Acupuncture
- 6) Moxibustion
- 7) Acupuncture kneading

3. Cupping

- 8) Rubbing reduction
- 9) Cupping
- 10) Fibrillation massage

4. Tapping

- 11) Lifting-pinching manipulation
- 12) Vibrating manipulation

5. Shiatsu-Deep

- 13) Thai massage
- 14) Shiatsu

6. Auto-Combination

- 15) Auto-Combination Massage Mode 1
- 16) Auto-Combination Massage Mode 2

RECOMMENDED PLACEMENT



LOWER BACK

Mode 1,4,5,6,7,10,11

Attach both pads on the lower back
with the backbone in the center.

Do not place on the backbone or spine



SCIATIC

Mode 1,2,3,4,5

Attach on pad below

and above the region in pain,

both on the same side(NOT on spine)



DELTOID

Mode 2,3,4

Attach one pad in front
and one in back of the muscle.



ARM

Mode 1,2,3,6,7,15,16


Attach both pads
on either side of the area.



KNEE

Mode 1,2,3,4,5,6,7,10

Attach both pads above the knee
or above and below joint.

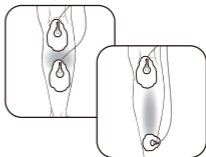


CALF

Mode 1,2,3,4,5,6,7,8,14,15

Attach both pads on the calf/leg
where you feel pain.


(Do not place electrode pads simultaneously
to the calves of both legs.)

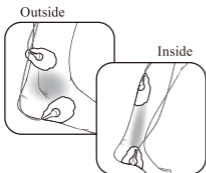

ANKLE/FOOT

Mode 1,2,14,15

Attach pads per the illustration,
on the left/ on the outside of your ankle/foot.
Attach the pads per illustration on the right/
on the other side of your ankle/foot

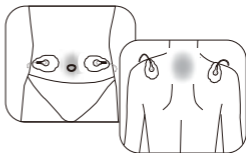

ABS/SHOULDER

Mode 1,4,5,6,7,10,11,12,14,15,16

Attach pads per the illustration.
(Do not put pads on the spine)

TIPS AND MAINTENANCE

- It is suggested that you do not use the massager on any given area of skin for a duration greater than 20 min 2 times per day. Consult with your physician for longer or more frequent uses.
- Start from the lowest intensity and gradually adjust the intensity to a comfortable level on a scale from 1 to 20.
- Good skin care is important to ensure comfortable use of the device. Be sure the treatment site is clean of dirt and body lotion.
- Keeping the electrode in its case after use will extend its lifespan. The electrode is disposable and should be replaced when it loses its adhesive. To purchase additional electrodes, please contact us at support@nursal.co.

TROUBLESHOOTING

If you encounter issues with your device, consult this troubleshooting guide. If you have any other questions, please contact us at support@nursal.co or 1-855-556-6888

Problem	Possible Cause	Solution
One pad feels stronger than the other	This is normal. Different areas of your body will react differently	Nothing needs to be done. Make sure the pads are moist and making good contact.
The intensity feels very weak, even at the highest settings	Pads are not attached to the body firmly	Attach both pads firmly to the skin
	Pads are not attached to the body firmly	Peel off the film on the adhesive surface of pads
	The transparent films are still on the pads	Do not allow pads to overlap.
	The cord is not properly connected to the unit	Connect cord securely into the jack
	The intensity seems to decrease over time at the same level.	Increase the intensity level as the sensation becomes less effective.
	The battery capacity is low	Charge the battery
Skin turns red or the skin feels irritated	The adhesive surface of the pads is dirty or dry	Wash adhesive surface of pads gently with your fingertips for about 3 seconds under slow running water
	The therapy time is too long or the intensity is set too high	Reduce the duration of use or intensity
	The electrode pad surface is worn out	Replace electrode pad. Contact us.

Problem	Possible Cause	Solution
No power source; nothing displayed on LCD.	The battery is depleted	Charge the battery
Power cuts off during use	The battery is depleted	Charge the battery
	The cord is broken	Replace the cord
It is difficult to attach the pad to the skin	The film on the surface of the adhesive pads has not been removed	Peel the film off the adhesive surface of pads
	The pad was applied to wet skin	Dry the pad
	The adhesive surface of the pad is damaged	Replace the pad
Adhesive surface of pad is not sticky	The pads have deteriorated	Contact us for replacements.
	Were the pads stored in high temperature, high humidity, or direct sunshine?	Replace the pad.

WARRANTY CARD

Certificate of Quality Warranty Card	
Product model	SM9126N
Product name	Electronic Mini Massager
Date of purchase	
Purchase Location	
Retailer's Name	
Retailer's TEL	
Problems of the Product	

CONTACT INFORMATION



Manufactured by:

Hong Qiangxing (Shenzhen) Electronics Limited Address:4F,
Jingcheng Building,Xicheng Industrial Area, Xixiang
Street,Bao'an District,Shenzhen,
Guangdong,China

Distributed by:

OMGD INC
ADD:245 East Main Street,Suite 115,Alhambra,CA 91801

Made in china

WEEE-Nr : DE83395812

ACTIVATE YOUR 12 MONTH WARRANTY & GET EXCLUSIVE GIFT

Register within 2 weeks after receiving your new product.

PLEASE VISIT NOW

→ www.nursal.co/warranty

CUSTOMER SERVICE

✉ support@nursal.co

📘 @nursalonline



NOTE

1. Avoid touching the pads when the device is turned on.
2. Do not place pads near the heart, head or carotid.
3. Not suitable for use by pregnant women, people with implanted devices or heart disease.
4. Apply a few drops of water if the pads are not properly adhering to skin

NURSAL

✉ support@nursal.co

📍 www.nursal.co

☎ 1-855-556-6888

WEEE-Nr: DE83395812



Made in China