

Hospital SNOO Bundle



Each Hospital SNOO Bundle includes:

- 1 Hospital Bassinet
- 1 SNOO Mobility Platform
- 1 Reusable Hospital Bassinet Mesh Cover
- 1 Reusable/Sealed Hospital Mattress
- 10 Disposable Mesh/Mattress Cover Sets
- 1 Set SNOO Sacks (S, M, L - 1 of each)
- 4 Additional SNOO Sacks (size S)
- 5 Additional Fitted Mattress Sheets

Cleaning Protocols

Prior to Use:

- Remove the storage bag covering the SNOO Smart Sleeper.
- Place a waterproof white reusable mesh cover OR a blue disposable mesh cover to completely cover the bassinet walls.
- (If using disposables) Place the sealed mattress inside the removable blue waterproof cover.
- Place a clean mattress sheet over the covered mattress.

Cleaning Instructions:

- *For reusable covers* - Use CaviWipes or a hydrogen peroxide-based disinfectant to clean if visibly soiled, between patients, and every 7 days (if in use).
- *For the SNOO* - Use CaviWipes or a hydrogen peroxide-based disinfectant to wet all exposed surfaces on the bassinet. The surface should remain visibly wet for 1 minute at room temperature.
- Clean SNOO Sacks and sheets per standard Hospital Laundering of Linens policy.
- For storage, cover cleaned SNOO with a plastic bag to designate it as clean and ready for use.