### **Infection Control Covers**

## **DISPOSABLE MESH & MATTRESS COVERS**



### **SPECIFICATIONS**

- Medical grade, breathable fabric
- 100% Latex-free Polypropylene
- Fabric meets AAMI
   PB70 Level 3 Testing



### **USAGE INSTRUCTIONS**

- For single patient use. DO NOT share covers between babies.
- Dispose of and replace cover if damaged, cut, or torn. Cover may be used for baby's length of stay if material is clean and intact.
- Whenever SNOO is used, swaddle the baby in the SNOO Sleep Sack and attach the sack wings to the safety clips in the bassinet.

To order additional covers, visit health.happiestbaby.com or email us at health@happiestbaby.com.

### **ASSEMBLY INSTRUCTIONS**

- 1 Remove mattress pad from inside SNOO bassinet.
- 2 With the disposable bassinet cover label facing out, align the cover label with the warning label at the back of SNOO.
- 3 Loosely drape the cover material inside the SNOO.
- 4 Pull the cover down along the outer perimeter of the SNOO on one side and then the other.
- Inside the bassinet, you will find an elastic opening on each side near the safety clips. Stretch the openings around the safety clips on both sides.
- 6 Slide the mattress into the disposable cover. Keep the label on the same side as the opening.
- 7 Cover the mattress with a cotton fitted mattress sheet and place it on top of the bassinet cover.



Bassinet Cover Label



Safety Clips



Mattress with Cotton Sheet



Covers with Sheet

# Infection Control Covers REUSABLE MESH COVERS



### **SPECIFICATIONS**

- Waterproof and flexible fabric
- 75% Polyester
- 25% Polyurethane
- 100% free of PFAS and PFOA
- CPSIA certified



### **USAGE INSTRUCTIONS**

- Covers MUST BE cleaned between patients to meet hospital infection control requirements.
- Additionally, clean the cover and sealed mattress if visibly soiled and every 7 days of use.
- Replace if damaged, cut, or torn.
- Whenever SNOO is used, swaddle the baby in the SNOO Sleep Sack and attach the sack wings to the safety clips in the bassinet.


### **HOW TO CLEAN**

**OPTION 1:** Wet all exposed surfaces on the mesh and mattress covers with *CaviWipes* or *hydrogen peroxide-based disinfectant*. The surface should remain visibly wet for 1 minute at room temperature.

**OPTION 2:** Wash on hot setting between 160-165°F using liquid detergent; may be washed with zero-residue, oxygenated bleach containing no chlorine. Dry on medium heat (<130°F).

### **ASSEMBLY INSTRUCTIONS**

- 1 Remove mattress pad from inside SNOO bassinet.
- With the shiny side of the cover facing out, locate the warning label and align it with the warning label at the back of SNOO.

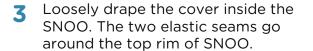

- 4 Situate the elastic seams so that the seams sit on either side of the top mesh rim of the SNOO.
- 5 Pull the cover down along the outer perimeter of the SNOO on one side and then the other.
- 6 Inside the bassinet, you will find an elastic opening on each side near the safety clips. Stretch the openings around the safety clips on both sides.
- 7 Cover the mattress with a cotton fitted mattress sheet and place it on top of the bassinet cover.

NOTE: Covers are designed for a snug fit. Material is durable and can be stretched.



Elastic Seams



Safety Clips



Cover on Bassinet



Mattress with Cotton Sheet