

SNOO SLEEP SACK

SPECIFICATIONS






- For use with the SNOO Smart Sleeper bassinet.
- 100% Organic cotton swaddle with special wings that attach to the SNOO's safety clips to prevent infant rolling.
- Designed to keep sleeping babies supine as advised by the American Academy of Pediatrics.
- Inner arm wrap and between-the-legs flap with quiet velcro for escape-proof security.
- 2-Panels of breathable mesh to reduce overheating.
- Arm openings can be unsnapped to allow the baby's arms out of the sack, if desired.
- Double zipper enables user to zip from top or bottom.
- Flexible material supports full head, neck, & legs movement and hip abduction.

SIZE GUIDE	
X-Small	5-8 lb
Small	5-12 lb
Medium	12-18 lb
Large	18-26 lb

HOW TO CLEAN

- Launder in accordance with standard hospital laundering practices and protocols; a temperature of at least 160°F (71°C) for a minimum of 25 minutes is recommended.
- SNOO Sacks MUST BE cleaned between patients and when visibly soiled.

INSTRUCTIONS

-  Lay your baby down on top of unzipped SNOO sack.
-  Fold over left side of inner band.
-  Secure the velcro on right side of inner band.
-  Pull diaper flap up and secure over the band.
-  Done!

NOTE: When SNOO is in use, always swaddle the baby in the SNOO Sack and attach the wings to the safety clips.