

Adam, Rouilly

SERVING MEDICAL EDUCATION WORLDWIDE

AR251/AR252-B INJECTION, VENEPUNCTURE, CANNULATION & INFUSION ARM

Product Leaflet





AR251 INJECTION, VENEPUNCTURE, CANNULATION & INFUSION ARM, WHITE
AR251-B INJECTION, VENEPUNCTURE, CANNULATION & INFUSION ARM, BLACK



Cast from life, our best selling Venepuncture and Infusion Arm been Trainer has been improved and now has more pronounced veins in the antecubital fossa region of the forearm and on the back of the hand.

With an extremely realistic finish, it shows a well developed male left arm in fine detail.

The arm features a flexible wrist for added realism, durable silicone skin and is simple to set up, operate and maintain.











AR251 Infusion Arm, White



AR251-B Infusion Arm, Black

Features

-  Easy to set up, operate and maintain
-  Continuous, closed vein system is clean to use. An innovative two bottle sealed blood assembly with automatic pressure release valve greatly reduces the risk of leakage
-  A realistic sensation is felt when the skin and vein are penetrated. Lifelike blood 'flashback' confirms correct needle location in the vein and blood can then be withdrawn
-  Flexible wrist for improved realism
-  The skin is moulded from a specially selected grade of silicone to improve feel and durability. Palpation of veins and identification of working and 'blind veins' is also possible.
-  The material used for the veins, and the silicone skin have self sealing characteristics that can be penetrated many times, enhancing durability
-  Many procedures can be carried out before a replacement skin is required. Replacement skins and veins are simple to fit

 Scan the QR code to access the product video <https://youtu.be/Y9wr5EP300o>



Skills

- Intravenous cannulation - on the back of the hand or forearm
- Setting up and monitoring infusions including application of adhesive dressings to secure cannula in place
- Withdrawal of blood using syringe or vacutainer

Accessible Veins

- Cephalic
- Basilic
- Median Cubital
- Dorsal metacarpal

Minimising Spillage and Staining

- The innovative sealed blood assembly reduces the risk of spillage as fluids can easily be pumped from one bottle to the other
- Special blanking plugs for the bottles are provided when transporting the arm



Simple set up



Unique pressurised blood system

Includes

- Arm
- Arm rest
- Blood bottle assembly (2)
- Blanking plugs for transportation (2)
- **AR204 BLOOD CONCENTRATE** to make up 2.5 litres artificial blood
- Instruction manual
- Rigid carrying bag for easy storage



Vein palpation

Replacement Parts

AR204 BLOOD CONCENTRATE TO MAKE UP 2.5 LITRES ARTIFICIAL BLOOD

AR253 SKIN AND VEIN SET, WHITE for AR251


AR253-B SKIN AND VEIN SET, BLACK for AR251-B

SPAR251 REPLACEMENT BOTTLES, PRESSURE SYSTEM AND ARM REST

Renewal Service

The AR251 and AR251-B can be re-skinned and re-veined. This is a relatively simple procedure for customers to undertake. However, if you prefer we can carry out this service for you.


Please contact our Sales Department for further information on this service.

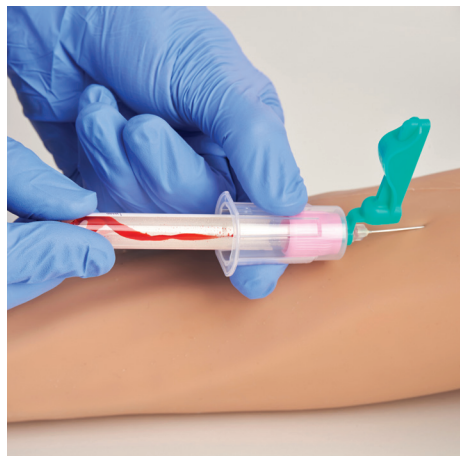


Realistic flashback



Vacutainer blood withdrawal

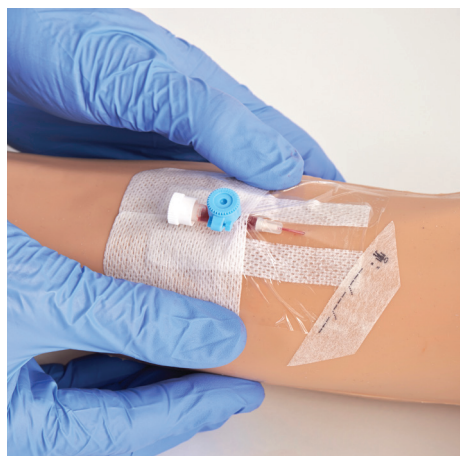

Cannula insertion. Adhesive dressing application

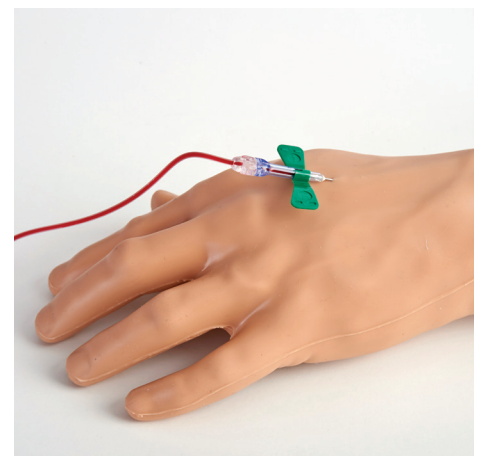

Cannula insertion in the back of the hand



2 Year Guarantee

All products manufactured by Adam,Rouilly are covered by our full 2 Year Guarantee. This guarantee applies to models which have been used correctly and covers durability and functionality.



Adam,Rouilly

As part of our policy of continual product development, the specification of products may alter without prior notice.

Adam,Rouilly has over 100 year's experience in providing quality medical models and simulators.

Should you require any further information please contact our Sales Department who will be pleased to help with your enquiry.

Adam,Rouilly Limited
Castle Road
Eurolink Business Park
Sittingbourne
Kent
United Kingdom
ME10 3AG

T: +44(0)1795 471378
F: +44(0)1795 479787
E: sales@adam-rouilly.co.uk
www.adam-rouilly.co.uk