

# **Instructions for use of Medicor-Lab product**

## **Intra-Venous ARM or HAND**

### First use:

- Make sure that the roll clamps are closed securely.
- Make sure that the collets are closed properly.
- Fill the fluid supply bag.
- Install the bag on the support of your choice so that you can use the integrated dropper in the tubing.
- Check that the unions of tubing are well joined.
- Position the tank bag at the same level as the IV product used.
- Check that the tank bag cap is closed.
- Open the collets to evacuate the air when filling the veins.
- Open the tank roll clamp completely.
- Open the feed roller clamp gently to allow the flow of fluid.
- Close the feed roller clamp as the liquid appears in the tank bag.
- When performing your training, adjust the fluid flow to one drop per second to avoid emptying your bag into the tank bag.
- Performed your training.
- Closed the power roller clamp.

### Next use:

- Make sure that the power roller clamp is securely closed.
- Fill the fluid supply bag as needed from the reservoir bag.
- Open the feed roller clamp gently to adjust the flow of fluid to one drop per second to avoid emptying your feed bag into the tank bag.
- Performed your training.
- Closed the power roller clamp.

### Storage:

- Make sure that the roll clamps are closed properly.
- Make sure that the collets are closed properly.
- Disconnect the bags of product IV by means of tubing unions.
- Empty the bags of their fluids by their caps.
- Open the roll clamp of each bag to purge the liquid from the tubing.
- Open the collets to purge fluid from IV
- Close the collets and the roll clamps.

### Notice of use:


- Please note that the product has deteriorated with each insertion of equipment into the component material (we recommend a maximum of 22G as the needle size).
- Holding the tank bag at the same level as the IV product makes it possible to increase the number of uses without the appearance of fluid droplet due to the previous insertions

For more information visit our channel : <https://www.youtube.com/channel/UC5OHLn9rLwI2TwHIQTpXntQ>



# **MEDICOR LAB**

[www.medicor-lab.com](http://www.medicor-lab.com)

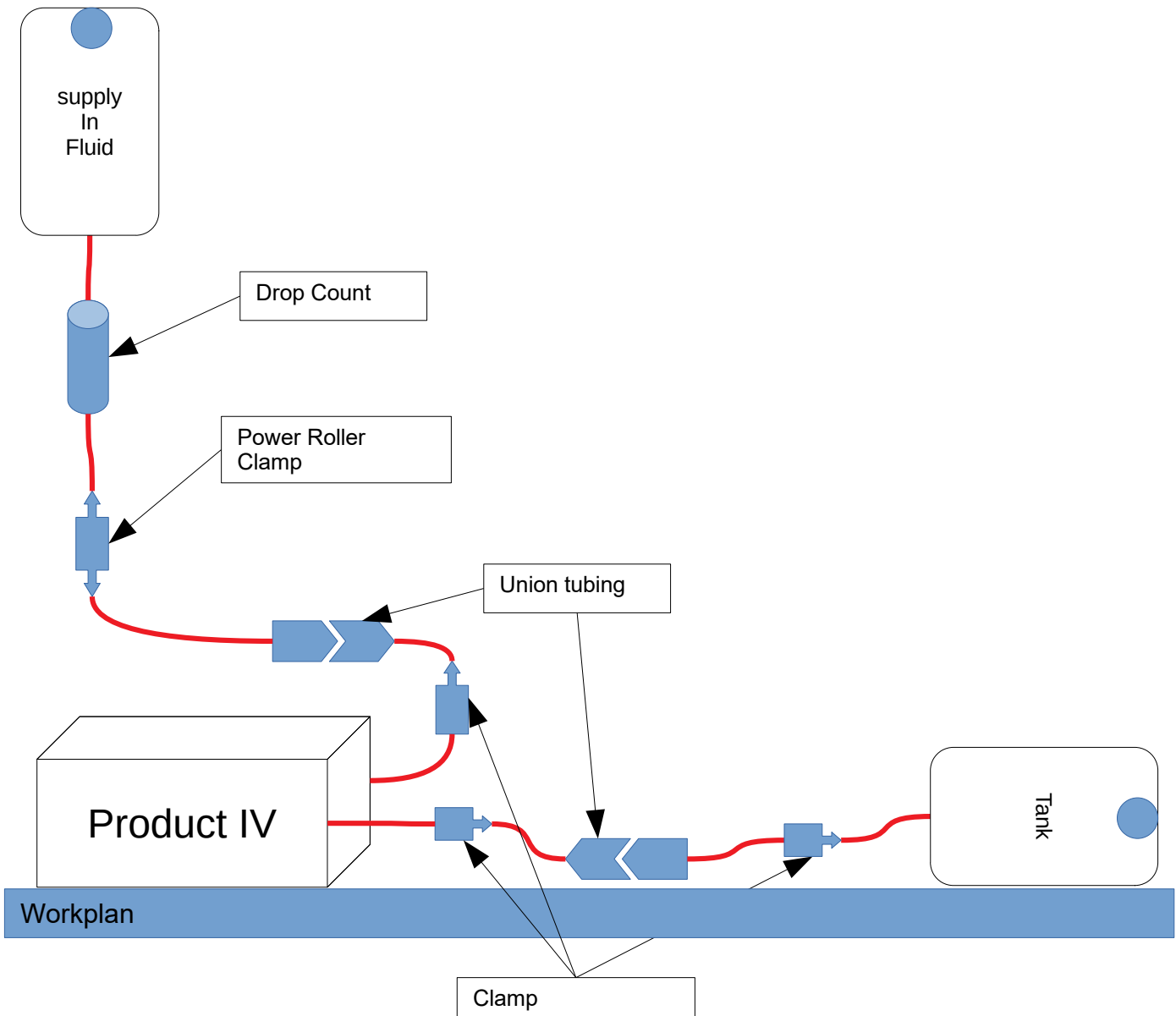

**Notice of use:**

- Please note that the product has deteriorated with each insertion of equipment into the component material (we recommend a maximum of 22G as the needle size).
- Holding the tank bag at the same level as the IV product makes it possible to increase the number of uses without the appearance of liquid droplet due to the previous insertions

For more information visit our channel : <https://www.youtube.com/channel/UC5OHLn9rLwl2TwHIQTpXntQ>



**MEDICOR LAB**

www.medicor-lab.com