

# Neo Nate Quick Start Guide

## Quick Start Guide: Neo Nate 101-340

### What's Included:

- Hand Pump System
- Squeeze Bulb to simulate birth cries
- Stethoscope
- Umbilical Cord
- Sheets (2)
- Head Cap
- Carry Bag
- Umbilical Clamps (2)

### Product Assembly

1. Connect tubing nodules to corresponding color-coded attachments on chest overlay (See Fig. 1 and 2).
2. Attach umbilical cord to the umbilical stump (See Fig. 3).
3. Product is ready for use (See Fig. 4).

For more information on this product, please contact our customer service team at 1-833-NASCOHC (627-2642) or email us at [info@nascohealthcare.com](mailto:info@nascohealthcare.com). We are happy to help you!

For complete instructions, please refer to the included QR code.



Figure 1

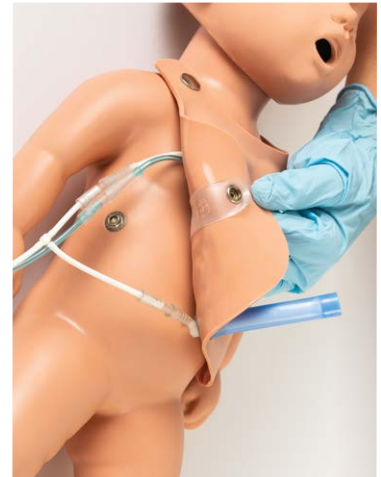


Figure 2



Figure 3



Figure 4



**Nasco Healthcare**  
16 Simulaid Drive  
Saugerties, NY 12477  
1-833-NASCOHC (627-2642)  
[info@nascohealthcare.com](mailto:info@nascohealthcare.com)  
[www.nascohealthcare.com](http://www.nascohealthcare.com)

©2023 Nasco Healthcare

PC 101040 09/23