


HAND / WRIST CONDUCTIVE INJECTION MODEL


SB15115 SET-UP INSTRUCTIONS

Thank you for purchasing the hand/wrist conductive injection model.

This injection trainer can be used to teach or demonstrate needle placement for aspiration and joint injection. With the correct needle insertion, an audible tone will sound indicating proper placement of the chosen area. Palpation of the major anatomical landmarks may also be performed. Additional conductive structures may be added upon request. With a little care and maintenance your new model should provide you with many years of service.

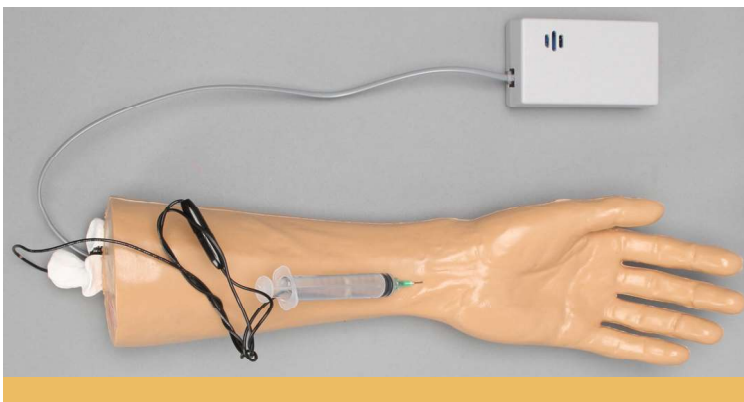

OPERATION

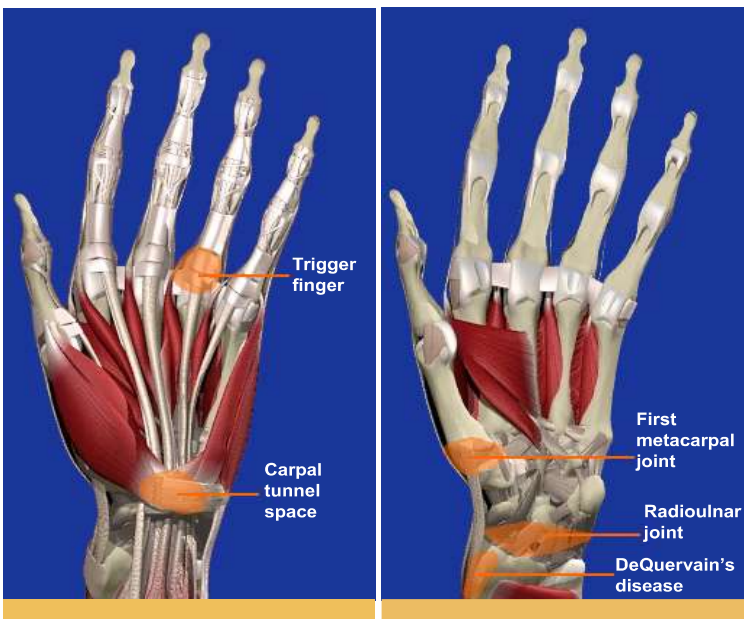

Locate the appropriate spot for injection (i.e. joint space bursa).
Insert needle into location. The audio feedback box will emit a tone when the tip of the needle has made contact with the correct area.
Use the dry erase marker to identify injection targets.

COMPONENTS

Hand/Wrist Conductive Injection Model include:


- Soft tissue injection hand/wrist
- Audio feedback box with leads
- Needle assembly with lead
- Dry erase marker



This model contains these conductive anatomical points:

- Trigger finger
- Carpal tunnel space
- First carpal metacarpal joint
- DeQuervain's disease
- Radioulnar & carpal joint

SET-UP INSTRUCTIONS

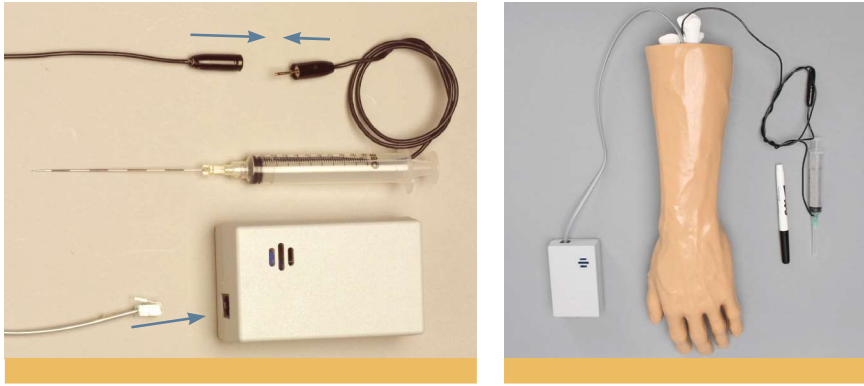

Step 1: Lay out all parts and inspect for shipping damage.

Step 2: Insert wire lead from syringe into the black wire.

Step 3: Insert male phone connector into audio feedback box.

REPLACEMENT PARTS



CARE & MAINTENANCE

IMPORTANT

To maintain audio feedback, do not remove plunger assembly or needle from the syringe. This model is NOT intended for injection of fluids.

Your model can be wiped down with any mild cleaner. Use only the dry erase marker on the model. Standard ink can not be removed. Store in a dry, dark place out of direct contact with sunlight.