

INJECTION KNEE WITH AUDIO FEEDBACK

SB15176 SET-UP INSTRUCTIONS

Thank you for purchasing the
conductive knee
injection model.

This injection trainer can be
used to teach or demonstrate
needle placement for aspiration and
joint injection. With the correct needle insertion, an audible
tone will sound indicating proper placement of the chosen area.
Palpation of the major anatomical landmarks may also be performed.
Additional conductive structures may be added upon request. With a
little care and maintenance your new model should provide you with
many years of service.



SB15176 and SB1517611
Stand is ordered
separately.



COMPONENTS


include:

- Soft tissue injection knee
- Audio feedback box
- Needle assembly with lead
- Dry erase marker



OPERATION AND ANATOMICAL FEATURES

This Knee injection Model contain these anatomical points:

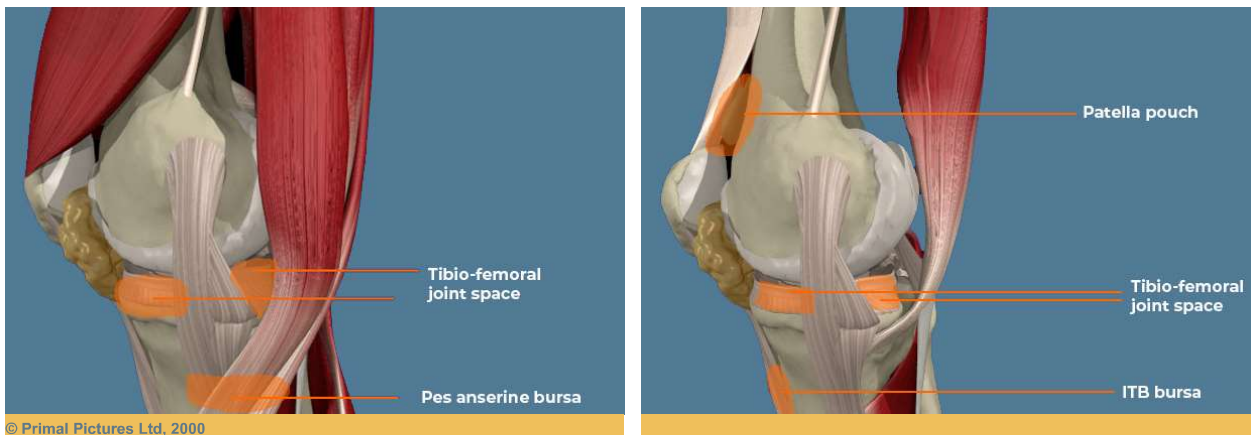

Locate the appropriate spot for injection (i.e. joint space bursa).
Insert needle into location. The audio feedback box will emit a tone when
the tip of the needle has made contact with the correct area.

Use the dry erase marker to identify injection targets.

SET-UP INSTRUCTIONS



SB15176 SET-UP INSTRUCTIONS



Step 1: Screw purple EMG needle onto syringe. Note that this is a standard Leuer lock connection.

Step 2: Plug the black male end pin plug from the syringe into the female black cord end coming from the knee.

Step 3: Plug the gray phone cord end into the receiving jack on the emitter.

Step 4: The battery is stored outside of the emitter box to extend life. Remove back cover from emitter box and install battery.

REPLACEMENT PARTS



SB1485106

Knee Skin Sleeve Replacement




SB162013

Conductive Needle Set, 2" Needle length



SB1620

Conductive audio feedback emitter

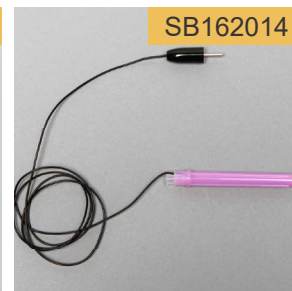

SB162014

Needle Assembly with Lead, 2" Needle length

CARE & MAINTENANCE

IMPORTANT

To maintain audio feedback, do not remove plunger assembly or needle from the syringe. This model is NOT intended for injection of fluids.

Your model can be wiped down with any mild cleaner. Use only the dry erase marker on the model. Standard ink can not be removed. Store in a dry, dark place out of direct contact with sunlight.