M184 Sakamoto Obstetric Assistant Trainer



User's Manual (151218)

-INDEX-

| Introduction/Table of Contents | 1 |
|--|---|
| • Purpose/Content | 2 |
| • Cautions on Use | 3 |
| • Instructions for Use/Storage Instructions | 4 |

Note

Confirm the main unit and accessories directly after the product arrives. If there is any defect or breakage, inform the dealer or Sakamoto Model.





Introduction

Thank you for purchasing our product, "Sakamoto Obstetric Assistant Trainer."

Read this user's manual carefully in order to use the product for a long time appropriately.

This product is closely checked before factory shipment. However, when this product arrives, open it soon, and check if anything on the main unit and accessories are missing or broken.

Table of contents

| Introduction / Table of Contents | 1 |
|---|---|
| Purpose / Content | 2 |
| Cautions on Use | 3 |
| Instructions for Use / Storage Instructions | 4 |

Purpose

This product is used to perform the following training:


- 1) Rotating the fetus
- 2) Protecting the perineum
- 3) Checking the placenta and amnion
- 4) Supine position and positioned on one's hands and knees

Content

1) Obstetrics model unit (with 1 genital model (skin color)

and 4 screws) - 1 set

- 2) Replaceable genitalia (transparent) Qty. 1
- 3) Vinyl fetus Qty. 1
- 4) Placenta (with umbilical cord and amnion) Qty. 1
- 5) Exchangeable umbilical cord Qty. 1
- 6) umbilical cord with connector and hook Qty. 1
- 7) Lubricant (hand soap) Qty. 1
- 8) Powder Qty. 1

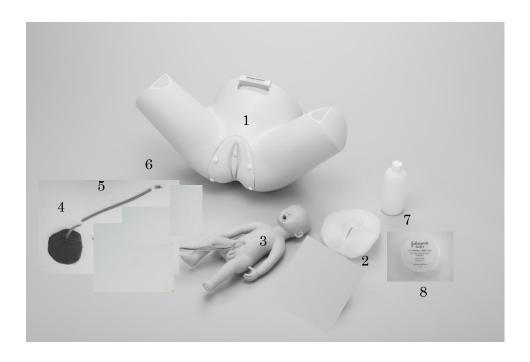

Cautions on Use

- 1) If the resin made products are fitted onto the genitalia for long periods of time, the genitalia may deform. Therefore, handle with care when using them.
- 2) Always perform training in groups with a minimum of two people.
- 3) Be careful when handling the training product because the genitalia material gets dirty easily. In addition, be particularly careful with ballpoint pens and magic markers because their marks cannot be removed.
- 4) If the genitalia get slightly dirty, a neutral detergent can be used to clean it. (Do not use any solvents such as thinners, because the resin will dissolve.) Then, use a dry cloth to remove all remaining moisture and apply the powder.
- 5) The frame for the genitalia can be removed for washing, etc. However, always store the product with frame attached.
- 6) If any excessive load is applied or if dropped, it will damage the model.
- 7) When pushing out the fetus, always apply the lubricant first to ensure a smooth delivery.
- 8) After you finish training, thoroughly clean the lubricant that is on the fetus and genitalia.
- 9) When attaching the genitalia to the training model unit, make sure that the screw holes on the genitalia match the screw holes on the model before inserting the screws. If you force the screws when inserting them, it may damage the product.
- 10) If a defect or problem with the product is discovered during use, stop using the product immediately and contact Sakamoto Model or an authorized dealer.
- 11) This product is a simulator capable of supporting advanced medical training. However, Sakamoto Model does not guarantee the same results during actual treatment.
- 12) Please note that the specifications of this product may be changed without prior notice in order to make improvements to the product.

Instructions for Use

- 1) Take out the training model unit from the hard case, and place it on a table with a non-slick surface in the desired position.
- 2) Please set the umbilical cord to Placenta and fetal.
- 3) Apply lubricant to the back of the fetus' head, shoulder and genitalia of the main unit.
- 4) The person in charge of the fetus is positioned at the rear of the training model and insert finger in mouth of the fetus. (This makes it easier to handle the fetus.)
- 5) The person in charge of the fetus holds the handles of the training model, positions the fetus in the mother's body and pushes out the fetus.
- 6) Begin training.
- 7) To replace the genitalia, turn the 4 screws that attach the genitalia to the training model in a counterclockwise direction, and remove the genitalia.

Storage Instructions

- 1) First, thoroughly clean the lubricant that is stuck to the fetus and genitalia, and then dry out the pieces in the shade. If the genitalia become sticky, apply powder before storing.
- 2) Store the genitalia so that no excessive load is applied to it. Otherwise, the genitalia may become deformed.
- 3) Place the product inside the special case, and avoid storing the product in direct sunlight or in locations with high temperature and humidity.

SAKAMOTO MODEL C O R P O R A T I O N

Head office 34,Higashitakagicho,Shimogamo Sakyo-ku, KYOTO 606-0865, JAPAN

Phone: +81-75-701-1135 Fax: +81-75-722-5638

E-mail:info@sakamoto-model.com URL:http://sakamoto-model.co.jp

Overseas Sales Department 1-9-29,Senbahigashi,Minoh City,Osaka 562-0035,JAPAN

> Phone: +81-72-729-8350 Fax: +81-72-729-8385

E-mail:trade@sakamoto-model.com