

User manual for the tremor simulator

**Note also the instructions supplied with the device.
Not for use with pacemakers or heart problems.**



1. Apply gloves to both hands



Two pairs of gloves in two sizes are supplied. Please choose the appropriate size. If necessary, you need to change the cables with the snaps. Apply the glove with the black connector right in case of a right-handed and left in case of a left-handed person. Wet the hands before to improve the conductivity if the effect doesn't occur sufficiently.

2. Switch on the control unit and increase the intensity very slowly


The effect occurs a little time-delayed. Do not increase the intensity any further when the hands tremble. The hands should be held loosely. The user can interrupt the effect at any time by bringing the hands together.

You can use either channel 1 or channel 2. We deliver with the assignment of channel 2.

To change the channel, plug the connection cable into the corresponding connector.

The channel that is not in use should always be switched off.

Always operate only the knobs on the side with the connection cable plugged in.

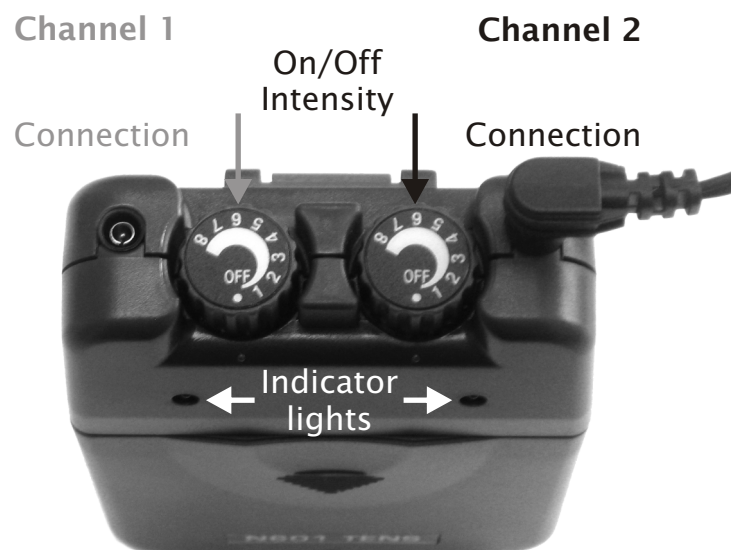

User manual for the tremor simulator

Default settings

The control unit is delivered preset. The controls for the settings are located under the lid, which you can pull down. Do not change the default settings arbitrarily. Changing the settings will void any liability.



Changing the battery

If the effect is no longer sufficient, the battery may be discharged. If you pull the lid down completely, you can replace the 9-volt block battery in the control unit with a new battery. Make sure that the polarity is correct when inserting the battery.

User manual for the tremor simulator

Safety instructions for the use

Through the tremor there is an increased risk of accidents while being busy with spiky, sharp or fragile objects. Objects made of metal can conduct the electricity. Thus, use only harmless non-metal objects for exercises with the Tremor Simulator.

Recommended are for example:

Cups, dishes and cutlery made of plastic for eating exercises

Pens made of plastic or wood for writing exercises

Please note that objects can easily slip out of the test persons hands. It is ideal if the test person is not operating the control unit by himself but is supervised by an expert. This person should observe the test person, operate the equipment cautiously and particularly increase the stimulation current to the desired intensity.

No operating of devices especially electrical devices!

No operating of vehicles and machines!

Washing instructions for the gloves

The gloves are washable by hand using lukewarm water and mild detergent.

Definitely remove the electrodes before washing!

The gloves can be reordered separately.

Contact

Global Technologies

10388 W. State Road 84, Suite 111,
Davie, Florida USA 33324-4252

Toll-Free: 1-888-437-3900

Intern.: +001-954-746-4270

Fax: 1-954-252-2181

sales@GTsimulators.com

www.GTsimulators.com
