eCareYou **enna**<sup>®</sup>  
**MAKE THE RULES**

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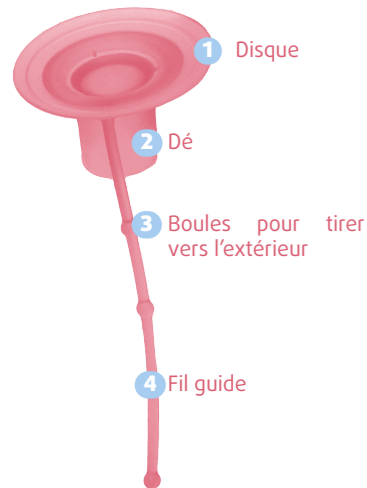
[ennawomen.com/fr/](http://ennawomen.com/fr/)

EN

## HOW TO INCREASE YOUR CHANCES OF PREGNANCY:

A healthy couple can have fertility problems due to environmental factors that affect sperm production and quality in men, while they can also cause alterations in the menstrual cycle and ovulation in women. Therefore, a healthy lifestyle is essential to improving your chances of pregnancy: reduce caffeine intake, avoid tobacco and alcohol, exercise, sleep well, make sure you eat a balanced diet that includes vegetables and cereals and try to reduce stress.

It is also important to have good knowledge of your fertile days (ovulation period). Methods designed to help you better understand this includes basal temperatu-

re control, the Billings method, which is based on vaginal secretions, or ovulation tests.

When it comes to optimal hygiene measures, it is strongly recommended that you maintain healthy vaginal microbiota or flora. The vaginal flora is the set of microorganisms, most of them lactobacilli (90%) that inhabit the vagina to prevent infections and ensure that your vagina does not become an unhealthy environment for getting pregnant. For men, the quantity of semen in the ejaculation and the quantity of spermatozoa are both significant factors. However, the most important thing is quality: good quality ensures viable spermatozoa with sufficient motility (ability to move) to reach the egg and fertilise it.

**enna fertility kit** enhances fertility and increases your chances of pregnancy in a completely natural way without invading a couple's privacy.

**enna fertility cap** is a reusable, soft and flexible medical grade liquid silicone medical device that is biocompatible with the vaginal mucosa. It allows more sperm to reach the cervix by isolating and protecting the sperm from unfavourable conditions in the vagina (acidic pH) and osmolarity (the concentration of salts in the environment).

These features help concentrate sperm in the cervix and stop them from being lost, prolonging their presence and contact inside the vagina.

**enna fertility gel** enhances the viability of the semen and promotes the fertility of the cervical mucus. It is a patented formula made using:

**Myo-inositol:** increases the number of motile spermatozoa and improves the viscosity of cervical mucus to facilitate fertilisation.

**Prebiotic(FOS):** strengthens the vaginal microbiota, creating an environment that is ideal for conception while also preventing dysbiosis.

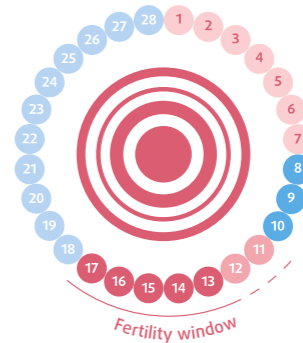
**Ca<sup>+</sup>** Calcium ions: increase sperm motility.

**pH** Provides an ideal pH (7-7.5) for semen.

**+** Provides an optimal salt concentration (osmolarity of 280-350 mOsm/ml) for sperm.

## INSTRUCTIONS FOR USE

Apply 2 ml of **enna fertility gel** inside the vagina using the syringe provided. Do this every night before going to sleep during the 2 days before ovulation and throughout the fertility window.


On these days, have sexual intercourse at least 3 times, separated by approximately 48 hours between each occasion, to facilitate proper sperm production.

After ejaculation, apply another 2 ml of **enna fertility gel** and insert **enna fertility cap** into the vagina.

By doing this, **enna fertility cap** will help the sperm to travel through the vagina until it reaches the seminal sac and the cervix. **enna fertility gel** will promote semen viability and sperm motility.

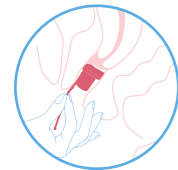Once you've inserted it, wait for about 20 minutes. **enna fertility cap** will concentrate sperm close to the cervix, in a favourable pH environment and ensure optimal salt concentration thanks to **enna fertility gel**.

Once you decide to remove it, sit in the bath and pull the guide thread. It has been designed for easy extraction by pulling the beads. You'll find that the **enna fertility cap** is easily released. It is normal for semen fluid, cervical mucus and intimate fluids to come out when removing the **enna fertility cap**.

Rinse the **enna fertility cap** with water and wash with enna period gel. Dry it well and preferably store it in a cotton bag for the next use.

Sterilise the **enna fertility cap** before the first use by boiling for 3 minutes. Clean the **enna fertility cap** with water and our enna period gel intimate hygiene soap after each use while you're ovulating. Once the fertility window is over, sterilise the **enna fertility cap** and store it until your next cycle.


You can use our ecological and hypoallergenic liner pads, **enna protect**, to absorb any fluids.

## RECOMMENDATIONS

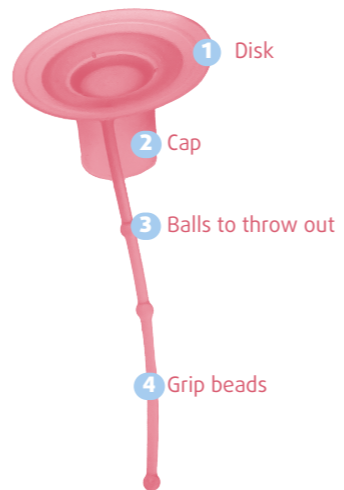- If you have a feeling of irritation and the symptoms continue, you should consult your doctor.
- Keep out of the reach of children.
- Keep in a dry and well-ventilated place.
- Once opened, keep in the fridge.
- Do not use **enna fertility gel** once it has expired.
- Do not use it if the container is damaged or if the cap is broken.
- **enna fertility kit** is designed to enhance fertilisation, but does not guarantee you'll get pregnant. **enna fertility kit** helps to increase the presence of spermatozoa near the cervix. However, there are other factors that affect fertilisation which are not influenced by enna fertility.

- If you have used enna fertility for one year and still haven't been able to get pregnant, consult your doctor.
- **enna fertility gel** is a product for topical use, do not swallow it. In case of accidental consumption. Please seek medical advice.
- **enna fertility gel** can be used without the enna fertility cup, as it also works as an intimate moisturiser and protector of the vaginal microbiota. Similarly, the **enna fertility cap** can be used without the gel, but it is better to use the two products together.

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