

EXPERIENCE THE



PACEY MEDTECH™


PACEY CUFF

**AN INNOVATIVE DEVICE TO HELP WITH YOUR
URINARY INCONTINENCE**

CUSHIONED TOP
PROTECTS CIRCULATION

VELCRO STRAP
EASY ADJUSTMENT

**ADJUSTABLE
BOTTOM**
*FOR EFFECTIVE
CUSTOM FIT*

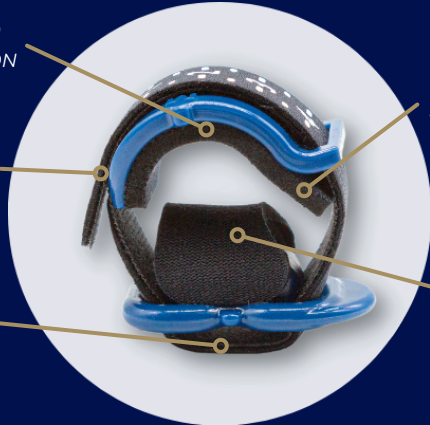

**ENHANCED
ANTI-PINCH TAB**
ALL DAY COMFORT

**URETHRAL
PRESSURE PAD**
TO STOP LEAKS

**STOP URINE
LEAKING**

ALL DAY USE

**NO BLOCKED
CIRCULATION**

FDA

Listed

Listing # 3013731289

GSA

Schedule

Listing # 36F79718D0423

USE PACEY CUFF IN 3 EASY STEPS

STEP 1



Apply the Power Sleeve
(included with each Cuff)

STEP 2



Apply the Pacey Cuff

STEP 3



Secure the Pacey Cuff

PACEY CUFF SIZING

FIND YOUR PACEY CUFF SIZE

ENSURE YOU HAVE THE LENGTH REQUIRED

Circumference

Measure mid-shaft around a non-erect (flaccid) penis.

Size	Inches	Centimeters
S	1 - 2.75"	3 - 7 cm
M	2.75 - 4"	7 - 10 cm
L	4 - 5.5"	10 - 14 cm

Penis Length

Measure from the base to tip of non-erect penis

Size	Inches	Centimeters
S	1.25"	3 cm
M	1.75"	4 cm
L	2"	4.5 cm

* if between two sizes choose larger size.



use strip to measure circumference

To order or ask questions
phone 1-877-377-2239 or visit paceycuff.com