# PACEY CUFF™

TURBO

URETHRAL CONTROL DEVICE

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## INDICATIONS FOR USE

The Pacey Cuff™ is indicated for provision of male urinary continence. The device may be used during waking hours as required. A light pad is recommended for use with the device.

## INSTRUCTIONS FOR USE

**NOTE:** Compression and tightening of the Velcro® strap may need to be attempted a few times to determine the correct compression needed to occlude the urethra.

If leaking occurs after first application:

1. Check the bottom strap setting (the Pacey Cuff should arrive set at around 2-3 on the bot-tom).
2. Slide the bottom strap to a higher setting (4-5).

**Recommended:** The use of brief-style undergarments (form-fitting) rather than looser boxer shorts

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
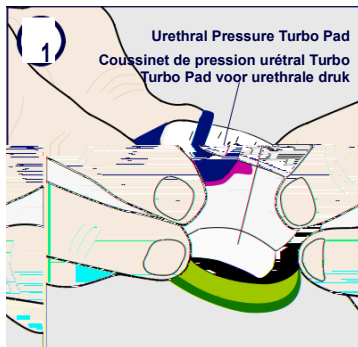
1. Thread the Velcro® strap through the guide on the top hood.

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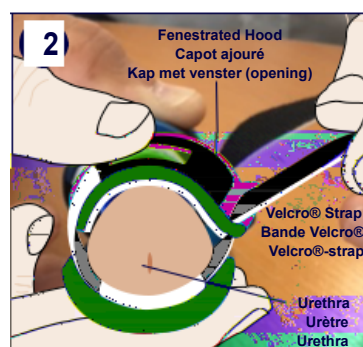
2. Slide over penis to rest mid-shaft. Press top hood and bottom urethral compressor together with your dominant hand. Ensure the top hood and bottom compressor are centered.

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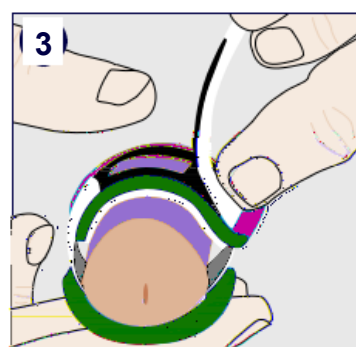3. While maintaining compression, pull up on the Velcro® strap to tighten. Attach strap to the top hood and note the number.

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4. Secure the remaining Velcro®. Ensure top hood and bottom compressor are centered on the penis. Inspect and treat any skin pinching that may have occurred between upper and lower parts.

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