harex

USER MANUAL / MANUAL DE USO /
MANUEL D'UTILISATION / GEBRAUCHSANWEISUNG

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Mera Medical Solutions S.L.

COMFORT

The HAREX Urinary Ring is made with sanitary-grade lightweight high quality plastics, technologically tested to avoid liquid absorption. The Device measurements are 48 x 54 x 18 mm so in addition to being safe, it is small in size and is very comfortable. Once in place neither you nor anyone else will notice that you are wearing it. The HAREX Device is fully adjustable, you just have to move the adjustment lever to adapt it to the pressure required.

ALLOWS BLOOD FLOW

The device does not press on the sides of the penis and therefore can be for several hours, however it is recommended to remove the HAREX Device at night while sleeping, to rest the penis. During the time spent lying down a protection mat can be used instead. If it is necessary to use the Device during the time you are lying down, loosen the strap with a lighter pressure than you would use during the day.

BOX FOR STORING DEVICE

The set contains a box for transporting and / or storing the device. It is made of plastic so it can be washed hygienically if required.

EASY CLEANING

The Device is easily cleaned by hand with warm water and soap. It is manufactured entirely in sanitary-grade plastic material. Given that HAREX has no metallic elements, it can pass without problem through metal detection systems, such as those used at airports, train stations, etc.

HELPFUL TIPS

If the natural urge to empty the bladder is not perceived, it is advisable to release the Device every 2 or 3 hours to avoid excessive filling of the bladder.

Placing the urethral cushion: When placing the device, make sure that the pressure cushion is in the correct position pressing on the urethra, and that it is not misplaced. This way the urethra will remain completely obstructed to avoid urine leakage.

Circumcised patients: Use the Device directly behind the head of the penis. If you are a little overweight, or are not circumcised, before placing the Device pull back the foreskin as this will prevent the Device from moving due to the pressure at the base of the penis.

Belt numbering: the adjustment belt has measurements added so that you have your own personal reference for the perfect fit. This way you can place the device at the optimum position for you.

PLACEMENT OF THE DEVICE

- 1 Move the belt adjustment lever (B) down and loosen the belt.
- 2 Insert the penis into the Device.
- 3 Adjust the belt (C) to the contour of the penis.
- 4 Move the lever (B) up to fix the strap (C).
- 5 Move the pressure activator (A) up, to block the urethra.

Keep the genital area clean to avoid irritation.

If you are aware that the Device is misplaced by either uncomfortable pressure or mild pain, it is a sign that the Device is too tight. Loosen the belt a little. Check that there are no folds in the skin of the penis as this could cause discomfort. With the right fit it should feel comfortable.

DEVICE REMOVAL

- 6 Move the pressure activator downwards (A)
- 7 Release the adjustment lever (B) and loosen the belt so that there is enough space to remove the penis from the Device.

TO URINATE

- 8 Move the urethra pressure activator (A) downwards. The device will release the pressure and you can start to urinate.
- 9 Once finished, release the belt and move the pressure activator (A) to the initial position.

The belt is designed to fit over the penis by applying light pressure on the urethra to stop the flow of urine. In case of incorrect placement, it may cause pain, be sure to follow the instructions indicated. If in doubt, please call us or send us an email and we will be happy to help you. We have the answers to all your questions.

TERMS AND CONDITIONS

You are using the Device at your own risk and responsibility. If you have a medical problem other than incontinence, consult your doctor before using the Device. If you experience problems you should stop using it immediately and notify your doctor.

WARRANTY

The manufacturer's guarantees against defects and breakages and is valid for 1 year from the date of purchase. Guarantee claims must be previously approved. The purchase receipt will be required for any type of claim. For more information, we invite you to visit our website at: www.harex.net

It is recommended to replace the Device after 12 months of use.

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Versión 06-2022

EN **ATTENTION**
Before using the Device, read the instructions carefully.

Urinary incontinence affects millions of men around the world. Some experience a total loss of urine control, while others may have a continuous involuntary dripping of urine. The Urinary Device is the best solution for an active and healthy life.

HAREX is a Device that is placed just below the head of the penis and by means of a gentle pressure on its base, it prevents and controls the involuntary leakage of urine. It is small, comfortable and fully adjustable. It is built with washable materials, and most importantly, it improves quality of life and recovers day to day normality as it relieves the person of the condition.

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