



Jar & Bottle Opener



Usage and Maintenance Instructions

BEFORE USE

Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may damage the surface of the product.

Check the product for any visible damage. If you see any damage or suspect a fault, do not use your product, but contact your supplier for support.

INTENDED USE

A two in one Jar & Bottle Opener; enables the user to easily open jar lids and bottle caps.

INSTRUCTIONS FOR USE

The Jar & Bottle Opener is simple to use:

- **To Open Jars:** Grip the Jar & Bottle Opener with the ergonomically-designed handle and place the product over the jar lid. Position the Jar & Bottle Opener so that it grips the jar lid and pull/push the handle to twist the jar lid off the jar.
- **To Remove Bottle Caps:** Grip the Jar & Bottle Opener with the ergonomically-designed handle and place the product over the jar lid. Position the Jar & Bottle Opener so that the central hole and metal lip are above/below the edge of the bottle tops. Pull the handle of the Jar & Bottle Opener up to flip the cap off the bottle.

CLEANING

Clean your Jar & Bottle Opener using a non-abrasive cleaner or mild detergent with a soft cloth. Abrasive cleaners e.g. AJAX and/ or abrasive cleaning pads may damage the product beyond repair and should not be used. Always ensure that you wipe dry the equipment after cleaning.

The Jar & Bottle Opener can also be cleaned on the top tray of a domestic dishwasher on a standard/eco or cool program.

REISSUE

If you reissue or are about to reissue this product, please thoroughly check all components for their safety.

If in any doubt, please do not issue or use, but immediately contact your supplier for service support.

CARE & MAINTENANCE

Please make a safety check of the product at regular intervals or if you have any concerns.