



Beaumont Leg Rest w Castors

BEFORE USE

Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may damage the surface of the item.

Check the product for any visible damage. If you see any damage or suspect a fault, do not use your product, but contact your supplier for support.

HEIGHT ADJUSTMENT

Remove the 'E' clip from the extension leg and extend all legs evenly to the desired height ensuring you do not exceed the last punched hole of the extension leg. (see fig. 1)

Once you have achieved the desired height; reinstall the 'E' clip, ensuring that it has passed through the extension leg and is fitting neatly and snug around the outside leg. (see fig. 2)

Never install the extension legs unevenly or slanting, this will compromise the safety of the user.

CLEANING

It is important that you only clean the upholstery using a non-abrasive cleaner or mild detergent with a soft cloth. Abrasive cleaners e.g. AJAX and/ or abrasive cleaning pads will seriously damage the upholstery beyond repair. Always ensure that you wipe dry the equipment after cleaning.

However the products in this group can be treated with formaldehyde if required, but it is important that this treatment be carried out by an approved contractor or an experienced person and under suitable conditions.

CARE & MAINTENANCE

Please make a safety check of the product at regular intervals or if you have any concerns.

REISSUE

If you reissue or are about to reissue this product, please thoroughly check all components for their safety.

This includes:

- Tightness of all nuts/bolts/castors and other screw in/bolt in/push in components.
- Also check all upholstery (if applicable) for security, splits, exposed staples, etc.

