

Raised Toilet Seat



CE

Usage and Maintenance Instructions

BEFORE USE

- Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may damage the surface of the product.
- Check the product for any visible damage. If you see any damage or suspect a fault, do not use your product, but contact your supplier for support.

INTENDED USE

The Viscount Raised Toilet Seat is available in three sizes - 2",4" and 6" with or without an integral lid, and is designed to fit the majority of UK toilet bowl shapes.

SAFETY

- Consult with your physician or supplier to determine proper adjustment and use of the device.
- Do not sit or step on the lid.
- The Viscount Raised Toilet Seat is intended for indoor use only.
- Carefully check all screws are tightened before use.
- Review weight limit and weight load before use. Do not use exceed the weight limit.
- Do not make any modifications on this device.
- Always check stability before use.

CLEANING

Clean your Viscount Raised Toilet Seat using a non-abrasive cleaner or mild detergent with a soft cloth. Abrasive cleaners may damage the product beyond repair and should not be used. Always ensure the toilet seat is wiped dry after cleaning.

If to be disinfected by heat, one of the following three temperatures and duration of exposition can be used:

- a) a temperature of 90°C for 1 minute
- b) a temperature of 85°C for 3 minutes
- c) a temperature of 80°C for 10 minutes

INSTALLATION

- 1 – Ensure you inspect the product for any visual damage before use.
- 2 – Carefully remove the existing toilet seat and lid from the toilet.



fig.1

3 – Unscrew the hand wheels of the raised toilet seat (see fig. 1).



fig.2



fig.3

4 – Carefully place the seat onto the toilet bowl (see fig. 2). Position the raised toilet seat correctly on the rim of the toilet ensuring the front lip is outside the bowl. (see fig. 3). To tighten, turn the hand wheels clockwise to tighten.

5 – Check the stability before use.

6 – To remove, simply unscrew the hand wheels and remove the raised toilet seat by lifting up.

CARE & MAINTENANCE

Please make a safety check of the product at regular intervals or if you have any concerns.

REISSUE

If you reissue or are about to reissue this product, please thoroughly check all components for their safety.

Usage and Maintenance Instructions

If in any doubt, please do not issue or use, but immediately contact your supplier for service support.

Packaged dimensions (HxWxL mm)	410x370x200
Display Footprint (mm)	410x370x140
Transport dimensions (mm)	410x370x140
Gross Weight (Packaged) (kg)	1.75
Packaging	
Item priced per	1
Inner box quantity	1
Outer box (HxWxL mm)	790x430x390
Outer box quantity	4
Gross weight outer carton (kg)	9
Amazon Asin	B07HWW5XF5
Warranty Period	12 Months
Gross Weight (kg)	1.75